syngo.share / Release VA25B / 2018-02-26 / Revision 2690

syngo.share webview

User Manual

Distributed by:

SIEMENS

Language Directory

Dansk	3
Deutsch	16
Ελληνικά	27
English	39
Español	52
Français	63
Magyar	75
Italiano	86
Nederlands	98
Norsk (nynorsk)	109
Polski	122
Português	133
Türkçe	144


syngo.share / Release VA25B / 2018-02-26

syngo.share webview

Brugermanual

Distributed by:

SIEMENS

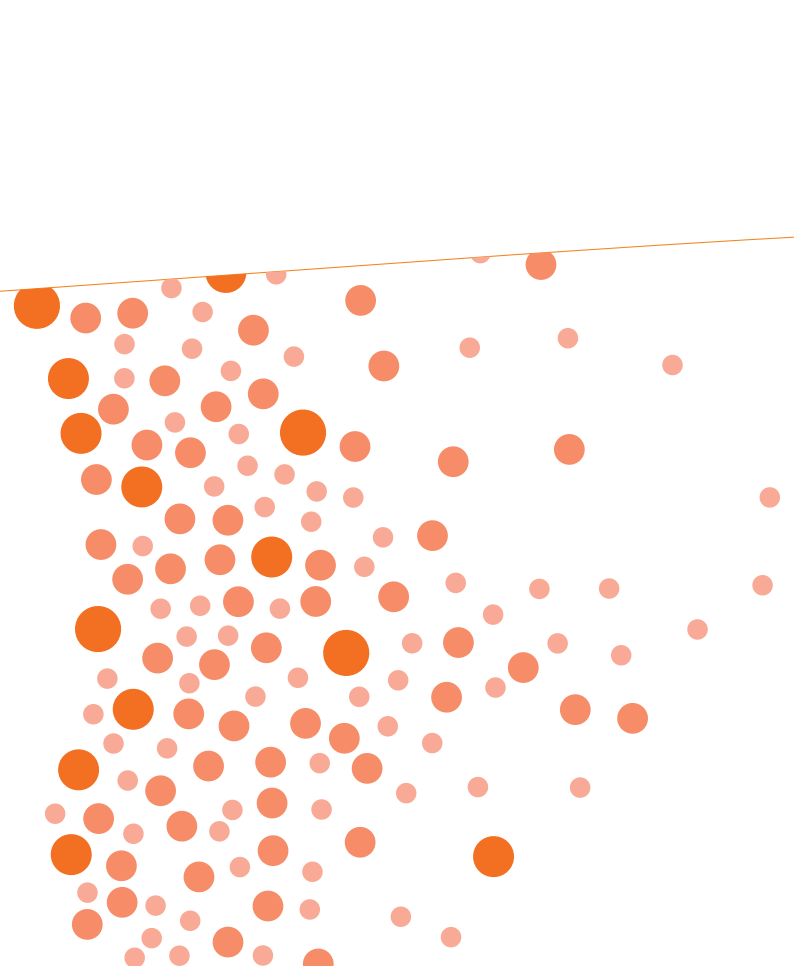Fordelene ved syngo.share webview

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ITH icoserve technology for healthcare GmbH
Innrain 98
A-6020 Innsbruck
Austria

2018

Distributed by

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens.com/healthcare

Legal Manufacturer

ITH icoserve technology for healthcare GmbH
Innrain 98
6020 Innsbruck
Austria
Phone: +43 512 89059

syngo.share / Release VA25B / 2018-02-26

syngo.share webview

User Manual

Distributed by:

SIEMENS

Benefits of *syngo.share* webview

- Consistent, company-wide archive for all clinical pictures, multimedia, and documents.
- A flexible mechanism for the acquisition of all your clinical data (DICOM-/HL7-interfaces, import of data, video recordings, print interfaces, etc.).
- Uniform software for viewing the most common clinical data formats. Special framework conditions can be adjusted (display, lighting conditions) to facilitate diagnoses. According to the working environment, a bright or dark background can be enabled.
- Authorization system based on functions and positions
- Continue working while doing long-lasting tasks
- Integration with existing an HIS (Hospital Information System)

Contents

- 1 About *syngo.share* webview 40**
 - 1.1 Intended Use 40
 - 1.2 System Prerequisites 40

- 2 Safety Notices 41**
 - Image and Result Distribution 41
 - Medical Education 41
 - Calculation Deviations 41
 - Units 41
 - Access Restriction 42
 - Effects of External Programs 42
 - Printing of Images 42

- 3 First Steps 42**
 - 3.1 User Security 42
 - 3.2 Keystroke Combination 43
 - 3.3 Search for Patients and Documents 43
 - 3.3.1 Advanced Search with Wildcards 43
 - 3.4 Document Search and Results (*syngo.share* view, *syngo.share* webview) 43
 - 3.5 Keyword Tagging 44
 - 3.6 Export/local Saving 44
 - 3.6.1 Exporting with *syngo.share* view 44
 - 3.6.2 Exporting with *syngo.share* webview 44
 - 3.7 Control State of Actions 45

- 4 Application *syngo.share* webview 45**
 - 4.1 Viewing Documents 45
 - 4.2 Viewing Patients 45
 - 4.2.1 Functions under the Patient Tab 46
 - 4.3 Context Menu for the List of Results 46

- EC Declaration of Conformity *syngo.share* webview 48**

- Australian Conformity *syngo.share* webview 49**

1 About *syngo.share* webview

syngo.share webview is a zero-footprint HTML5 web application to be used in clinical image and result distribution. It provides access to content from *syngo.share* VNA, and integrates with EMR and information systems, providing basic review and annotation support.

1.1 Intended Use

syngo.share webview is software for the electronic display of data, without controlling or altering the functions of any connected medical devices.

1.2 System Prerequisites

syngo.share webview can be run using the following web browsers:

Optimized for: $\geq 1280 \times 1024$

- Google Chrome ≥ 58
- Microsoft Internet Explorer ≥ 11
- Mozilla Firefox ≥ 53

Supported:

- Apple Safari ≥ 10
- Microsoft Edge

On mobile operating systems such as:

- Android
- Apple iOS
- Microsoft Windows 10

☰ Touch gestures

syngo.share webview supports touch gestures, but is not optimized for them.

☰ Video streaming in Microsoft Internet Explorer

This functionality is available in Microsoft Internet Explorer ≥ 11 from Microsoft Windows ≥ 8 onwards.

2 Safety Notices

Image and Result Distribution

syngo.share webview is only intended for clinical image and result distribution.

Do not use *syngo.share* webview for direct diagnoses.

Medical Education

The use of *syngo.share* webview should be granted to qualified staff members only.

Calculation Deviations

Any calculations and calibrations not made in *syngo.share* webview will have a level of precision that correlates to the precision of the program of that in which they were created—not of *syngo.share* webview. Additionally, the level of precision is only as precise as the data collected by the modality used. This means that each time a modality is connected, it should be calibrated. The calibration of the device and interpretation of calculations used in a diagnosis must be reviewed by qualified staff members for plausibility. Deviations may be caused by the image itself—in particular for conventional X-ray scans (CR) e.g. pelvis or hips. The representation of the distance between the part of the body being recorded and the imaging plate could deform the length on a two-dimensional X-ray image representation. Even if there is a point of reference included in the scan (e.g. a ball), there will always be a deviation if the point of reference is not positioned precisely at the height of the body part being calculated at the time of the scan. The area of polygons whose edges overlap will not be calculated correctly in each case. In addition, the extrapolation algorithm used in *syngo.share* webview for displaying images can occasionally lead to imprecise rendering. Calculations of such images could be inaccurate.

The following defines how calculations will be rounded:

- › Length and surface area: to the nearest tenth
- › Density (HU): to the nearest whole number

Units

The application *syngo.share* webview conforms to Directive 80/181/EEC, which contains specifications related to units of calculation. Calibrated lengths are to be calculated using millimeters. Surface areas

are to be calculated in square millimeters (mm²). Plane angles are to be calculated in degrees. Densities will be calculated using the industry-standard unit, Hounsfield units (HU), which are not regulated by Directive 80/181/EEC.

Access Restriction

Be sure to log out of *syngo.share* webview when finished working in order to avoid unauthorized access to the system.

Effects of External Programs

The software manufacturer does not ensure that third-party applications will not negatively affect the system. Any external programs accessed by *syngo.share* webview have standards which correspond to their respective producers.

Printing of Images

When printing images using the image caption option “As camera”, it is possible that some labels will be hidden from view due to the physically limited space on the page. As a result, possible transformations such as mirroring, rotation, and applied presentation states etc.; and even patient information may not be visible on the printout. If a series has active presentation states in both the view and print layouts, deactivation of the presentation states in the view layout will also deactivate it in the print layout of the series.

3 First Steps

This chapter contains the most important features of *syngo.share* webview as well as operational instructions.

3.1 User Security

A valid username and password is required by *syngo.share* webview, and will be assigned by a system administrator at your institution. The allocated roles and permissions determine which features will be

enabled for a given user. An inactive application will be automatically locked to ensure that there is no unauthorized access. To continue working where you left off, simply log in again and the program will resume at the point just before the automatic lock was activated.

3.2 Keystroke Combination

You can use keystroke combinations to initiate functions. The shortcuts are shown in the context menu or in the tool tip of the particular function. A complete list of all keystroke combinations can be found in the online help manual.

3.3 Search for Patients and Documents

To begin a search, insert the appropriate search criteria into the search field and press the "Search" button. If your search criteria is insufficient, the "Search" button will remain inactive. To remove all search criteria quickly, press the "Reset" button. The fields of the search mask can be defined at the applications view and *syngo.share* webview by using the button "Configure search fields" and by (de)activating the needed search fields. This setting will be saved by the programs. The use of capital and small initial letters won't be regarded.

3.3.1 Advanced Search with Wildcards

The following characters are used to perform advanced searches with wildcards in the search interface:

- The sign "%" is a variable for one or more characters
- The underscore "_" is a variable for exactly one optional character

Example 1: Search Using Placeholders (wildcards)

"m%er" would result in: "Meier", "Mayer", "Moser", "Mariacher", etc.

"m_ier" would result in: "Meier", "Maier", etc.

3.4 Document Search and Results (*syngo.share* view, *syngo.share* webview)

The search results columns can be arbitrarily ordered by simply dragging them with the mouse pointer. The order you chose will be saved in your user profile and used in future searches.

To sort a column, click on the column header.

- In the web application, you can click on the arrow in the column header to show or hide columns. These settings can be configured in the context menu of the application *syngo.share* view. Your settings will be saved and available the next time you launch the program.

- The search results can be exported as a CSV file in the web applications *syngo.share* webview and *webadmin*.

To open a search result, double click it in the list of results:

- In the application *syngo.share* view, the patient record will be loaded into the document tree.
- In the applications *syngo.share* webview and *webbox*, the selected item will be shown in the viewer.

3.5 Keyword Tagging

In the applications *syngo.share* view, *syngo.share* print, *syngo.share* import, and *syngo.share* webview you can annotate documents. Special authorization from an administrator is needed for this functionality. Tags add another layer of description to documents such that they can be easily found in the future. A configuration determines whether the tags can be chosen from a catalog, dynamically added to a catalog, or entered as free text (e.g. comments). The following DICOM levels support keyword tags: studies, series, and images. Generic containers also support tagging. While searching using keywords, the results are shown independently of the level in which the keyword tag was added.

For further details for tagging read the online help.

3.6 Export/local Saving

In the applications *syngo.share* view, *syngo.share* webview, and *webbox*, there are export functions that allow the data to be saved locally. To export data, two different levels of permission are possible:

- Anonymous-only export
- Full permissions export

3.6.1 Exporting with *syngo.share* view

Exporting of studies and series in the document tree:

- Click "Export all series" button to export every file.
- Select the desired items and export them by using the button "Export selected series".

Exporting of individual images in the print area:

1. Drag the requested element out of the document tree and into the print area.
2. Click the "Export..." button.

3.6.2 Exporting with *syngo.share* webview

Exporting a selection in *syngo.share* webview:

1. Click on either "Export..." or "Export as..." in the context menu.
2. The export wizard will open where you can configure your export.

3.7 Control State of Actions

All desktop applications have a status area, which contain

- If the requested action was successful the state will show a green check mark.
- An exclamation point indicates that the action could not be completed successfully. Click on the status for detailed information.

4 Application *syngo.share* webview

The application *syngo.share* webview is similar to *syngo.share* view, except that runs in a web browser. You can view medical documents (e.g. radiological images, clinical report, patient data) and change the appearance or perspective using built-in tools (e.g. zoom, rotate, brightness, etc.). You can use *syngo.share* webview to show documents to patients directly as well as carry out administrative work.

Examples: Send, share, and tag documents with keywords.

4.1 Viewing Documents

In order to view examinations in the viewer, do one of the following:

- Double-click on an examination in the list
- Click the `View document` button.
- Right-click on the item and click `View document`

The “selection list” tab displays all available documents in the archive for a patient. More documents can be displayed in layouts next to or above.

4.2 Viewing Patients

A tab is opened for the desired patient after you click on the `View Patient` button or `View Patient` from the context menu. Under this tab, you can find view filters on the left side:

- Patient records.
- Patient visits.

4.2.1 Functions under the Patient Tab

- Reset view filters

You can remove the filtration of files and visits.

- Download patient documents

You can export anonymous data as an archive file or compile it as a single PDF file.

- Show patient data

You can view patient information and additional data as well as related people such as the closest relatives or insured.

- Show patient URL

This provides a patient-URL for the active patient record that can be opened in an internet browser.

- Show upload dialog

To use this function, you will need special authorization to associate locally saved documents with the active patient record. The "upload data" window will open and the patient context will be preselected automatically for you.

4.3 Context Menu for the List of Results

- View document

Open a document in the viewer or provide a download link if the document type is not supported.

- View patient

Open a tab for the selected patient with access to:

- View filters
- Other documents & DICOM studies
- DICOM series
- Image view and options

- External search

This allows you to quickly access *syngo.share* view and *syngo.share* import:

- ↳ Lookup with patient ID

Open *syngo.share* view with selected patient ID.

- ↳ Lookup with visit ID

Open *syngo.share* view with selected visit ID.

- ↳ Acquire using patient ID

Open *syngo.share* import with selected patient ID.

- ↳ Acquire using visit ID

Open *syngo.share* import with selected visit ID.

- Delete

Delete the document.

- Send

Send document to an AET configured as a recipient in the application *webadmin*.

Register reference pointer

Make a document available to subsystems. You can manually register documents which are available to *syngo.share* webview. In the background, a reference pointer is sent to the subsystem that contains information about the document.

Share

Share documents over the application *webbox* with colleagues and local physicians.

Show URL

For direct access to an item in the browser, you can retrieve a URL.

Export...

Export as...

Enables you to save documents locally.

Open locally...

If the application *syngo.share* view is installed locally, you can open the document in the local viewer.

Add tags...

You can apply keyword tags to documents.

Edit tags...

You can remove keyword tags from documents.

Copy...

You can copy a document into another repository.

Move...

You can relocate a document into another repository.

Reassign...

Misplaced documents can be reassigned to another patient or visit.

Change document properties...

You can change the description or date of an examination for a document.

Note

User roles and their associated permissions assigned by the system administrator determine which functions are accessible.

EC Declaration of Conformity

Product Details

Product Name	syngo.share webview
Type	VA25B
Model Number	11250006
Classification according to Directive 93/42/EEC, Annex IX	Class I

Manufacturer

Address	ITH icoserve technology for healthcare GmbH Innrain 98 6020 Innsbruck Austria Phone: +43 512 89059
---------	--

Assessment Details

Procedure according to Directive 93/42/EEC	Annex VII
--	-----------

Applied Standards

EN ISO 13485:2012 / AC:2012
EN ISO 9001:2008/AC:2009
EN ISO 14971:2012
EN 62304:2006/AC:2008
EN 62366:2008

We declare under sole responsibility that the products described above are in compliance with Directive 93/42/EEC, if their installation and use occurs corresponding to the user and product documentation. This declaration supersedes any declaration previously issued for this product.

Innsbruck, 2018-02-26

Dr. Bernhard Hirsch
Managing Director

Mag. (FH) Herbert Plankl
Head of Quality & Documentation

EC Declaration of Conformity

Manufacturer's Declaration of Conformity
Australian Therapeutic Goods (Medical Devices) Regulations 2002
Full Quality Assurance Procedure

This is a declaration made in accordance with the requirements of Clause 1.8 of Schedule 3 of the **Australian Therapeutic Goods (Medical Devices) Regulations 2002** relating to the stated device.

Reference

syngo.share webview

Legal Manufacturer Business Address

ITH icoserve technology for healthcare GmbH
Innrain 98
6020 Innsbruck
Austria
Phone: +43 512 89059

Facility

ITH icoserve technology for healthcare GmbH
Innrain 98
6020 Innsbruck
Austria
Phone: +43 512 89059

Medical Device(s)

syngo.share webview VA25B
11250006

Authorised Signatory

2018-02-26

2018-02-26

Dr. Bernhard Hirsch
Managing Director

Mag. (FH) Herbert Plankl
Head of Quality & Documentation

Declaration of Conformity AU

☐ Classification (per TGA)

Class I

(Schedule 2, Part 4, Rule 4.1 of the Therapeutic Goods (Medical Devices) Regulations 2002)

☐ GMDN Code and Term

41670: Radiology picture archiving and communication system application software

☐ Scope of Application

This declaration of conformity covers *syngo.share* webview:

Model Number: 11250006

Product Description: *syngo.share* webview VA25B

Each kind of medical device to which the Full Quality Assurance Procedures have been applied complies with the applicable provisions of the essential principles, the classification rules, at each stage, from the design of the device until its final inspection before being supplied.

Full Quality Management System Certificate

EN ISO 13485:2012

Certificate No.: 17062224

Standards Applied

(European Harmonised Standard)

- EN ISO 9001:2008, AC:2009
- EN ISO 13485:2012, AC:2012
- EN ISO 14971:2012
- EN 62304:2006, AC:2008
- EN 62366:2008

ITH icoserve technology for healthcare GmbH
Innrain 98
A-6020 Innsbruck
Austria

2018

Distributed by

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens.com/healthcare

Legal Manufacturer

ITH icoserve technology for healthcare GmbH
Innrain 98
6020 Innsbruck
Austria
Phone: +43 512 89059

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