syngo.share / Release VA30C / 2021-12-16 / Revision 6272

syngo.share import

Online Help

Language Directory

Deutsch	3
English	41
Français	77

syngo.share / Release VA30C / 2021-12-16 / Revision 6272

syngo.share import

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Online Help (English)

Contents

1	Introduction	44
1.1	Intended Use	44
1.2	Technical Terms and Notation	44
1.2.1	Generic Document	44
1.2.2	DICOM Document	44
1.2.3	DICOM Information	44
1.2.4	EXIF Information	45
1.2.5	Modality	45
1.2.6	Organizational Unit	45
1.3	Safety Notices	46
	Data Import	46
	Medical Education	46
	Accuracy of Calculations	46
	Units	46
	Access Restriction	46
	Effects of External Programs	47
2	Settings	48
2.1	General	48
2.1.1	Program-specific Options	48
2.1.2	Enabled User Interfaces	49
2.2	Specific	49
2.2.1	Data Sources	49
2.2.2	Context Selection	49
2.2.3	Context Set by URL or Worklist	49
2.2.4	Special	49
2.2.5	Patient Search	50
2.2.6	File Modifications	50
2.2.7	Barcode	50
2.2.8	Tooth Status	51
2.2.9	Filesystem	51
2.2.10	Video	51
2.2.11	Still Image Capture	52
2.2.12	Quick Import	52
2.2.13	Zero-Click Import	52
2.2.14	Windowing Default Settings	52
2.2.15	Viewer	52
2.3	Communication	53
2.3.1	<i>syngo.share</i> core	53
2.3.2	DICOM	53
2.3.3	HL7	53
2.3.4	Worklist	53
2.3.5	Order List	53
2.3.6	IRWF-specific Options	53
2.4	Video Postprocessing	53
2.4.1	External Video Postprocessing	53
2.4.2	External Video Transcoding	54
2.5	DICOM Import Profiles	54
2.5.1	Default Rules	54
2.5.2	Additional Rules	54

2.6	Personal Settings Profile	55
3	Usage Instructions	56
3.1	Getting Started	56
3.1.1	User Interface	56
3.1.1.1	Main Menu	56
3.1.1.2	Main Area	57
3.1.1.3	Document List	57
3.1.2	Login and Logout	57
3.1.3	Changing the Password	58
3.2	Setting a Context	58
3.3	Adding Documents	61
3.3.1	File System	61
3.3.1.1	File Browser	61
3.3.1.2	Dragging and Dropping Files from Windows Explorer	62
3.3.2	Archive	62
3.3.2.1	Types of Search Fields	63
3.3.2.2	Saving a Search Query	65
3.3.2.3	Results List	65
3.3.2.4	Series Area	66
3.3.2.5	Editing Files from <i>syngo.share core</i>	68
3.3.3	Video	68
3.3.4	Scan	69
3.3.5	DICOM	69
3.3.6	Tooth Status	69
3.3.7	Camera	70
3.4	Viewing and Editing Documents	70
3.4.1	Viewing Documents	70
3.4.2	Editing Documents	71
3.5	Tagging Documents with Keywords	73
3.6	Changing DICOM Elements	74
3.6.1	Editing the Context	74
3.6.2	Creating New UIDs	74
3.6.3	Applying DICOM Import Profiles	74
3.7	Archiving Documents	74
3.8	Keyboard Shortcuts <i>syngo.share import</i>	76

1 Introduction

1.1 Intended Use

syngo.share import is software for the electronic conversion and transfer of data and provides the ability to import clinical data from various sources (such as from local filesystems, DICOMDIR media, digital video cameras, digital photo cameras, documents, and DICOM scanners) into connected archives (such as Vendor Neutral Archives, Picture Archiving and Communication Systems, DICOM archives). This product is not intended for diagnosis, prevention, monitoring, prediction, prognosis, treatment, cure or alleviation of diseases or other medical conditions.

1.2 Technical Terms and Notation

1.2.1 Generic Document

This term refers to any non-DICOM file that can be viewed in or archived by *syngo.share*. Non-DICOM files include images produced by digital cameras or imaging applications (e.g. JPEG, MPM, TIFF) as well as various text, video, and audio file formats (e.g. PDF, DOC, MPEG4, MP3) produced by various external applications (e.g. Microsoft Office, Adobe Acrobat).

Generic files are wrapped in generic containers, analogous to DICOM studies.

1.2.2 DICOM Document

A DICOM document consists of at least one DICOM study, series and instance.

DICOM documents are produced by medical imaging devices compliant with the DICOM standard. They can be viewed in or archived by *syngo.share*.

Study

A DICOM study contains all the data of a medical examination during which image data is acquired. This includes the actual image data (e.g. X-ray, CT, ultrasound) as well as the patient data and descriptions relevant to the examination (such as the examination date or type). A DICOM study contains 1-n DICOM series.

Series

A DICOM series usually equates to a specific type of data acquired/produced by a particular device or to the position of the patient during acquisition. A series contains 1-n DICOM instances.

Instance

A DICOM object instance (or simply DICOM instance) is the actual data captured by the device, such as an image or waveform object.

1.2.3 DICOM Information

DICOM (Digital Imaging and Communications in Medicine) is a standardized file format used in medical imaging.

In addition to object data (typically image data), the DICOM format contains additional information such as patient name, admission date, device parameters, and physician's name. The DICOM format also contains the information necessary for data exchange between DICOM-capable devices.

1.2.4 EXIF Information

EXIF (Exchangeable Image File Format) is a standardized data format generated by, e.g., digital cameras when taking a picture.

In addition to the image data commonly contained in JPEG or TIFF format, the EXIF format contains additional information describing the image in more detail, such as date and time information or the type of device used for image capture.

1.2.5 Modality

An imaging modality is a device for acquiring medical images such as X-rays or CTs. In *syngo.share* import, modalities are identified by abbreviations such as CR for Computed Radiography (e.g. when searching).

1.2.6 Organizational Unit

An *organizational unit* is an association of arbitrary actors in a medical organization, e.g. a department, medical devices, or groups of people. Organizational units can be customized and expanded with more constituents as needed, and these constituents can be uniformly managed.

All data to be archived is allocated to exactly one organizational unit.

1.3 Safety Notices

Data Import

syngo.share import is only intended for the importing of data from various sources.

Medical Education

The use of *syngo.share* import should be granted to qualified staff members only.

Accuracy of Calculations

Images may contain calibrations. Please note that the accuracy of these calibrations correlate to the precision of the program or modality with which they were created. Calibrations must be checked according to the manufacturer's instructions. Calculations, calibrations and results derived therefrom must be checked for plausibility by clinical professionals. Inaccuracies may arise during image capture by clinical professionals or due to the image capturing technology of a given modality. For instance, X-ray images may contain inaccuracies due to the fact that the x-rayed body part is projected onto a two-dimensional imaging surface. Inaccuracies during visualization may be due to hardware settings.

This product rounds calculations as follows: Length and area: if pixel spacing ≥ 0.1 mm: 1 decimal place, if pixel spacing < 0.1 mm: 2 decimal places; density (HU): no decimal place; density average (HU): 1 decimal place; density area (HU): no decimal place. This product uses an interpolation algorithm for image display. This product may not be able to calculate the area of polygons whose edges overlap; this will be indicated by the value "n/a".

Units

The application *syngo.share* import conforms to Directive 80/181/EEC, which contains specifications related to units of calculation. Calibrated lengths are calculated using millimeters. Surface areas are calculated in square millimeters (mm²). Plane angles are calculated in degrees. Densities are calculated using the industry-standard unit, Hounsfield units (HU), which are not regulated by Directive 80/181/EEC.

Access Restriction

Be sure to log out of *syngo.share import* when finished working in order to avoid unauthorized access to the system.


Effects of External Programs

The software manufacturer does not guarantee that third-party applications will not negatively affect the system. The standards of any external programs accessed by *syngo.share import* are set by their respective manufacturers.

2 Settings

The *syngo.share* import installation comes with a set of standard settings. You can find the configuration menu by clicking on  in the main menu.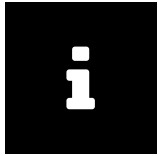

Note


These usage instructions provide a comprehensive description of the functions of *syngo.share* import. Due to the configuration settings of your organization or missing permissions, some of these functions may not be available to you. If you have any questions, please contact your system administrator.

You can either browse the settings by clicking on the individual categories or use the search function to quickly find particular entries. You can reset/undo your search by clicking the  in the search field.

A star next to a list category means that the settings in this section have changed. To save your changes, click .

You may have to restart the application in order to implement some settings. In these cases, you can either agree to an immediate restart or continue working with the previous settings (the changes become effective at the next program start).

Your configuration adjustments are saved on the server and loaded from there when you log in to *syngo.share* import. If the server cannot be contacted, the last locally saved configuration is used.

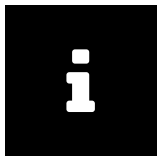Note

In the event that local settings cannot be transmitted to the server (e.g. because it is temporarily unavailable), they may be overwritten when you next start *syngo.share* import. This occurs when the server is reached during start-up and the last known configuration is loaded from there.

2.1 General

2.1.1 Program-specific Options

Here you can configure the presentation and general behavior of *syngo.share* import, e.g. the language of the user interface, the default fullscreen display, or a particular color scheme. The entries in the configuration menu are largely self-explanatory, but some have certain special features:

Language

Selecting the entry  applies the language settings of the Windows operating system.

Minimize to taskbar

This option puts *syngo.share* import in sleep mode upon closing the program, which shortens its startup time. The application icon remains visible in the Windows taskbar.

Automatic login

The Windows single sign-on function logs the user in to *syngo.share* import with their Windows login credentials. If this option has not been enabled by the system administrator, you can still log in manually with your *syngo.share* import credentials. It is also possible to use Kerberos for automatic login.

2.1.2 Enabled User Interfaces

This section allows you to determine which areas of *syngo.share* import are available in the main menu. You can also (un)hide the patient QR code with [Show QR Code controls](#).

2.2 Specific

This section contains settings that affect context and data acquisition:

- [Data Sources](#)
- [Context Selection](#)
- [Context Set by URL or Worklist](#)
- [Special](#)
- [Patient Search](#)
- [File Modifications](#)
- [Barcode](#)
- [Tooth Status](#)
- [Filesystem](#)
- [Video](#)
- [Still Image Capture](#)
- [Quick Import](#)
- [Zero-Click Import](#)
- [Windowing Default Settings](#)
- [Viewer](#)

2.2.1 Data Sources

Here you can toggle which means of data acquisition are available in the **Archiving** area.

2.2.2 Context Selection

The different methods for setting a context can be (de)activated here and are then (in)visible in the **Archiving** area.

2.2.3 Context Set by URL or Worklist

This section lets you determine which aspects of the patient or study context (e.g. the study date) can be edited after they have been set by worklists or URL parameters.

2.2.4 Special

Activate **Combine study description and body part examined** to add the information contained in the DICOM tag *BodyPartExamined* to the DICOM tag *StudyDescription*.

Furthermore, you can set the number of days allowed for an examination date to be in the past before a warning message is displayed.

2.2.5 Patient Search

Here you can customize the quick search available in the header bar of the **Archiving** area. You can expand or narrow the search by selecting a particular search mode as well as define a string which should be entered in the quick search field by default. The option `HL7-managed patients and visits` restricts the search results to HL7-managed patients and visits.

2.2.6 File Modifications

In this section, you can adjust file conversion settings.

☰ DICOM-wrapping

DICOM files are always stored in DICOM format. Generic files like PDFs can be converted into DICOM format in *syngo.share* import, a process called DICOM-wrapping or dicomization. You can determine which file types you want to be converted by default.

The dicomization of videos is a special case. For more information, see [Section 2.4](#).

☰ Force creation of new UIDs

This option creates new UIDs every time DICOM files are archived.

☰ Allow mirroring and rotating

This option enables mirroring and rotating in the viewer. These functions are deactivated by default to ensure images are not mistakenly archived in a reflected state.

☰ Alternative Specific Character Set

This option allows you to automatically correct character errors in DICOM files by defining one or more alternative character sets here. The naming of character sets follows the DICOM standard, e.g. the ISO-8859-1 character set (Latin 1) is input as ISO_IR 100. For more information on specific character sets, please consult the DICOM standard.

2.2.7 Barcode

In this section, you can customize the use of a barcode scanner for setting a patient context:

☰ Enable barcode for quick search

If this option is selected, *syngo.share* import responds to the start key sequence of the barcode scanner.

☰ Start/Stop key sequence

This field issues a key command for which your barcode scanner has been programmed as well. The indicated command activates the quick search input field. When scanning a barcode, the barcode scanner automatically sends the activation character in advance. As a result, the content of the read barcode is transferred directly into the quick search input field and a search is started. If the activation character combines multiple keys, use + as a separator, e.g. `CTRL+SHIFT+F3`.

☰ Ignore check digit

This defines whether the last number (check digit) of the barcode should be ignored. Even when this option is activated, the pattern has to match the barcode including the check digit. This option only deletes the check digit after the barcode was recognized correctly.

☰ Ignore padding

This option defines whether the leading zeros of a barcode should be ignored. Even when this option is activated, the pattern has to match the barcode including the leading zeros. This option only deletes the leading zeros after the barcode was recognized correctly.

2.2.8 Tooth Status

Adjust the display of dental X-rays in the **Tooth Status** tab. The default size of the tooth status image is 2500x1280 pixels. You can also set the default viewing/editing tool.

2.2.9 Filesystem


You can determine which drives or directories are shown in **Archiving** > **Filesystem** by (un)hiding them. Furthermore, you can select drives whose files you want to cache in a temporary directory before archiving. Caching files can speed up the archiving process. The temporary directory will be cleared automatically after successful archiving or, in case of an error, when you next launch *syngo.share* import.

2.2.10 Video

This section lets you configure video and image recordings as well as recording devices. For instance, you can determine the maximum permissible length of a video recording to save space or adjust the image quality for lossy compression formats.

To ensure correct video playback, only import videos with a constant bitrate. Videos with a variable bitrate may not be displayed properly.

To keep loading times to a minimum, import videos to an Organizational Unit which does not compress videos per its archiving rules.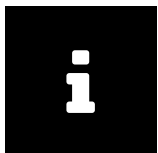

Note

Adjustments to the individual camera settings should only be made by qualified personnel. If you cannot select a device from the list, the configuration options have been disabled by your system administrator. Should you wish to make adjustments, please contact your organization's IT department.


☰ Camera Settings

Click on **Edit** to make adjustments to a selected camera in the list. You can change the resolution for video recordings and still image capture, as well as the recording bitrate. Moreover, you can (de)activate video preview and audio recording. To record audio, select your preferred source device.

☰ Offset Correction for Audio Tracks

Some video recording devices may introduce an unwanted offset between the video and audio tracks during streaming. To counteract this, *syngo.share* import offers a delay feature for correcting an offset introduced by a given device.

On the **Edit** screen, enter a delay period (in ms) for the selected device. You may have to test multiple values in order to find the appropriate value for the given device.

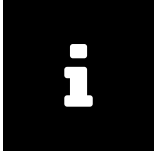**Note**

Be sure to check the delay setting whenever changes have been made to the device (hardware, drivers, updates, etc.), and make adjustments accordingly.

2.2.11 Still Image Capture

Select the serial port that should be used to connect the footswitch for image capture.

You can also enable an acoustic signal to confirm successful still image capture.

2.2.12 Quick Import

Enable this option to allow the quick import of DICOMDIRs from the DICOM Overview (see [Section 3.3.1.1](#) for more information).

2.2.13 Zero-Click Import

Enable this option to allow the immediate import of DICOMDIRs from CDs or USB flash drives.

Drives

This defines the drives that allow zero-click import. Multiple drives can be selected.

Destination Type

The destination for zero-click-imported files can be a DICOM C-STORE target, an AET or an Organizational Unit. The defined DICOM C-STORE targets can be selected, AETs and Organizational Units have to be entered manually.

Eject CD-ROM after import

If activated, the CD drive opens automatically once the import is completed.

2.2.14 Windowing Default Settings

In this section, you can create presets for automatic windowing. Windowing values are saved when files are archived. It is recommended that you test various presets and save only those that yield satisfactory results.

Lower threshold

Lower limit of the brightness area (Level/width control).

Upper threshold

Upper limit of the brightness area.

2.2.15 Viewer

You can choose the default tool used in **Edit** mode from a list. If you make no changes, the selection remains set to the mouse pointer.

2.3 Communication

This section contains settings that govern communication with the system. These settings should only be made or checked by the system administrator.

2.3.1 *syngo.share* core

If *syngo.share* import is implemented with *syngo.share* core, the connection parameters are set here.

2.3.2 DICOM

This tab contains all the settings for communication with an external PACS. Depending on the use case, the corresponding options and services are configured here, e.g. C-STORE for archiving.

2.3.3 HL7

In this section, HL7 peers are configured for the verification of patient data via the patient index (PIX) and the *Patient Demographics Query* (PDQ). For PDQ, the fields to be queried can be set.

2.3.4 Worklist

Choose the query period and the update interval for the worklist. The set query period determines which documents will be displayed in the worklist. The standard setting is 24 hours.

2.3.5 Order List

Similar to the worklist area, you can set the query period, which determines which entries are shown in the order list. Furthermore, you can set the update interval for the list and designate an Alternate Scheduled Station AET.

2.3.6 IRWF-specific Options

Here you can adjust settings related to the Import Reconciliation Workflow (IRWF) profile. This IHE profile manages the import of image data from external CDs, etc., and attempts to match their identifiers to the values stored in the system.

2.4 Video Postprocessing

2.4.1 External Video Postprocessing

It is possible to configure an external application (FFmpeg) to make changes to video files. This allows you to postprocess even video formats that *syngo.share* import cannot normally postprocess. Videos can be converted to another format or trimmed before importing them into the archive.

Warning

It is not possible to guarantee data integrity of transferred data between external applications (FFmpeg) and *syngo.share*.

2.4.2 External Video Transcoding

Set the target codec, bitrate and maximum resolution for transcoding here.

When Videos are set to be dicomized (see [Section 2.2.6](#)), special conditions apply because the DICOM standard only supports three video codecs. Accordingly, *syngo.share* import will provide alerts in certain cases when videos are to be archived.

- If the generic video file is of a format that can be embedded directly into a DICOM object as MPEG2, H.264 or H.265 and its resolution conforms to the DICOM standard, it will be dicomized and archived without delay.
- If the generic video file can be embedded directly but its resolution does not conform to the DICOM standard, you will receive a notification and the file will be marked with a yellow warning triangle. Still, the file can be archived as usual.
- If the video file is of a format which cannot be embedded directly into a DICOM object, you have to specify an external application (FFmpeg) for the job. In this case, you will receive a notification that the file is being transcoded. However, if you have not yet specified an external application, the file is rejected from archiving and marked with a red X.

2.5 DICOM Import Profiles

A DICOM import profile is a predefined bundle of DICOM data elements that can be applied to documents during import. Their purpose is to overwrite existing patient contexts, thus anonymizing patient data and providing a means for “de-identification” as required by the DICOM standard.

syngo.share import makes it possible to ensure that a DICOM import profile is always applied during document import or during DICOMDIR quick imports. This function can be applied using the selection menus **Enforce use of import profile** and **Use import profile for DICOMDIR quick import**.

To create a new DICOM import profile, click on **Add** and provide a name. You are then able to specify the rules for the handling of various DICOM elements:

2.5.1 Default Rules

This area specifies how private DICOM elements and the personal data of patients are to be handled. Note that if you elect to remove or de-identify DICOM elements, Type 1 DICOM elements will remain unaffected by this setting.

2.5.2 Additional Rules

In this area, rules can be applied to additional DICOM elements. The elements are selectable from a menu.

The following actions are possible:

☰ Clear

Apply a DICOM element with an empty value.

☰ Remove

Delete the affected DICOM element.

☰ Set random value

Apply a random value to the DICOM element.


Set value

Apply a specified value to a DICOM element.

2.6 Personal Settings Profile

You can save and load your *syngo.share* import settings as a personal user profile. This is especially helpful when you share a workstation with several other users or want to use *syngo.share* import on multiple workstations.

To create a new user profile, click on **Save Profile...** in the bottom bar of the configuration menu (⚙️). Specify a file name and target folder before clicking **Save**. Your profile will be saved with the .ics (ITH icoserve Config File) extension.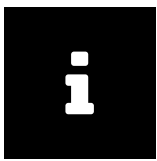


Note

Save the profile on an external media device or network drive if you would like to have access from another workstation.

To apply a personalized user profile, click on **Load Profile...** in the bottom bar. Select the directory and file of the desired profile and click **Open**. *syngo.share* import adopts all the settings contained in the profile.

3 Usage Instructions

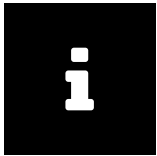Note

These usage instructions provide a comprehensive description of the functions of *syngo.share* import. Due to the configuration settings of your organization or missing permissions, some of these functions may not be available to you. If you have any questions, please contact your system administrator.

3.1 Getting Started

3.1.1 User Interface

The *syngo.share* import user interface is divided into the following areas:

- [Main Menu](#)
- [Main Area](#)
- [Document List](#)

3.1.1.1 Main Menu

The main menu is used for navigation between the different areas and contains the following sections:

☰ Home

Login area, contains release and application information.

☰ Context

Patient search area, to acquire the correct patient context.

☰ Archiving

Main working area for data acquisition.

☰ Quick search/Context set

Quick search for a patient. If a context is already set, this context replaces the quick search field. You can also switch to the viewing and editing area here.

☰ Online Status

Shows the host, *syngo.share* import is connected to, on mouse over. You can select to work offline here.

☰ Job Queue

A list of executed tasks and their status. The symbol in the main menu also changes to provide visual feedback on the queued tasks.

☰ Settings

In this area, you can change the program settings.

☰ Help

Contains this help file.

☰ User

Options to log in/out, user information, application information.

3.1.1.2 Main Area

syngo.share import always archives the acquired data with the set context of a patient and his examination. In the **Context** area, you can set the context through different methods: quick search, manual patient search, entity search or worklist, or with the help of a barcode scanner. Only the quick search is enabled by default.

In order to add data, different sources are available in the **Archiving** area: a filesystem browser, an archive search function as well as scan and video tabs.

The **Archiving** area also offers a tagging section. Its visibility/appearance depends on the selected Organizational Unit and your user permissions.

3.1.1.3 Document List

The document list contains all loaded or recorded files. Every list entry includes a preview image (either a thumbnail or generic file icon) and available information (patient name, examination date, description, etc.). Additional information (e.g. filename and file type) is shown in a tool tip on mouse-over.

Clicking on  lets you perform the following actions for each entry:

View

Allows you to view and edit a document in the display area before archiving. Documents of a non-supported format are not opened in the display area, but in an appropriate external application (if available). For more information, see [Section 3.4](#).

Tag

Lets you add keywords to a document before archiving. General tagging for all loaded elements is done in the context setting area.

Toggle generation of new UIDs


This function creates new UIDs (StudyInstanceUID, SeriesInstanceUID, SOPInstanceUID) for the selected file during import.

Apply DICOM import profile

You can select a DICOM import profile from the list to be applied during import.

Remove

The document is removed from the list and will not be archived.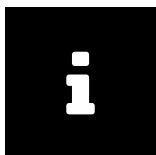

Note

If you remove recently recorded videos or still images from the document list, these elements cannot be reloaded.

All documents in the document list will be archived into the selected storage area (Organizational Unit). The context that is visible in the document list will be used for archiving.

3.1.2 Login and Logout

You will be automatically asked to log in, or you can manually log in via the **Home** or the **User** section. To log in, enter your user name and password, select the domain (if applicable) and click on .

If your system administrator has activated the automatic login with your Microsoft Windows user credentials, you will be automatically logged in to *syngo.share* import.

Login is not required, however, some Organizational Units may ask for authorization upon upload. Always terminate logged-in sessions by clicking on **Logout** to prevent unauthorized access.

At/after login, the following events may occur:

⌵ Login message


You may be shown a login message. Click on **Accept** to finish logging in to *syngo.share* import. If you accept the message once, you will not receive it again until your system administrator sets a new login message. If you click on **Decline**, your login will be canceled.

⌵ Incorrect password entry

If you enter your password incorrectly too often, you will receive a message that your user account has been locked for security reasons. Only your system administrator can unlock your account.

⌵ Screen lock

If you are logged in, you can manually lock *syngo.share* import to prevent unauthorized access. Click on **Lock** in the **Home** or **User** section. To continue working, enter your password on the lock screen. This unlocks *syngo.share* import and you will see the current status of your work. You can also log out the current user on the lock screen in order to allow another registered user to log in.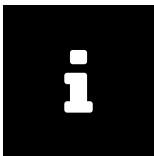


Note

Note that logging out the current user means that this user's unsaved work will be lost.

3.1.3 Changing the Password


In the **Home** area, click on **Change password** and follow the on-screen instructions. Save your changes.

The **Change** button will only become clickable once you have entered all the requisite information correctly.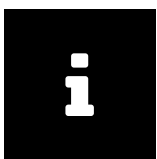

Note

Passwords in *syngo.share* import are case-sensitive.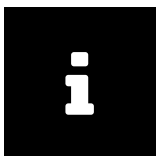

Note

Passwords may expire if this feature has been configured by your system administrator, meaning that you will have to pick a new password after a defined period of time. The password provides access to patient data. Please keep it confidential and change it regularly.

3.2 Setting a Context

All documents are archived in the context of a patient or an entity. This means that each document is saved as part of the electronic patient folder for retrieval at a later date.

The context to be used is displayed in the context header bar. Before archiving documents, you must search and select a context (e.g. patient data) to allocate the document accordingly.

syngo.share import also offers the possibility to apply an anonymous context. See [Section 2.5](#) for the configuration of so-called import profiles.

To set a context you can use the quick search in the main menu to search for a patient ID and/or alternate patient ID and/or visit ID, depending on the settings. For a more detailed search go to the **Context** area and use one of the following options:

Manual patient/visit search

Enter at least one search criterion. Select a patient from the results list and confirm with **Select**. You can also define a visit ID or search for existing visits via **List visits**.

Manual entity search

Select an entity type from the drop-down menu and enter your search criteria.

Create a new patient or entity

Click on **Create new** in the appropriate tab. When creating a new patient, you must always provide the patient ID. The visit ID is always optional. When creating a new entity, at least one field has to be filled. The input fields vary according to entity type and can be named differently.

Worklist

At some workstations, *syngo.share* import offers a list of work orders which shows patients to whom documents should be archived. The patient context can be recorded automatically via the worklist. If you have questions regarding the configuration of a worklist for your workstation, please contact your system administrator.

Order list

The order list is another means of recording the patient context. Unlike the worklist, which uses DICOM to receive and set patient data, the order list can retrieve non-standardized information like keyword tags. These keywords will then be set for the study during the gRPC import (if necessary, you can edit the keyword tags beforehand).

Barcode scanner

If a barcode scanner is available and configured, you can use it to read visit ID or patient ID barcodes. All relevant patient data for the current visit is set directly in the context area. This method saves time and prevents manual entry or typos.

Automatic data transfer from HIS via *syngo.share* import URL interface


With this alternative for capturing the patient context, you can search for the patient directly in the HIS. A *syngo.share* import button in the HIS lets you transfer the data to *syngo.share* import. There, the data is entered in the **Archiving** area, and you can now add documents.

Patient verification

If the automatic query for patient ID is activated under **Settings** > **Communication**, the DICOM data of the files to be imported is checked against the selected target archive. Depending on the settings, different criteria are queried.

If the patient is found and the data can be verified in its entirety, this patient context is set automatically. A green checkmark indicates the verification was successful. In case the patient does not exist or a definite allocation is not possible, an exclamation mark is displayed next to the patient context. The **Patient Verification** tab opens automatically with a list of potential matches. Choose the correct patient from the list or create a new patient entry in the **Patient** tab.

The verification process can also be started manually. Under **Context** **Verification** in the main menu, choose the tenant, the DICOM C-Find source or the PIX/PDQ peers. The verification starts immediately.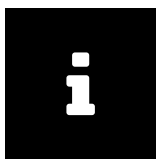

Note

The quick search as well as the manual patient ID or visit ID search only search for HL7-managed entries by default. You can adjust this and other settings in the **Settings**.

To start a search, click the search button or hit **Enter**.

Setting the Date and Time of an Examination


In addition to the context, date and time of the examination is needed. The current date and time are displayed by default, although these entries can be changed manually. To undo changes and set the current date/time, click on **⌚**.

For DICOM images, the date and time contained in the DICOM information will be applied; they will not be overwritten by the currently displayed or configured date/time settings.

For generic file formats (e.g. JPEG or TIFF), date and time will be set depending on the first available information from the following list:

1. Manually set study date
2. Study date from worklist or URL call-up
3. Oldest creation date from the EXIF metadata of all elements in the container
4. Current Date and time

If the study date is set to non-editable in the configuration, you can only enter date and time manually if this information has not been set by the worklist or a URL call-up. In this case the first and second entry in the previous list are swapped.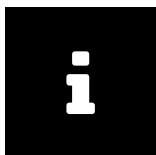

Note

If pictures from a digital camera do not contain EXIF metadata and have to be retroactively archived for a patient or visit, you have to manually set the timestamp to the date and time when the actual examination took place.

Setting the Generic File Type

For generic documents, you can also set the generic file type in the document list. Setting the generic file type can be used to trigger certain workflows and/or affect the display order of the document in other *syngo.share* applications. The available generic file types are determined by your system administrator.

Note that you have to disable DICOM-wrapping for PDF, image and video files under **Settings** **Specific** **File modifications** in order to be able to set the generic file type for these documents.

Patient QR Code

Once the context is set, the patient's name or the entity appears in the main menu. For patients an information button **i** is shown. Click it to show all patient information available and a QR code. You can scan this code using the mobile app Visual Capture to quickly find the correct patient and thus add health data gathered with your mobile device (image, video and audio files) to their health record.

3.3 Adding Documents

With a context set, documents can be added and archived for the selected patient or entity. You can load different document types from the following sources:

- File System
- Archive
- Video
- Scan
- DICOM
- Tooth Status
- Camera

Automatic Archive Search

syngo.share import can automatically search the archive to determine if a study or images to be imported already exist in the configured target archive. This option has to be activated under **Settings** **Communication**.


In case of a match, the image is marked with an exclamation mark in the document list and the patient context is set automatically. It is not possible to import this image to another patient. If another patient context is selected, the import is blocked. To import the image nevertheless to the context set, new UIDs have to be created (see [Toggle generation of new UIDs](#)).

3.3.1 File System

This option allows you to load images and other documents that already exist to *syngo.share* import, using the following methods:

- File Browser
- Dragging and Dropping Files from Windows Explorer

There is also the option to automatically delete the original files after a successful import. This can be configured in the settings.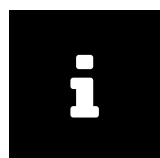

Note

Note that documents in *syngo.share* import are always recorded with context. Before loading an image or another document, you must record the correct context. This work step is not needed when loading DICOM files that already include patient and examination data and when working with worklists.

3.3.1.1 File Browser

The file system browser resembles Microsoft Windows Explorer. It consists of a navigation tree and an area that shows the contents of the selected drive or folder. You can select a directory to browse in one of two ways:

- Scroll through the navigation tree and click on the desired folder.
- Click on in the header bar to quickly select a specific folder, device or the Windows Favorites folder.


You can set a selected directory as your start directory by clicking on .

Use to update the filesystem browser display in order to see any changes to the folder contents that may be happening in the background, e.g. the deletion of a file.

In the footer bar, you can adjust the display style and size of the preview icons. The list view provides detailed information on the elements and lets you sort the various columns. The tile view offers a compact thumbnail view. The DICOM overview (to be selected via the CD symbol) shows all DICOM files in the selected directory (see [Quick-Importing DICOM Files \(e.g. from a Patient CD\)](#)).

Loading Files into the Document List

There are several ways to add files to the document list: Either double-click on the desired item (only possible for single files), drag and drop the element(s) into the document list, or choose . To select multiple files, make use of or while selecting the desired items, or use or to select the entire file contents.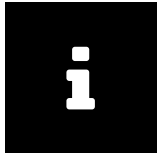

Note

In case the set patient context does not coincide with the information in the loaded DICOM files, the respective DICOM files are marked with an exclamation mark or a cross. Assign new UIDs to re-assign the files and delete the saved patient context (see [Section 3.6](#)). When loading data without change, these entries remain as they are.

Quick-Importing DICOM Files (e.g. from a Patient CD)

In the DICOM overview, *syngo.share* import offers a quick import function for DICOM files accessible via the CD symbol in the footer bar. If the selected directory or medium (e.g. a patient CD) contains a DICOMDIR directory structure, all DICOM files included therein are listed immediately. In case no DICOMDIR directory file is found, you can scan a CD, USB stick or any directory manually.

In the list of all the DICOM files found, you can click the corresponding button to quick-import

- all data found,
- the data of a specific patient,
- a specific study or series.

Alternatively, drag the desired file(s) into the document list and start the import manually.

The quick-imported files will be archived to the Organizational Unit selected above the document list.

3.3.1.2 Dragging and Dropping Files from Windows Explorer

syngo.share import supports the option of adding files from Microsoft Windows Explorer directly to *syngo.share* import.

Select a file in Windows Explorer and drag it into the *syngo.share* import document list while holding down the left mouse button.

3.3.2 Archive


This tab allows you to search for documents which have already been archived to *syngo.share* core and edit their context.

The search offers numerous fields which you can customize by clicking on the small arrow next to . Use the arrow buttons in the configuration window to add or remove search fields or change their order of appearance.

The entry method varies according to the search field type. If you wish to delete search terms, either click the in the desired field or use to empty all search fields at once.

After sending your search via , you can find the results in list format in the **Studies** area (for more information, see [Section 3.3.2.3](#)).

When you select a result from the list, the **Series** area will display a thumbnail view of the documents contained therein (see [Section 3.3.2.4](#) for details).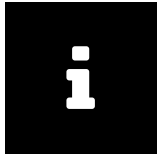

Note

Due to visibility restrictions you might not be authorized to view all parts of a document. This might lead to an adjustment in the series and/or object count once you click on an entry in the results list.

3.3.2.1 Types of Search Fields


Depending on the category a search field belongs to, input methods vary and certain additional conditions may apply.

Free Text

You can enter search terms directly into this type of field (e.g. First and Last Name, Referring Physician, Patient ID). The search is case-insensitive. You can also use wildcard characters to widen or narrow the scope of your search.

↳ Wildcards % and *

The % or * character can be used as a placeholder for any number of characters and is always automatically (and invisibly) appended to the search. For example, entering the letter “m” in a free text search field will list all entries starting with this letter. However, a search using only % or * is not possible.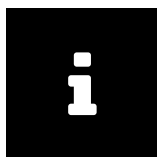

Note

This automatic addition of % does not occur in ID fields such as Patient ID, Accession Number or Visit ID in order to avoid false positives when searching for patients via barcode scanner.

Example 1: Search using %

If you search for last names using “W%r”, the results will include not only names like Wheeler and Walker, but also Waterfield, Whorton, Workowski, etc.

↳ Wildcards _ and ?

_ or ? act as a placeholder for a single character, which allows for a narrower search than % and *.

Example 2: Search using _

If you search for last names using “m_ller”, you will receive results for e.g. Miller and Muller, but not Mueller. However, longer names such as Mellerson and Mullerbach are still included due to % being automatically appended to the search.

Special Free Text Search Fields

↳ Mother’s Maiden Name

The mother’s maiden name may contain special characters. Due to technical reasons, the search field does not support some of these special characters. If the name you are searching for contains one or more of the special characters `\ | ~ & ^`, please use a wildcard at this position to conduct your search.

↳Description

This field searches the DICOM tag StudyDescription. The % wildcard is added automatically to the front and back of the search term.

↳Patient Age/Study Age

These fields require a particular syntax. In addition to entering the desired number, you have to indicate the desired unit of time (y or a = years, w = weeks, d = days, h = hours). Searches with only the numerical values are not possible. By adding > or <, you can search for certain age ranges. Without a lesser- or greater-than sign, the search will only include the precise age.

Example 3:

Searching for patients younger than 20 weeks: <20w

Searching for patients exactly 70 years of age: 70a or 70y

☰ Selection List

Fields of this type (e.g. Sex or Document Type) require you to choose an entry from a predefined list. Certain selection lists contain items with subcategories. Simply click the arrow symbol next to the item in question in order to view and select a subcategory.

Certain selection fields, e.g. Modality, allow multiple list entries to be selected in order to expand the search.


Special Selection List Fields

↳Region

This search field filters for the body part or body region examined. The available items are based on the DICOM tag BodyPartExamined.

☰ Date

Date fields offer several options: You can either set the current date by simply clicking in the field, input the date manually, or select a date from the calendar by clicking on the arrow symbol.

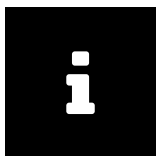Note

The date format and calendar entries depend on the format and region configurations of the operating system on the local machine (the Microsoft Windows configuration option is found under **Control Panel** > **Clock Language and Region** > **Date and Time** > **Region**).

Special Date Fields

↳Study Date

This field combines a selection list with a date search. First, select a time period indicator from the list. If the indicator is connected to the current date or time (e.g. "today" or "yesterday"), you can start the search immediately. However, if you wish to select a certain date ("on", "from", "until") or a certain date range ("between"), one or two date fields will open where you can enter the desired date as described above.

☰ Keyword

You can also search for documents in *syngo.share* core according to their keyword tags. The configurable keyword group fields depend on your user permissions.

To search for a keyword click on the arrow button in the keyword search field. A list of keywords from all associated keyword catalogs opens. To filter the list, enter part of the desired keyword.

Click on the desired keyword to add it to the search field (you can add more than one keyword to a search). Keyword search fields associated to free text keywords do not provide a selection list, you have to enter the search terms manually.

A search for keywords from catalogs is always an exact search. For searches in a free text catalog however, a placeholder is always added automatically at the beginning and the end of the text entered into the search field. These search terms are set in italics in the search field.


All documents will be found that are tagged with at least one of the specified search terms on at least one tagging level, i.e. if e.g. one instance of a series is tagged with a keyword searched for, the entire corresponding study will be included in the results.

To remove a particular keyword, click the  right next to it.

For more information on keywords, see [Section 3.5](#).

3.3.2.2 Saving a Search Query

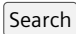You can save frequently used search queries to avoid having to reenter them every time. Doing so will also save the entered search terms, which will be restored when the saved search query is selected.

1. Fill in the desired fields in the search area.
2. Click  to start the query.
3. If you are satisfied with the result, you can click on  to save it.
4. Enter a name for the query (max. 50 characters, comma and semicolon are invalid) and click .
5. You can find the saved search query as a new button above the search field area. Clicking it will execute the saved query.

Setting a Search Query as Standard

You can set a saved search query to start automatically as soon as you log in to *syngo.share import*. Click on the arrow next to the desired search query button and select  from the submenu. The standard search will be starred. At your next login, *syngo.share import* automatically switches to the appropriate tab and loads the results list of the standard search query.

Editing a Saved Search Query

Click on the arrow next to the button of a saved search query to modify it. You can edit the displayed fields, rename the query or delete it.

3.3.2.3 Results List

Similar to the search fields, you can configure the results list according to your needs. Via , you can use the arrow buttons to add or remove columns from the list or change their order of appearance.

In addition, you can

- drag columns to change the order,
- adjust the column width manually by dragging their border or automatically by double-clicking on it,
- sort the columns by clicking on them (the displayed up- or down-arrow indicates the type of sorting in use).

Grouping

You can group the results based on up to five columns. These columns have to be arranged in the exact order according to which grouping is to be performed. Grouping will subsume entries with matching values within one column into a group. In case of multiple groups, grouping is performed hierarchically from left to right. The grouping icon ▲ appears in the header of every grouped column, indicating the number of grouping levels.

Setting Automatic Updates for the Results List

The results list is static between searches, meaning that if archive data is changed following a search query, these changes will not be reflected in the results list. You can update your results list manually by repeating the search or by setting an update interval by clicking on ⌚.

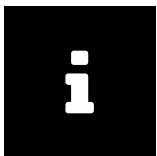
Warning

The automatic update of the results list causes a higher system load. The shorter the update interval, the higher the load. To avoid an unnecessarily high system load, choose longer intervals or, if unneeded, disable the automatic update function altogether.

Copying Results to Microsoft Excel

You can export search results to Microsoft Excel by selecting the desired items from the results list. Press **Shift** while clicking lets you select multiple consecutive entries; pressing **Ctrl** while clicking allows you to select multiple non-consecutive entries.

If you are satisfied with your selection, either use **Ctrl + C** or click on  to copy the items to the clipboard.

Note

Note that the context menu entry **Copy** only copies the contents of the particular results field used to call the menu.

Now the copied items are ready to be inserted into a new Excel worksheet by clicking on a table cell and using **Ctrl + V** or by clicking the paste icon in the toolbar.

Restricted Document Parts

Documents may contain restricted elements, i.e. series or generic files which you do not have permission to view. These restricted elements will not be shown in the preview area or the document list. However, depending on your other permissions, you can still perform operations which affect the entire document.

Documents with restricted elements are indicated in the **Objects** column of the results list, which lists the total number of elements contained in a document (series or generic files). When you select a document with restricted elements, its total number of elements will be divided into visible and hidden elements.

3.3.2.4 Series Area

The series area below the results list allows you to view the contents of selected search results and receive detailed information on the individual documents. You can adjust the area size by clicking and dragging the  symbol, or minimize it by clicking on .

A thumbnail preview of each document is shown. These thumbnails are created when documents are uploaded to *syngo.share* core; if available, they are shown for file formats which can be displayed in *syngo.share* import. Files from external applications are displayed with their generic file icon.

Further information on the documents is available on mouse-over via tooltip (e.g. Patient ID, Examination Date, Document Description). The available information varies depending on the type of document.

≡ Image number/Filename/File count

Depending on the presentation options, files in the thumbnail view are displayed with their image number (DICOM series) or name (generic document) or the number of files in a stack (only available in stack view).

≡ Stackability

The stack symbol indicates elements that are stackable or, if stack view is activated, elements that have been stacked. Note that generic files cannot be stacked.

≡ Tagging

An orange triangle lets you know which files have been tagged with keywords.

≡ Version

Hovering the mouse pointer over an element will also display a  symbol. Click on it to receive information about the document version.


When you select multiple results or when a result contains multiple documents, you can order and filter these documents in series view.

≡ Filter according to ICOTAGS keywords

You can filter the elements shown in thumbnail view by choosing to display only those tagged with a certain keyword from the ICOTAGS catalog, e.g. "findings-relevant". Simply choose the desired keyword from the selection list.

≡ Grouping and Sorting

Grouping documents is useful when e.g. multiple series belonging to different patients are shown in the thumbnail view. Select the desired entry from the grouping list to group the thumbnail elements beneath their respective headers (e.g. grouping elements by patient arranges the elements beneath headers bearing their associated patient's name). You can also change the sorting of a group by ticking the checkbox **Sort descending** (the checkbox is deactivated by default, meaning that groups are sorted in ascending order).

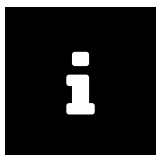Note

Grouping according to study always shows the most current study first. The older the study, the further down it is found on the list. The same is true for grouping according to examination date. If the **Sort descending** checkbox is activated, older groups are shown first.

≡ Stack series

You can use the **Stack series** checkbox to achieve a better overview of series containing a great number of images (>100). When stack view is activated, the number of files contained in a stack is displayed (e.g. #167) alongside the stack symbol.

≡ Adjust thumbnail size

To increase or decrease the size of the thumbnails, either drag the slider to the left or right or click on the  and  icons.

3.3.2.5 Editing Files from *syngo.share* core

In order to edit *syngo.share* core documents, double-click on the the desired result or use to add them to the document list. The documents will be labeled "From the archive".

The set context appears, just like in other tabs, above the search area. You can now change the context by clicking on the small arrow, meaning you can assign the documents to a different patient or edit the patient context (e.g. patient ID, name, sex) itself.


You can also add or remove tags (see [Section 3.5](#), edit images (see [Section 3.4.2](#)) or change the creation date and time (see [Section 3.2](#)).

Depending on the configuration of *syngo.share* import, edited documents will either be saved separately from the original documents, or the old version will be deleted and replaced by the edited version ().

For more information on viewing and editing documents, see [Section 3.4](#).

3.3.3 Video

In this tab, you can directly record videos as well as single images and archive them in *syngo.share* import. This requires a connected video source that supports Microsoft Direct Show. Recorded videos are saved as generic files in the archive.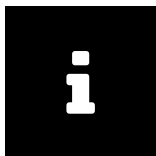

Note

This tab is grayed out if no video source is connected to your computer or it is not detected/ installed correctly.

Select the video source from the drop-down menu. You can make changes to the device configuration under , see [Section 2.2.10](#).


The video preview shows the current video stream that is transmitted from the connected video signal. To start recording, click on or use the configured foot switch.

You can pause and resume the recording by clicking on . Clicking on finishes the recording, and the newly recorded video file is added to the document list.

If a maximum recording duration has been set under , the recording will automatically stop at the set amount of time.

To take a still image of the current video stream, click on or use the configured foot switch. Still images immediately appear in the document list.

To use a foot switch, you first have to configure its serial port under .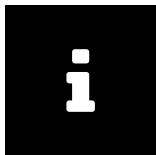

Note

If you remove recently recorded videos or still images of the video stream from the list, these elements cannot be restored and reloaded.

For information on video editing, see [Section 3.4.2](#).

3.3.4 Scan

Physical documents can be scanned and archived directly in *syngo.share* import.

Select a TWAIN source (scanner) from the drop-down menu. If the scanner is not automatically detected or does not seem to be functioning correctly, you can activate **Scanner compatibility mode**.

All scanned documents show up in the preview panel. As soon as a document is chosen for archiving, it shows up in the documents list and is grayed out in the preview panel. If deleted from the documents list, the document is again available for selection in the preview panel. The preview panel can be cleared by clicking .

Scanned documents can be saved in TIFF, JPEG, or PDF/A format.

3.3.5 DICOM

This tab allows you to search for files in a configured third-party archive (external PACS). If more than one external PACS has been configured, you can select the PACS you wish to search in the drop-down menu of the header bar. Similar to the archive search, there are various fields available for entering search parameters (for more information, see [Section 3.3.2.1](#)). Double-click to load a result into the document list.

3.3.6 Tooth Status

This tab allows you to assign individual tooth X-rays to a patient's dental chart.

- Set the patient context.
- Load the desired X-ray image(s) into the document list.
- Click on the image to load it into the viewer area.
- Select the teeth visible on the X-ray by clicking the corresponding checkboxes on the dental chart. You can switch between an adult's set of teeth and children's deciduous teeth in the drop-down menu. It is possible to check different teeth on one X-ray as permanent and deciduous teeth. Furthermore, permanent and deciduous teeth of the same position can be marked on both dental notations. If applicable, you can check one or more complete teeth's quadrants at a time.
- When you then select the next X-ray for assignment, the already assigned teeth can no longer be checked in the chart.
- Incorrect assignments can be easily corrected by re-selecting the image in question and making the necessary corrections in the dental chart. If you want to undo all assignments at once, select in the footer bar.
- Assigned images are highlighted with a tooth symbol in the document list.

The Tooth Status tab also offers rudimentary viewing and editing tools such as contrast, rotating and zoom. The tools are available in the context menu (by right-clicking in the image) and the toolbar. You can adjust the toolbar according to your preferences: Simply right-click to open the configuration window and use the arrow buttons to add/remove tools or reorder them.

Edits made in the Tooth Status are also shown in the image preview in the bottom half of the screen.

Once you are done assigning images, click on .


syngo.share import can also create a composite image from the assigned X-rays for overview purposes. Simply click on in the footer bar.

3.3.7 Camera

In this tab, you can load images directly from a connected digital photography device (e.g. camera, tablet).

It resembles the **Filesystem** tab in functionality and appearance. Double-click on the desired device in the area or use the ▼ symbol to expand its storage directory. The right-hand area shows a preview of all the elements contained in a selected folder. You can adjust the preview area according to your needs (see [Section 3.3.1.1](#)).

Load the desired images into the document list via double-click or drag and drop (see [Section 3.3.1.1](#)).

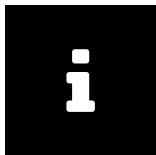Note

If a device is not shown correctly, check its connection and click on .


3.4 Viewing and Editing Documents

After setting the patient context and adding documents to the document list, you can view and edit images.

You can access the viewing and editing area in one of the following ways:

- Double-click the file you wish to view/edit in the document list.
- Select  **View** from the document list menu (ⓘ).
- Click on the context set or the icon next to the quick search field in the main menu.




You can return to the **Archiving** area via the main menu.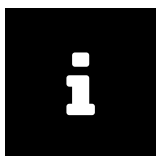


Note

Unsupported file formats are opened in an appropriate external program, provided it has been installed on your workstation.

The viewing and editing area offers a variety of tools in the context menu. You can also use the listed keyboard shortcut to quickly select a tool, or add frequently used tools to the toolbar at the bottom. To adjust the toolbar, right-click on it and use the arrow buttons in the configuration window to add/remove tools and arrange them in the desired order.

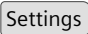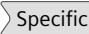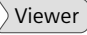Note that you can also choose your preferred default tool for viewing/editing under  **Settings**  **Specific**  **Viewer**  **Standard tool**.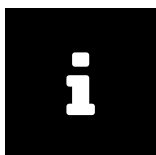

Note

Any changes you make to the image with editing tools will remain active if you clear the viewer or load another file for viewing/editing. Editing changes will be saved permanently upon archiving. However, changes made with viewing tools will neither be retained when the viewer is cleared nor saved during archiving.

3.4.1 Viewing Documents

Depending on the document type, *syngo.share* import offers a variety of viewing tools in the context menu.

☰ Pointer

This is the normal mouse setting. Use it to deactivate a previously selected tool and reset the mouse to the standard pointer.

☰ Scroll

Scroll back and forth between the individual images of a DICOM series or between the individual frames of a multiframe image. Hold down the left mouse button and move the mouse down to scroll forward, or move the mouse up to scroll back. Alternatively, you can use the mouse wheel for scrolling.

☰ Zoom

Changes the zoom factor of the displayed image. Use either the mouse wheel or hold down the left mouse button while moving the mouse up (to zoom out) or down (to zoom in).

☰ Magnifier

Enlarges a section of the image. You can freely move magnifying window by dragging it with the mouse.

☰ Brightness/Contrast

Changes the brightness and contrast values of generic images like JPGs or PNGs. Drag the mouse right or left to increase or decrease the contrast. Drag the mouse up or down to increase or decrease the brightness.

☰ Level/Window

Allows you to adjust brightness and contrast of DICOM images. It is used the same as the Brightness/Contrast tool. In addition, you can use **Level/Window Presets** to apply predefined brightness/contrast values (can be configured under **Settings > Specific**).

☰ Pan

Lets you move the image freely within the image display by dragging it with the mouse.

☰ Clear Viewer

Empties the viewer window. When using the keyboard shortcut, take care to click inside the viewer window first; otherwise the document list will be emptied instead.

☰ Fit to window

This function allows you to adjust the display of an image to the size of the current viewer window. This is useful when you have changed the display for a single image (e.g. by enlarging it with a zoom tool) and want to quickly return to an optimal display size.

☰ Video Viewing

The toolbar offers rudimentary video replay functions. ▶ and ■ start and stop the playback. Use the slider to adjust the volume.

3.4.2 Editing Documents

The viewing and editing area also offers tools for editing images or videos.

☰ Annotations

Here you can define calibrations and do a length calculations.

↳ Calibration

Draw a line whose length is known to you with the crosshair cursor (click to set a start and end point). Enter the length of the drawn line in millimeters in the calibration window. The line now serves as a benchmark for all following calculations.

↳ Calibration Presets

For taking pictures in a photo laboratory, where a camera with the same lens is used for various types of shots (e.g. full-body, face, hand, etc.) and is placed on different tripods or markers for this purpose, you can save calibration presets and use them on the captured images.


Select **Add new calibration presets** in the context menu and draw a line on a reference whose length is known to you. Enter the length of the drawn line in millimeters and name the preset. These new preset contains the camera type, the focal length of the lens as well as the custom description, e.g. Canon 70D 24mm: Full Body.

You can select more than one image and apply a chosen calibration in one go.

The existing calibration presets can be removed by selecting **Calculations** > **Calibration presets** > **Edit** from the context menu.

↳ Show Length

Draw a line using the crosshair cursor. The distance is calculated according to the current calibration. Click the left mouse button to set the start and interim points for the line. Double-click to set the endpoint. Use the pointer tool to edit your annotation by clicking on and dragging the desired line or start/end/interim points. To remove an annotation, press **Delete**. To remove all calculations at once, use the context menu entry **Remove all annotations**.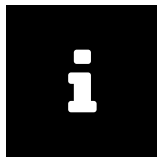

Note

If you remove an image from the document list, any annotations you may have made are not saved or archived.

≡ Rotate and Flip

The submenu offers a number of tools for rotating or mirroring an image. As soon as you mirror an image, the respective tool button in the toolbar is set to active to clearly indicate that the image is mirrored. By default, mirroring and rotating is deactivated in the settings in order to prevent accidental use.


≡ Video Editing

The video toolbar offers the following editing functions:

↳ Video Cutting


You have two means of shortening a video:

- By dragging the start and/or end marker in the cutting bar (below the playbar) to the desired position. The video will be trimmed accordingly and only show the trimmed sequence upon replay.
- By cutting a sequence from the middle of the video. Move the play marker to the desired spot, which will cause a small dot to appear in the cutting bar. Drag this dot to the left or right to select the sequence to be cut. You can adjust the selected sequence to your liking. Click outside the selected sequence to cut it. To undo the edit, click on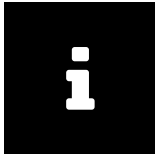

Note

Cutting sequences from the middle of the video is not possible when you have configured an external application (FFMPEG) for trimming. FFMPEG only allows the editing of start and end points.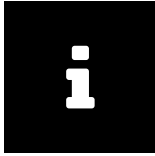

Note

Note that video cutting has a tolerance for imprecision of up to four seconds.

↳Add Bookmark

You can mark and comment on parts of a video by clicking on and entering a description. Bookmarks appear as small markers in the playback bar. Using and , you can jump between the set bookmarks. To remove a bookmark, either click on its position on the slider and press or click on next to and select .

↳Capture Image

To capture a still image from the video, click on . The still image is added to the document list and can be archived as a generic file or DICOM image. For archiving as a DICOM image, has to be enabled.

≡Split

A series containing more than one instance can be split into multiple series. Click the button with the split icon to open a preview bar showing all the instances of the series currently viewed. Select the instances you want to split into a new series. Click the button next to the preview bar to create a new series with the selected instances. It will be shown in the document list immediately.

You can also drag the selected instances with the mouse to the document list to split the series. Furthermore, you can move instances from one series to another by dragging them with the mouse. Instances may only be moved to series of the same study and only series that were previously split may be merged again.

3.5 Tagging Documents with Keywords

syngo.share import allows keyword tags to be applied to documents either on an examination, image, or document level. You can archive these documents with this keyword tag information. The keyword catalog and keyword groups intended for this purposes are set up by the administrator in *syngo.share* Configuration.

≡Tagging a Study


You can assign keywords to studies in the tagging area above the document list. Select a destination (Organizational Unit) from the drop-down menu to see the available catalogs and tagging fields. If no tag selection is visible, the selected destination does not provide keyword tagging.

≡Tagging a Single Image

Select the document in the document list and choose .

The process of tagging is the same for both cases. Depending on the selected destination, different tagging fields are available. The keyword groups can be selected from the drop-down menu. If a catalog is

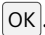expandable, a new keyword can be typed directly into the text field. It is also possible that no keyword catalog has been stored for a field. These are freetext catalogs that allow tagging via manual text entry.


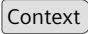
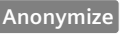
The respective keyword catalog from a keyword group is displayed by clicking on the arrow button. To tag a study or document, choose the desired keywords from the catalog menu and, if prompted, click .

3.6 Changing DICOM Elements

Usually, DICOM documents are archived with their existing DICOM tag data. However, *syngo.share* import offers the option of changing certain DICOM tags before or during archiving.

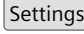
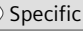
3.6.1 Editing the Context

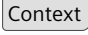Depending on the settings in   , you can change the set examination context, the time and date of the examination, as well as the allocated keyword tags.


To change the patient context, click on ▼ next to the patient's name and select   or  . The former allows you to set a new patient context via the search function; the latter allows you to edit the existing context data or set a randomly generated context for purposes of anonymization. You can also anonymize the context directly selecting  .


For changing the examination date, see [Section 3.2](#). For adding keywords, see [Section 3.5](#).

3.6.2 Creating New UIDs

If you activate the option  under   , *syngo.share* import will generate new values for the DICOM tags StudyInstanceUID, SeriesInstanceUID and SOPInstanceUID when you click on  in the document list.

3.6.3 Applying DICOM Import Profiles

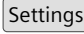If you have created one or several import profiles (bundles of DICOM elements) under  , these can be applied during archiving. For information on the creation of import profiles, see [Section 2.5](#).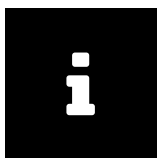

Note

Not every DICOM file can be displayed in *syngo.share* import, such as Presentation States. For these files, patient-relevant DICOM tags will be changed and they will be archived as any other file.

3.7 Archiving Documents

Archiving the captured data is the last step of the *syngo.share* import workflow. After setting the context, adding documents to the document list and performing any necessary viewing/editing steps or tagging, you can start the archiving process.


The data will be saved in the central archive of your healthcare institution. All documents loaded into the document list will be archived.

Select the desired destination (Organizational Unit) for archiving in which the data should be available afterwards. To start the archiving process, click on the respective button or press **F8**.

The archiving process will start in the background. You can track its progress **Job Queue** area. Click the **Detail** button to receive more detailed information. The symbol **Job Queue** icon in the main menu also provides direct feedback on the archiving process.

In case the archiving process was started from the image display area window, the patient context will be unset, the document list will be emptied, and the image display area window will be closed automatically as soon as the archiving process has been completed. You will be returned to the **Archiving** area.

While archiving is in progress, you can continue working in the **Archiving** area as usual.

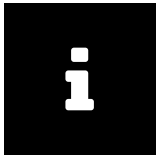Note

In case of connection disruptions between *syngo.share* import and the archive, all newly acquired data will be kept until the connection can be reestablished. You can manually delay the import into the archive by clicking the colored lamp in the main menu. If you sever the connection manually, you also have to reenale it manually before you can transfer data to the archive.

3.8 Keyboard Shortcuts `syngo.share import`

General	
F1	Help
F6	New context
F7	Anonymize context
F8	Archive
F10	Switch between archiving and edit panel

PDF Viewer	
Ctrl A	Select All
Ctrl Home	Move to beginning of document
Ctrl End	Move to end of document
Page ↓	Next page
Page ↑	Previous page

Search	
F5	Start search
Ctrl C	Copy selected search results

Video Capture	
Ctrl Shift Alt R	Start recording
Ctrl Shift Alt S	Stop recording
F5	Capture screenshot
Ins	Set/Edit Bookmark

Viewer	
→	Next image in series
←	Previous image in series
Page ↓	Next entry in document list
Page ↑	Previous entry in document list
Del	Remove image from viewer
Esc	Pointer tool
B	Scroll through series
F	Windowing
Shift F	Default windowing value
Shift 1-3	Windowing presets
L	Length
V	Pan
Z	Zoom
Backspace	Reset

Scan	
F5	Scan

Document List	
F2	Edit context
F3	Change context
F4	Clear document list
Del	Remove all selected entries from document list
↓	Next entry in document list
↑	Previous entry in document list

Video Viewer	
Space	Play/Pause
→	1 second forward
←	1 second backward
↓	30 seconds forward
↑	30 seconds backward
Home	Move to beginning of video
End	Move to end of video
F5	Capture screenshot

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ITH icoserve technology for healthcare GmbH
Siemens Healthcare GmbH