syngo.share / Release VA30C / 2021-12-16 / Revision 6272

syngo.share webview

Online Help

Language Directory

Deutsch	3
English	47
Français	88

syngo.share / Release VA30C / 2021-12-16 / Revision 6272

syngo.share webview

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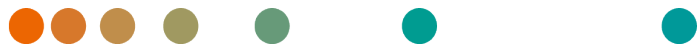
Online Help (English)

Contents

1	Introduction	51
1.1	Intended Use	51
1.2	Technical Terms and Notation	51
1.2.1	Generic Document	51
1.2.2	DICOM Document	51
1.2.3	DICOM Information	51
1.2.4	EXIF Information	52
1.2.5	Modality	52
1.2.6	Organizational Unit	52
1.2.7	Hounsfield Unit	52
1.3	Safety Notices	53
	Image and Result Distribution	53
	Medical Education	53
	Accuracy of Calculations	53
	Units	53
	Access Restriction	53
	Printing of Images	53
	Effects of External Programs	54
	Network Connection	54
	Workstation Accessibility	54
1.4	Supported File Formats	54
1.4.1	DICOM Storage SOP Classes	54
1.4.2	Generic Files	55
2	Usage Instructions	56
2.1	Notes Regarding the Usage of <i>syngo.share</i> webview	56
2.2	Getting Started	56
2.2.1	General Actions	56
2.2.1.1	Language	56
2.2.1.2	Date Format	56
2.2.1.3	Adjusting the User Interface	57
2.2.1.4	Full-Screen Mode	57
2.2.2	Login and Logout	57
2.2.3	Changing the Password	58
2.3	Patient Search	58
2.3.1	Search Fields	58
2.3.1.1	Free Text	59
2.3.1.2	Selection List	60
2.3.1.3	Date	60
2.3.1.4	Keyword	61
2.3.2	Saving a Search Query	61
2.3.2.1	Editing a Saved Search Query	62
2.3.2.2	Setting a Search Query as Default	62
2.4	Entity Search	62
2.5	Results List	62
2.5.1	Results List Columns	63
2.5.2	Customizing the Results List	63
2.5.3	Setting Automatic Updates for the Results List	64
2.5.4	References	64
2.5.5	Other Examinations	64

2.5.6	QR Code for Patient ID	65
2.6	Actions	65
2.6.1	Opening and Loading Documents	66
2.6.1.1	Opening Documents in the Viewer	66
2.6.1.2	Restoring and Fetching Documents	66
2.6.1.3	Opening Documents in External Applications	66
2.6.1.4	Opening Documents in Concentriq Dx®	67
2.6.2	Quality Assurance Operations	67
2.6.2.1	Deleting Documents	67
2.6.2.2	Copying Documents	68
2.6.2.3	Moving Documents	68
2.6.2.4	Reassigning Documents	68
2.6.2.5	Rearranging Studies and Series	68
2.6.2.6	Changing Document Properties	69
2.6.2.7	Setting a Workflow State	70
2.6.2.8	Rotating Generic Images	70
2.6.2.9	Setting the Thin Slice State	70
2.6.2.10	Viewing and Correcting Warnings and Errors	70
2.6.3	Keyword Tagging	70
2.6.4	Searching Externally	71
2.6.4.1	Searching in Other <i>syngo.share</i> Applications	71
2.6.4.2	Searching for External Documents	71
2.6.5	Distributing Documents	72
2.6.5.1	Sending Documents	72
2.6.5.2	Exporting Documents	72
2.6.5.3	Downloading Documents	73
2.6.5.4	Registering Reference Pointers	73
2.6.5.5	Registering Documents in the Patient Portal	73
2.6.5.6	Sharing Documents	73
2.7	The Viewer	74
2.7.1	Layout	74
2.7.2	Pointer	74
2.7.3	Scaling Tools	74
2.7.4	Scrolling	75
2.7.5	Pan	75
2.7.6	Windowing	75
2.7.7	Calculation Tools	75
2.7.7.1	Distance	75
2.7.7.2	Angle	76
2.7.7.3	Cobb Angle	76
2.7.7.4	Density	76
2.7.8	Drawing and Annotation Tools	76
2.7.8.1	Rectangle/Square	76
2.7.8.2	Ellipse/Circle	76
2.7.8.3	Freehand	76
2.7.8.4	Text	76
2.7.9	Reset	76
2.7.10	Image Navigator (Localizer)	77
2.7.11	Transformation	77
2.7.12	Calibrating Calculations	77
2.7.13	Inversion	77
2.7.14	Synchronized Scrolling	77

2.7.15	VOI LUT Presentation	78
2.7.16	Presets	78
2.7.17	Presentation States	78
2.7.18	ECG Tools	78
2.7.18.1	Metadata	78
2.7.18.2	Caliper	78
2.7.18.3	Time Calibration/Paper Feed Speed	79
2.7.18.4	Voltage Calibration	79
2.7.18.5	Channel Layout	79
2.7.18.6	Channel Group	79
2.7.18.7	Fit in View	79
2.7.19	Whole Slide Images for Digital Pathology	79
2.7.20	Playback	79
2.8	Inbox	80
2.8.1	Actions	80
2.8.2	Requesting Documents	81
2.8.3	Availability Period	81
2.9	Archiving Documents	82
2.10	Job Queue	83
2.11	Keyboard Shortcuts <i>syngo.share</i> webview	84
3	Emergency Access	85
4	Mobile Devices	86
4.1	Functionality	86
4.1.1	Search Results List	86
4.1.2	Viewer	87
4.2	Touch Gestures	87

1 Introduction

1.1 Intended Use

syngo.share webview is a web application to be used in clinical image and result distribution and provides access to clinical data from connected systems (such as Vendor Neutral Archives, Picture Archiving and Communication Systems, DICOM archives, XDS-I Affinity Domains, Electronic Medical Record Systems, Clinical Information Systems) and the local file system for the electronic display of data. This product is not intended for diagnosis, prevention, monitoring, prediction, prognosis, treatment, cure or alleviation of diseases or other medical conditions.

1.2 Technical Terms and Notation

1.2.1 Generic Document

This term refers to any non-DICOM file that can be viewed in or archived by *syngo.share*. Non-DICOM files include images produced by digital cameras or imaging applications (e.g. JPEG, MPM, TIFF) as well as various text, video, and audio file formats (e.g. PDF, DOC, MPEG4, MP3) produced by various external applications (e.g. Microsoft Office, Adobe Acrobat).

Generic files are wrapped in generic containers, analogous to DICOM studies.

1.2.2 DICOM Document

A DICOM document consists of at least one DICOM study, series and instance.

DICOM documents are produced by medical imaging devices compliant with the DICOM standard. They can be viewed in or archived by *syngo.share*.

Study

A DICOM study contains all the data of a medical examination during which image data is acquired. This includes the actual image data (e.g. X-ray, CT, ultrasound) as well as the patient data and descriptions relevant to the examination (such as the examination date or type). A DICOM study contains 1-n DICOM series.

Series

A DICOM series usually equates to a specific type of data acquired/produced by a particular device or to the position of the patient during acquisition. A series contains 1-n DICOM instances.

Instance

A DICOM object instance (or simply DICOM instance) is the actual data captured by the device, such as an image or waveform object.

1.2.3 DICOM Information

DICOM (Digital Imaging and Communications in Medicine) is a standardized file format used in medical imaging.

In addition to object data (typically image data), the DICOM format contains additional information such as patient name, admission date, device parameters, and physician's name. The DICOM format also contains the information necessary for data exchange between DICOM-capable devices.

1.2.4 EXIF Information

EXIF (Exchangeable Image File Format) is a standardized data format generated by, e.g., digital cameras when taking a picture.

In addition to the image data commonly contained in JPEG or TIFF format, the EXIF format contains additional information describing the image in more detail, such as date and time information or the type of device used for image capture.

1.2.5 Modality

An imaging modality is a device for acquiring medical images such as X-rays or CTs. In *syngo.share* webview, modalities are identified by abbreviations such as CR for Computed Radiography (e.g. when searching).

1.2.6 Organizational Unit

An *organizational unit* is an association of arbitrary actors in a medical organization, e.g. a department, medical devices, or groups of people. Organizational units can be customized and expanded with more constituents as needed, and these constituents can be uniformly managed.

All data to be archived is allocated to exactly one organizational unit.

1.2.7 Hounsfield Unit

Density calculations of body tissue with the density calculation tool use the Hounsfield scale. The following equation is used to calculate the attenuation coefficients of various body tissues in CTs. It is assumed that the absorption value of tissues and the physical density have a linear dependence (μ = linear attenuation coefficient):

$$HU = \left(\frac{\mu_{Object} - \mu_{Water}}{\mu_{Water}} \right) \times 1000$$

Table 1: Relative Attenuation Coefficients of Body Tissues and Water

Coefficient	Object
0	Water
1000 bis 100	Bone
100 bis 0	Liver/Muscle/Kidney
-100 through -200	Fat
-1000	Air

The relative attenuation values can be determined easily from any point or area of the image. Although it is technically possible to render any number of attenuation values in grayscale, the human eye is not equipped to distinguish between more than 20 shades of gray. Instead, the use of windowing is recommended in order to visually highlight different kinds of body tissue. ²

Please read the safety notices, see [Section 1.3](#).

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1.3 Safety Notices

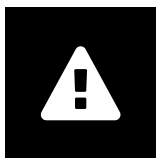
Image and Result Distribution

syngo.share webview is only intended for clinical image and result distribution.
Do not use *syngo.share* webview for direct diagnoses.

Medical Education

The use of *syngo.share* webview should be granted to qualified staff members only.

Accuracy of Calculations

Images may contain calibrations. Please note that the accuracy of these calibrations correlate to the precision of the program or modality with which they were created. Calibrations must be checked according to the manufacturer's instructions. Calculations, calibrations and results derived therefrom must be checked for plausibility by clinical professionals. Inaccuracies may arise during image capture by clinical professionals or due to the image capturing technology of a given modality. For instance, X-ray images may contain inaccuracies due to the fact that the x-rayed body part is projected onto a two-dimensional imaging surface. Inaccuracies during visualization may be due to browser or hardware settings.

This product rounds calculations as follows: Length and area: 1 decimal place; density (HU): no decimal place; area density (HU): 1 decimal place. This product uses an interpolation algorithm for image display.

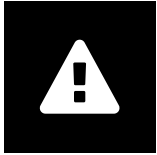
Units

The application *syngo.share* webview conforms to Directive 80/181/EEC, which contains specifications related to units of calculation. Calibrated lengths are calculated using millimeters. Surface areas are calculated in square millimeters (mm²). Plane angles are calculated in degrees. Densities are calculated using the industry-standard unit, Hounsfield units (HU), which are not regulated by Directive 80/181/EEC.

Access Restriction

Be sure to log out of *syngo.share* webview when finished working in order to avoid unauthorized access to the system.

Printing of Images

To print *syngo.share* webview transfers the data to the print function of the browser. Thus there may be differences between the view in the Image Display Area in *syngo.share* webview and the printout, e.g. the image could be cut off, annotations could be incomplete or missing entirely. Possible transformations such as mirroring, rotation, and applied presentation states are not marked explicitly in the printout.

Effects of External Programs

The software manufacturer does not guarantee that third-party applications will not negatively affect the system. The standards of any external programs accessed by *syngo.share* webview are set by their respective manufacturers.

Network Connection

syngo.share webview requires an active network connection. For IT security reasons, a (temporary) failure of the network connection may result in you being logged out and having to log in again.

Workstation Accessibility

When using *syngo.share* webview, make sure that you can operate the application at all times.

1.4 Supported File Formats

1.4.1 DICOM Storage SOP Classes

syngo.share webview specifically supports the DICOM Storage SOP Classes listed below. In addition, *syngo.share* webview supports viewing of all SOP Classes used for the purpose of image and video storage. Technical limitations (e.g. regarding the Transfer Syntax) may apply.

- 12-lead ECG Waveform Storage (1.2.840.10008.5.1.4.1.1.9.1.1)
- General ECG Waveform Storage (1.2.840.10008.5.1.4.1.1.9.1.2)
- Grayscale Softcopy Presentation State Storage (1.2.840.10008.5.1.4.1.1.11.1)
- VL Whole Slide Microscopy Image Storage (1.2.840.10008.5.1.4.1.1.77.1.6)

- Basic Text SR Storage (1.2.840.10008.5.1.4.1.1.88.11)
- X-Ray Radiation Dose SR Storage (1.2.840.10008.5.1.4.1.1.88.67)
- Encapsulated PDF Storage (1.2.840.10008.5.1.4.1.1.104.1)

1.4.2 Generic Files

syngo.share webview supports the following generic file formats for viewing:

BMP	JPEG (JFIF)	PDF
CDA (XML)	JPEG 2000	PNM
FLV	JPEG-LE	TIFF
GIF	JPEG-LS	VP8
H.264	MPEG1	VP9
H.265	MPEG2	WBMP
INDEO2	MPEG4	WMV1
INDEO3	PCX	WMV2
INDEO4	PNG	WMV3

2 Usage Instructions

2.1 Notes Regarding the Usage of *syngo.share* webview

Brief Differences of Document Statuses

For system performance improvement reasons, the status of documents in *syngo.share* webview may briefly differ between the individual user and the archive.

Search results always reflect the document status in the archive at the precise moment of the query. Note that the results list is not updated automatically if there are changes after the search was triggered. Either repeat the search manually or set an automatic update interval for the results list (see [Section 2.5.3](#)) to see the most recent data.

Cache of Document Data

When viewing documents, note that the document data is cached for a maximum of 10 minutes by default. This caching ensures faster reloading of documents and tool usage (e.g. windowing of DICOM images). Changes to cached documents only become visible upon their automatic removal from the cache. *syngo.share* core documents are cached on the user level.

Browser Zoom

syngo.share webview is not optimized for use with activated browser zoom. Please disable any zoom settings in your browser preferences.

Potential Differences in the Display of *syngo.share* webview and Proprietary Browser Features

Different webbrowsers render websites differently. This might lead to slight differences in the display. These differences do not affect the functionality of *syngo.share* webview. Different proprietary browser features, that can not be controlled by *syngo.share* webview, might interfere with the user interface (e.g. picture-in-picture display of videos, auto-fill, etc.). These features might have to be turned off in the browser settings.

Scope of Functions Described

These usage instructions provide a comprehensive description of the functions of *syngo.share* webview. Due to the configuration settings of your organization or missing permissions, some of these functions may not be available to you. If you have any questions, please contact your system administrator.

2.2 Getting Started

2.2.1 General Actions

2.2.1.1 Language

By default, *syngo.share* webview uses the browser language. You can change the language by clicking on your user name and selecting **Settings** from the drop-down menu. Confirm your changes with **OK** and reload the page.

2.2.1.2 Date Format

By default, *syngo.share* webview uses the date format of the tenant. You can change the date format by clicking on your user name and selecting **Settings** from the drop-down menu. Confirm your changes with **OK** and reload the page.

2.2.1.3 Adjusting the User Interface

You can hide or display certain parts of the user interface using small buttons.

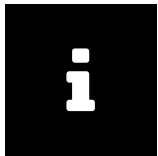
2.2.1.4 Full-Screen Mode

syngo.share webview can also be used in full-screen mode. You can activate and deactivate the full-screen mode by clicking on your user name and selecting **Full Screen** from the drop-down menu.

2.2.2 Login and Logout

On the login screen, enter your user name and password, select the domain (if applicable) and click on **Login**.

To log out, choose **Logout** from the drop-down menu next to your user name. Always terminate your sessions by logging out in order to prevent unauthorized access.

Note

Not all options may be available for your institution or workstation, meaning that you may not see some or any of these messages and/or the logout option.

At/after login, the following events may occur:

• Login message

You may be shown a login message. Click on **Accept** to finish logging in to *syngo.share* webview. If you accept the message once, you will not receive it again. If you click on **Decline**, your login will be canceled.

• Incorrect password entry

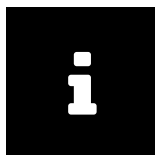
If you enter your password incorrectly too often, your user account will be locked for security reasons. The lock can only be undone by your system administrator.

• Screen lock or automatic logout

Depending on the configuration in *syngo.share* Configuration, *syngo.share* webview may engage a lock screen or initiate an automatic logout to prevent unauthorized access. The time remaining until screen lock or logout is visualized next to the user name.

↳ Screen lock

This function will blur the screen and display the lock screen. To continue working, enter your password on the lock screen. This unlocks *syngo.share* webview and you will see the current status of your work. You can also log out the current user on the lock screen in order to allow another registered user to log in.

Note

Note that logging out the current user means that this user's unsaved work will be lost.

↳ Automatic logout

Automatic logout logs out the current user and the login screen is displayed.

2.2.3 Changing the Password

To change your password, open the drop-down menu next to your user name, select **Settings** to be taken to your user profile settings. Click on **Edit** and enter your old as well as your new password in the respective fields. Save by clicking **Confirm**.

Note

Follow the instructions on the editing screen. These may inform you that your password needs to contain certain characters or be of a certain length. The **Confirm** button is only set to active if you fulfill all password conditions. Note that passwords are case-sensitive.

If the **Edit** button is deactivated, this means that your user access is managed by an external system (e.g. LDAP) and you are not allowed to change your password. Please contact your system administrator.

2.3 Patient Search

In the **Patient Search** tab, you can search for documents within a patient context. You can customize the available search fields according to your needs and refine or broaden your search by combining different search terms.

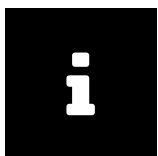
To customize and start a search, proceed as follows:

1. Click on  **Search fields** and select/ deselect search fields to add or remove them to/ from the search area. Your changes to the **Search** tab will be saved automatically.
2. Fill in the desired search fields. See [Search Fields](#) for details on the available types and input methods.
3. You may have more than one archive available for searching. In this case, you have to select at least one archive before you can start the search. Click on **Sources** to select one or multiple archives.
4. Click on **Search** or press **Enter** to start the search.
5. The results will be displayed in the results list below the search area. See [Results List](#) for information on customizing and working with the results list.
6. You can also save frequently needed search queries. See [Saving a Search Query](#) for more information.

2.3.1 Search Fields

syngo.share webview offers a variety of search fields in the categories patient, document, order, series, visit, and keywords (if defined). You can show or hide these search fields according to your needs (see [Patient Search](#)).

The input methods for search terms vary depending on the type of search field. However, clearing search fields always works the same: Click on  next to a search field to clear this particular field. You can also clear all search fields at once by clicking on .

Note

When searching via DICOM in third-party archives, some search fields cannot be considered, e.g. Body Part or the multi-selection of modalities. *syngo.share* webview indicates which fields have not been considered. If *syngo.share* webview is implemented without *syngo.share* core, the respective search fields are not available.

2.3.1.1 Free Text

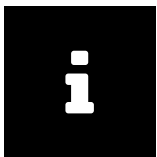
Most search fields are of the type free text, e.g. Last Name, First Name, or Patient ID. Here you can enter your search term directly. Possible leading and/or trailing spaces and/or tab characters are removed automatically. The search is not case-sensitive.

Pasting Text from the Clipboard

Text from the Windows clipboard can be pasted into free text search fields using the keyboard shortcut **CTRL + V**. Alternatively you can paste text from the clipboard directly with a right-click on the mouse. Depending on the browser you might have to allow pasting data from the clipboard in the browser settings for this mouse action. (This mouse function is currently not available in Mozilla Firefox.)

Placeholders

The % character can be used as a placeholder (wildcard) for all search activities and is automatically appended to the search. * can be used equivalently.

Note

The automatic addition of % does not occur in ID search fields (e.g. Patient ID, Accession Number, Visit ID) in order to avoid false positives when searching for patients via barcode scanner.

Example 1: Using the % Placeholder

Searching for M%er will show names such as Mustermann or Musterfrau but also Meier, Mayer or Moser or similar in the results list. If you only enter m, % is automatically appended and the program searches for all records that begin with m.

The _ character also serves as a wildcard, but only for one character. ? can be used equivalently.

Example 2: Using the _ Placeholder

If you search for m_ier: you will get Meier as well as Maier. Because % is appended automatically, e.g. Meierle and Maierle will also be found.

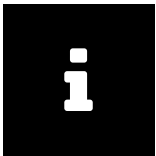
Special Free Text Search Fields

Name

This search field allows you to search for last name and/ or first name and different combinations thereof. The following searches are valid and will be mapped correctly to the fields first name and last name of the database:

- firstname lastname
- lastname, firstname
- lastname,
- , firstname
- lastname
- DICOM notation:
 - lastname^firstname
 - lastname^

- ^firstname
- lastname^firstname^^^
- lastname^^^^
- ^firstname^^^
- Any other combination of `firstname` and `lastname` with up to four carets that follows the standard DICOM notation (i.e. `lastname` always before `firstname`, always one caret preceding `firstname`).

Note

Spelling in the list above represents the way search terms have to be entered. If first name and last name are not separated by a comma or a caret, they may only consist of one word each, because in this case the space marks the separation between the two. Names containing hyphens or other special characters are considered as one word, e.g. Smith-Miller or O'Brien.

≡ **Mother's Maiden Name**

The mother's maiden name may contain special characters. Due to technical reasons, the search field does not support some of these special characters. If the name you are searching for contains one or more of the special characters described in [Table 4](#), please use a wildcard at this position to conduct your search (see [Placeholders](#)).

≡ **Description**

Description of an examination/study (contents of the DICOM tag `StudyDescription`). The placeholder % is automatically prepended and appended to the search term.

≡ **Body Part**

Here, you can filter for the part of the body examined (contents of the DICOM tag `BodyPartExamined`).

2.3.1.2 Selection List

In fields of the type selection list, e.g. `Modality`, `Document Type` or `Producer`, you select an entry by clicking on the arrow symbol and choosing from the predefined list.

Some selection lists allow the selection of multiple entries (e.g. `Modality` or `Producer`): Simply select one entry after the other. To remove an entry, click on the next to it.

Special Selection List Search Fields

≡ **Producer**

The department (Organizational Unit) which produced the images/documents. Only departments of which you are a member are displayed.

2.3.1.3 Date

In date fields, you can either enter the date directly or select it from the calendar. By selecting the date from the calendar, you can see the currently valid date format.

Special Date Search Fields

Study Date

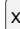
This search field combines a selection list with a date search. First, select a time period from the selection list. If the time period is related to the present day or the current time, no further input is required. However, if you want to search for a specific date (“on”, “from”, “to”) or a date range (“between”), one or more date fields will open after selecting the time period. There, you can enter the desired dates manually or select them from a calendar.

2.3.1.4 Keyword

You can also search for documents according to their keyword tags. The configurable keyword fields depend on the available keyword groups.

You can enter the desired keyword manually or search for keywords in the keyword catalogs. Click the  icon in the search field to open a list of keywords from all associated keyword catalogs. The catalogs show the keywords in hierarchical order; click on the arrow next to a keyword to expand the next level. To filter the list, enter part of the desired keyword.

Click on an entry in the list to add it to the search field. You can add more than one keyword to a search. The search automatically includes all keywords that are hierarchically below the selected level, e.g. if you select “J00-J06 Acute upper respiratory infections” from the ICD-10 catalog, all diseases/keywords in that section will be searched for. If you select “J01.4 Acute pansinusitis” on the lowest level, only results matching that exact keyword will be shown. Automatically included keywords are marked with a square, explicitly selected keywords are marked with a checkmark.

A search for keywords from catalogs is always an exact search. For searches in a free text catalog however, a placeholder is always added automatically at the beginning and the end of the text entered into the search field. These search terms are set in italics in the search field. To remove a particular keyword, click the  right next to it.

Note that if you run a search with a set keyword, the results will only include documents which have been tagged with this keyword on at least one tagging level. For more information on keyword tagging, see [Section 2.6.3](#).


2.3.2 Saving a Search Query

You can save frequently used search queries to avoid having to reenter them every time.

Example 3: Search Query: Display all CTs of a given day in the results list

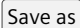If you have a daily CT meeting and want to see all CTs of that particular day in the results list, you can create a CT_Meeting search query, save it, and execute it by clicking on its name.


To save a search query, proceed as follows:

1. Customize and fill in the desired search fields and, if necessary, select your desired archive(s) as described in [Patient Search](#).
2. Optionally, execute the search query and adjust the display of the results list to your liking. Changes to the results list are saved automatically. See [Customizing the Results List](#) for details.
3. Once you are satisfied, click on  to save your query. A new window opens where you can name the search query.
4. Your query will appear as a separate tab next to the **Search** tab.

To execute a saved search query, simply click on the corresponding tab.

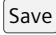2.3.2.1 Editing a Saved Search Query

You can rename or delete a saved search query by selecting it and clicking on the corresponding entry in .

To edit a saved search query, select it and apply the desired changes (e.g. hide/show search fields, enter additional search parameters), then click on  to overwrite the previous query. If you want to save the changes as a new search query instead, click on  and enter a new name. If you would rather not save the changes, click on  to revert them.

2.3.2.2 Setting a Search Query as Default

You can set a saved search query to be executed automatically as soon as you log in to *syngo.share* webview. Select the desired search query tab and click on . At your next login, *syngo.share* webview automatically switches to the patient search and executes the default search query.

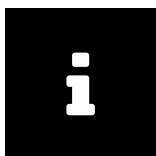
2.4 Entity Search

To search within an entity context click the **Entity Search** tab and choose an entity from the drop-down menu. The entity search works in a similar way to the patient search (see [Patient Search](#) for details), however, a limited number of search fields is available. Entity search queries cannot be saved.

2.5 Results List

After performing the search, the results are displayed in the results list. A result can be a DICOM study (examination) or a generic document (generic container).

You can refine the displayed results by entering a string in the filter field.

Note

Depending on permissions and/or system settings the results list might not show all documents or document parts matching your query. In case you are missing results, please contact your system administrator to check your permissions and/or settings.

You can set an automatic update interval for the results list (see [Section 2.5.3](#)). Be aware that the results list is static between two searches.

If the results list update is deactivated, changes to data in the background (e.g. deletion of a file from the archive) are not automatically shown in the list. In this case, click on  to update the results list manually. Changes you make to the search results (e.g. setting a new workflow state) are shown immediately.

Exporting Search Results

You can export the search results to a CSV file by clicking on the . Note that if you use the CSV export with an active filter, only the filtered results will be exported.

You can also copy individual column entries to the clipboard for further use, e.g. when filling out a form or spreadsheet. Right-click on the desired entry and select  from the context menu.

Previewing Documents

When you select an entry, the series contained therein are loaded in the **Series** tab. The **References** tab lists generic documents linked to the study, provided such documents exist (see [Section 2.5.4](#)). You can switch between a list view and thumbnail view in both tabs.

In the image preview below, you can choose between a list view of all images contained in the series and a preview image. By default, the first image in the selected series is shown in the preview.

The functions described above are also available for generic containers and their files.

2.5.1 Results List Columns

The results list contains several columns, e.g. patient ID, name, date of birth, modality, producer, and many more. You can show and hide columns according to your needs (see [Section 2.5.2](#)). Your view is saved automatically. Saved search queries (see [Section 2.3.2](#)) also save a changed display of the results list, which will be applied automatically when the search query is executed.

The content of the columns is self-explanatory in most cases; for certain columns, an explanation is provided below:

Online Information

The availability status of a document differs depending on whether the document is stored in *syngo.share* core or a DICOM archive. [Table 2](#) provides an overview of the various statuses.

Table 2: Online Information

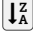
Status		<i>syngo.share</i> core		DICOM archive
Online	Green	Data is physically available in <i>syngo.share</i> core.	Green	Data is immediately available from the Retrieve AET.
Nearline	Orange	Data is stored in a third-party VNA or an LTA; metadata is available in <i>syngo.share</i> core. The document can be restored and viewed in <i>syngo.share</i> webview by double-clicking it.	Orange	Data can be retrieved, but is stored on relatively slow media and retrieval may take some time.
Offline		–	Red	Manual intervention is needed to retrieve the document.
Partially Online	Green/Orange	Parts of the study or series have the status "Online", other parts have the status "Nearline".		–
Unavailable		–	Red	The document cannot be retrieved.
Unknown		–	Gray	The third-party archive does not deliver the instance availability.

LTA State

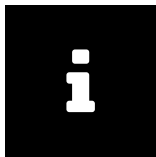
The LTA state is displayed if a document must be loaded from the long-term archive. If the document is online or the retrieval was successful, there is no entry in the column.

2.5.2 Customizing the Results List

The results list can be customized in many ways. You can choose between a list view grouped by patients ( Patient) or documents ( Document). For each view, the following adjustments are available:

- Move the columns freely by simply dragging them with the mouse.
- Adjust the column width by clicking and dragging the mouse.
- Sort the columns by clicking on the arrow shown on mouse-over in the column header and selecting  or .
- Sort the columns quickly by clicking on the column header.
- Show/hide the columns as desired. Click the arrow shown on mouse-over in the column header and (de)select the desired columns in the **Columns** list.

These settings are saved automatically. Note that for saved search queries, you have to manually save any changes you make via  **Save**. For more information on saved search queries, see [Section 2.3.2](#).

Note

Columns are not always sortable. This occurs when the system pre-groups and pre-sorts the results in the background. In these cases, subsequent manual sorting would interfere with this arrangement.

2.5.3 Setting Automatic Updates for the Results List

syngo.share webview automatically updates the results list if you set an update interval. You can choose among several intervals.

The results list update interval selected is valid for saved search queries as well as for manual searches. If a saved search query is set as default, the update interval starts with your login to *syngo.share* webview.

Warning

The automatic update of the results list causes a higher system load. The shorter the update interval, the higher the load. To avoid an unnecessarily high system load, use a longer duration between updates, or, if unneeded, disable the automatic update function.

2.5.4 References

There are several ways to reference DICOM studies with generic reports. The Study Instance UID has the highest relevance for referencing documents. Thus, *syngo.share* webview first tries to identify referenced documents using the Study Instance UID. Only if no references for the given Study Instance UID are available, the Accession Number in combination with the Assigning Authority is considered to identify referenced documents. In any case references are limited to the same patient, i.e. the Patient ID has to be the same for all linked documents.

If available, the references are ordered by their generic file type. References that do not have a generic file type assigned are listed in a separate section at the end of the references list. The generic file type can be changed or assigned in the document properties (see [Section 2.6.2.6](#)).

2.5.5 Other Examinations

In the **Other Examinations** area below the results list, you can view available other examinations for found DICOM studies. The **Other Examinations** area is collapsed by default; click on  to expand it.

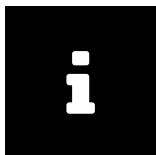
To view other examinations, proceed as follows:

1. Expand the area and select a study from the results list to automatically load matching other examinations.
2. Alternatively, you can use the results list context menu entry **Show other examinations** to expand the area and load the matching examinations.
3. You can filter the loaded examinations by prior examinations (checked by default), modality, or body part.
4. You can also add/remove, rearrange and sort the various columns just like in the results list (see [Customizing the Results List](#)).

Select an other examination to receive a preview of the contained series and instances. For guidance purposes, the current study remains highlighted in the results list.

Double-click on an other examination to open it in the Viewer. To display an other examination alongside the initial examination, choose **Compare with initial examination** in the context menu of the **Other Examinations** area. You can then view and edit the images contained in either examination (see [Section 2.7](#)).

Note that you can change the position of the initial examination in the user menu via **Settings** **Other Examinations**. If multiple Viewer tabs are configured, the context menu entry **Compare with initial examination** will open a new tab for the comparison even if there already is an open tab for the same patient.

Note

Note that other examinations are always loaded from the same archives selected for the initial search. If you suspect other examinations in other archives, you have to first select these archives to be included in the search. See [Patient Search](#) for details.

2.5.6 QR Code for Patient ID

A QR code representing the patient ID can be scanned with a mobile device and open the corresponding patient record e.g. in the Visual Capture application. You can access the QR code via   next to the patient's name from any location in *syngo.share* webview where patient information is displayed, e.g. in the patient-grouped results list or the patient tab in the Viewer.

2.6 Actions

Actions can be found by opening the context menu (right-click) in the search results list. The available actions depend on the entry level, your user permissions and whether the actions can be applied to the currently selected data. If an action is not shown, you are missing the corresponding permission (user permissions are assigned by your system administrator). If an action is not applicable to the currently selected data, it is grayed out.

The selected action is executed on the level on which it is called: on the patient level (in the patient-grouped view), on the DICOM study level or generic document level, DICOM series level, or DICOM instance or generic file level. If you call an action on the DICOM study or generic document level, it is applied to all files contained therein.

For studies from third-party archives or any installation without *syngo.share* core, only a reduced number of actions is available.

The following sections describe the most important context menu actions.

2.6.1 Opening and Loading Documents

syngo.share webview offers several means of opening and loading documents.

2.6.1.1 Opening Documents in the Viewer

You can open any part of a document in the Viewer by either double-clicking on it or selecting **Open** from the results list context menu. Note that if you open a lower-level item, the corresponding higher-level item(s) will also be opened in the **Document Preview** of the Viewer. For example, if you open a DICOM series in the Viewer, the corresponding study and any other series it contains will also be displayed in the **Document Preview** (see [The Viewer](#)).

Opening Additional Documents

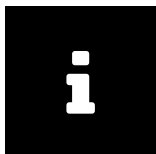
The Viewer tab stays open when you switch back to the search tab. Like this, you can easily open additional documents for the currently displayed patient. The documents are added to the **Document Preview**. The already opened viewports remain in place. To view a newly opened document, either change the layout (see [Section 2.7.1](#)) to have more viewports available and open the document in an empty viewport, or drag the document into an active viewport to replace the displayed document.

Opening Multiple Documents at Once

You can open up to nine DICOM studies and/or generic containers of the same patient for viewing at once. The documents have to be in the same archive. Press **Ctrl** and click to select multiple documents, then select **Open** from the context menu to open all the selected documents in the Viewer. Each document will be opened in its own viewport in one Viewer tab.

Opening Documents from Multiple Patients

Depending on the configuration, you can open up to three Viewer tabs for different patients and/or entities. Adding more documents to already displayed patients/entities is possible without limitation.

Note

Please be aware that any quality assurance operations (see [Section 2.6.2](#)) performed on an opened document will only become visible once you close and reopen the document.

2.6.1.2 Restoring and Fetching Documents

Documents that are stored in a long-term archive and have the status *Nearline* can be restored. Once restored, the documents can be displayed in *syngo.share* webview and all actions are available as usual.

The **Fetch** function allows you to send studies and series from a DICOM archive to a Move Destination AET via a C-MOVE operation (the Move Destination AET is defined in *syngo.share* Configuration). Only documents in third-party archives that have the status *Online* or *Nearline* can be fetched. If no Move Destination AET is defined, the context menu entry is not available.

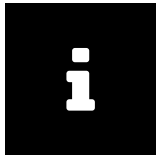
For more information on the online status, see [Section : Online Information](#) .

2.6.1.3 Opening Documents in External Applications

syngo.share webview allows you to open documents in external applications, e.g. another viewing application or a print application. These external applications are configured in *syngo.share* Configuration and are available in the context menu. For opening documents, one or more default applications can be set that all open simultaneously when you double-click on the desired study or series.

2.6.1.4 Opening Documents in Concentriq Dx®

You can open generic documents in Concentriq Dx®, a platform for digital pathology. Select the desired document and click on **Open with Concentriq Dx®** in the context menu. You can also select and open multiple documents. A new browser tab will open for each document.

Note

This action is only available for documents which originate from Concentriq Dx®. In order to use this feature, access to Concentriq Dx® is required.

2.6.2 Quality Assurance Operations


syngo.share webview offers a variety of functions to assure quality of the data archived. [Table 3](#) provides an overview of the functions, states their availability for DICOM and generic data, and links to the corresponding sections.

Table 3: QA Operations in syngo.share webview

Function	DICOM	Generic	Description
Change patient metadata ^a	✓	✓	Changing Document Properties
Change study/document metadata	✓	✓	Changing Document Properties
Change series metadata	✓	✗	Changing Document Properties
Change workflow state	✓	✗	Setting a Workflow State
Move series	✓	✗	Reassigning Documents, Rearranging Studies and Series
Move study	✓	✓	Moving Documents, Reassigning Documents
Merge series	✓	✗	Rearranging Studies and Series
Merge study	✓	✗	Rearranging Studies and Series
Copy study	✓	✓	Copying Documents
Split study ^b	✓	✓	Copying Documents, Deleting Documents
Delete study	✓	✓	Deleting Documents
Delete series	✓	✓	Deleting Documents
Delete report (generic document)	✗	✓	Deleting Documents
Rotate image (generic file)	✗	✓	Rotating Generic Images
Change thin slice state	✓	✗	Setting the Thin Slice State

^a Patient data can only be changed for non-HL7-managed patients.

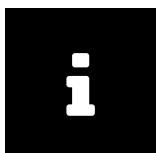^b via Copy, Delete

Note

While quality assurance (QA) operations are being carried out on a document, the corresponding study or generic container is locked in order to prevent multiple users from executing QA operations at the same time. Users can still view the document but will be informed of the ongoing QA operations via warning.

2.6.2.1 Deleting Documents

To delete documents from the archive, use the **Delete** function in the context menu. This function is available on the studies and series level, or, for generic files, on the container and file level. When

deleting a document, a dialog box opens where you have to select a reason for the deletion from the drop-down menu. The available reasons for deletion depend on the settings made by your system administrator.

2.6.2.2 Copying Documents

This function allows the copying of documents to a different producer (Organizational Unit) and make them available to other users. Choosing **Copy** opens a dialog window with a selection list of possible storage areas. Specific entries can be filtered via the search field.

Selecting an Organizational Unit and confirming with **OK** starts the process. The document will subsequently exist twice in the archive, once in each Organizational Unit. Any keywords and annotations are also copied.

2.6.2.3 Moving Documents

This function lets you move documents to a different producer (Organizational Unit). Choosing **Move** opens a dialog window with a selection list of possible storage areas. Specific entries can be filtered via the search field.

Selecting an Organizational Unit and confirming with **OK** moves the selected study or file to the specified Organizational Unit. The results list is updated immediately.

2.6.2.4 Reassigning Documents

A document can be reassigned to another patient or visit; furthermore, DICOM studies and series imported incorrectly can be reassigned to the appropriate requested procedure (on the study level) or scheduled procedure step (on the series level) of HL7 orders. To reassign a document, select it in the results list and choose **Reassign** in the context menu.

Reassigning Documents to Another Patient or Visit

To reassign a document to another patient or visit, select **To another patient/visit**. The patient and visit search screen is shown. You can either search for the exact visit ID or for the patient. Various search fields are available for the patient search. Note that you cannot search for a visit ID and patient simultaneously.

Select the entry to which the document should be reassigned in the results list and click **OK**.

Reassigning Documents to Another Order

To reassign a DICOM study to a requested procedure or a DICOM series to a scheduled procedure step, select **To another order**. Search for the correct order. You can fill several search fields to narrow the results, however, you may only search for either the order ID or requested procedure ID.

Select the entry to which the DICOM study or series should be reassigned in the results list and click **OK**.

2.6.2.5 Rearranging Studies and Series

syngo.share webview allows you to rearrange DICOM studies and series via a cut-and-paste operation. This function is useful for correcting acquisition errors. The following operations are available:

- Merge studies
- Merge series (of the same modality)

- Move series to another study

To rearrange studies or series, take the following steps:

1. Select the desired studies or series in the corresponding results list. Press **Ctrl** to select more than one study or series.
2. Open the context menu and choose on **Cut**. The selected studies or series are cut to the clipboard.
3. Search for the target study or series and select the desired target document.
4. Open the context menu on the desired target study or series and click **Paste**. A dialog box opens informing you about what exactly will be done and what will be the consequences of the desired operation. Confirm the operation by choosing whether you want to keep or delete the source document. The copied studies or series are pasted into the target study or series, and the source is deleted or kept depending on the previous selection.

Once the rearrange operation is completed, the results list is reloaded and the target document is selected. If you chose to delete the source document, it will no longer be listed.

As long as a document is part of a rearrange operation, it is locked and highlighted in color. You cannot cut and paste a locked document or merge it with another document.

The cut documents remain in the clipboard until

- they are pasted into another series or study. Upon completion of the operation, the clipboard is cleared.
- new documents are cut to the clipboard. In this case, the previously cut documents are overwritten.

2.6.2.6 Changing Document Properties

You can access the document properties from the context menu on any level of the results list. Depending on the entry level, click on **Properties of patient** (available in the patient-grouped results list), **Properties of study**, **Properties of series** or **Properties of instance** or, for generic documents, on **Properties of document** or **Properties of file**. This opens a dialog window containing tabs for every available level of the document, with the tab of the current entry level set to active.

You can now make the desired changes. Fields or tabs that cannot be modified are deactivated or hidden. Once you confirm the changes, the entry changes immediately.

Mother's Maiden Name

The field mother's maiden name is only available for Spanish-speaking countries. It allows several special characters, however, these have to be entered using the escape sequences described in [Table 4](#):

Table 4: Escape Sequences

Special Character	Escape Sequence
\	\\
	\\
~	\\~
^	\\^
&	\\&

2.6.2.7 Setting a Workflow State

syngo.share webview allows you to define the workflow state of a document. Open the results list context menu and select the desired step listed under **Workflow State**.

The steps of a workflow are defined by your system administrator in syngo.share Configuration.

Example 4: Possible Steps of a Workflow

- New
- Prepared
- Reported
- Signed-off

2.6.2.8 Rotating Generic Images

In the image preview, you can rotate JPEG images clockwise and counterclockwise in steps of 90 degrees. This function is only available from the preview image, not from the list view. The rotation can be saved. Please note that only the rotation, no other operation on the image, is saved.

2.6.2.9 Setting the Thin Slice State

On the series level, you can (de)select a checkbox to specify whether a DICOM series is a *Thin Slice Series*. Thin Slice Series are specially highlighted in the thumbnail view.

2.6.2.10 Viewing and Correcting Warnings and Errors

syngo.share webview provides a document error and warning log. An error log is created when an error occurred during the import of DICOM files. In case corrective measures set in the DICOM configuration are taken during a re-import, a warning log is created.

To find files with an error or warning, use the search filter **Document Log**.

Follow these steps to access the document log and correct entries:

1. Right-click on a document in the results list to open the context menu and select **Show document log**.
2. A new window opens and lists all warnings and errors logged for the selected file.
3. Click on the desired entry to see the entire log entry.
4. If the warning is acknowledged or the error is resolved, e.g. after a re-import, you can delete it from the list. Select it and click **Set as resolved**. You can select several entries and resolve them all in one step. For traceability you may also keep the entries in the document log.

To resolve issues use the command line tools available. For more information please see the respective chapter in the OnlineHelp syngo.share core.

2.6.3 Keyword Tagging

syngo.share webview lets you apply keywords to documents. Documents tagged with keywords can be searched for and grouped by their keywords. Keywords can be based on catalogs (e.g. ICD 10) and/or

on free text. The advantage of catalogs is that they contain exactly defined keywords which can be easily searched. The advantage of free text keywords is that any desired keyword can be applied to a document. However, the disadvantage is that it might become difficult to find entries when a keyword has multiple forms of expression, e.g. "femoral fracture" and "broken thigh bone".

☰ Catalog

This contains predefined keywords.

☰ ICOTAGS

This is a special catalog that allows only one keyword to be set. This exclusive behavior is useful for keywords like "diagnosis-relevant".

☰ Free Text

These keywords are not restricted to a predefined catalog. Any desired keyword can be applied to a document.

You can assign any number of keywords from any level of the keyword trees to studies, series and images as well as general documents and generic files. If keywords were incorrectly assigned, they can be easily removed as well.

To assign keywords on the various levels, right-click on the element on the desired level and choose **Keywords** > **Add keyword** from the context menu. The form to assign keywords opens. If available, keywords from the *ICOTAGS* catalog are displayed directly in the context menu entry **Keywords** for quick selection.

The keyword groups can be chosen from a selection menu. If a catalog is extendable, you can enter a new keyword directly in the text field. It is also possible that no keyword catalog has been assigned to a field. These are so-called free-text catalogs which allow keyword tagging via manual text entry.

Click the arrow key next to the keyword search field to view the keywords of the respective keyword catalog. Navigate the tree to find the desired keyword or use the filter function to search for the keyword in the catalog. To actually apply the keyword tagging, select and click the appropriate entries in the catalog. The keyword tagging is saved when you click **OK**.

To remove a keyword, open the properties of a study, series or instance (or a generic document or file) and remove the desired keywords by clicking **x**. Confirm your changes with **OK**.

2.6.4 Searching Externally

You can search for documents in other *syngo.share* applications or across healthcare institutions.

2.6.4.1 Searching in Other *syngo.share* Applications

Use **External search** to find patients or visits in *syngo.share* import or *syngo.share* view. To use this function, *icourl* must be installed.

2.6.4.2 Searching for External Documents

In the patient view (**👤 Patient**), you can use **Search for external documents** to start a cross-enterprise search (XDS search) for documents for a given patient. To use this function, the XDS search must be configured in *syngo.share* Configuration (requires an eHealth Solutions Consumer Adaptor).

The found external documents are displayed in a pop-up window which offers many of the same customization options as the regular search results list (e.g. showing/hiding columns, sorting or filtering

the results). You can open and edit the external documents in the Viewer. Note, however, that external documents are not added to the **Patient Jacket** and cannot be archived.

2.6.5 Distributing Documents

syngo.share webview offers various means of distributing documents for further processing or use, such as sending documents to DICOM-capable applications, sharing documents with colleagues or downloading documents to a local storage device.

2.6.5.1 Sending Documents

Documents can be sent to all DICOM-capable applications or devices integrated with the *syngo.share* system. These devices are shown as recipients (targets) and are configured by your system administrator. You can call the **Send** function on any level of the results list and accordingly send entire studies, series, single images, or generic containers.

To send a document, proceed as follows:

1. Select the desired document. You can select multiple documents by pressing **Ctrl** or **Shift** while clicking.
2. Right-click to open the context menu and choose **Send**.
3. You can select bookmarked recipients directly or click on **Other targets** to see the complete list of all available recipients.
 - If there are many possible recipients, enter part of the recipient's name in the filter field to narrow your options.
 - If you have added recipients to your list of favorites, they will be listed separately (see [List of Favorites](#)).
4. Click to select the desired recipient from the list. To select multiple recipients at once, press **Ctrl** or **Shift** while clicking.
5. Confirm your recipient(s) with **Start sending**. You can track the progress of send operations in the **Job Queue**.

List of Favorites

The list of favorites usually contains preconfigured entries (although it may be initially empty). These favorites are available for quick selection in the context menu and are listed separately in the list of all recipients. You can adjust the list of favorites according to your needs:

- To add an entry to your list of favorites, click on its star symbol.
- To remove the entry, click on its star symbol in the list of favorites.

2.6.5.2 Exporting Documents

Archived documents and images can be exported to a local data storage device. The **Export** function opens a panel where you can choose between ZIP and ISO as export format. The options available for each format are described below. For both formats you can select anonymization of DICOM patient data for privacy protection.

Exporting a ZIP file

When exporting the selected documents as ZIP file you can choose to encrypt the file. In this case, you have to set a password. Furthermore, *syngo.share* webview can create subfolders for the individual DICOM series during export. According to your selection, these subfolders will be named after the contents of one of the following DICOM tags:

- **SeriesInstanceUID**
- **SeriesDescription**: If the **SeriesDescription** tag contains any of the special characters `\/:*?"<>|`, they will be replaced by `_` in the subfolder name. If **SeriesDescription** is empty, the subfolder will be named after **SeriesInstanceUID**. If two or more series have the same **SeriesDescription**, the duplicate subfolder names will be numbered.

Exporting an ISO file

When exporting the selected documents as an ISO file, a DICOMDIR is created and wrapped into an ISO file ready to be burned to a CD/DVD with an external application. You can opt to include files which cannot be converted to *Explicit VR Little Endian* transfer syntax (e.g. videos). Please be aware that the DICOMDIR created in this case does not meet the DICOM standard. You may furthermore choose to use ISO 9660 level 1 for limited length of filename and directory name (8.3 filename convention). Please be aware that this may lead to issues if the export includes one or more DICOM instances that exceed 4 GiB.

2.6.5.3 Downloading Documents

The **Download** function saves your selected documents as an unencrypted ZIP file to a local data storage device. It presents a one-click alternative to the **Export** function (see [Exporting Documents](#)).

2.6.5.4 Registering Reference Pointers

You can manually register documents in order to make them available to subsystems. This means that for each selected document, a reference pointer containing all relevant information about the document is sent to the subsystem.

Select the documents to be registered and choose **Register reference pointer** from the context menu. If you have bookmarked favorites you can select them directly. To open a list of all possible recipients click **Other targets**. If there are many entries in your list of recipients, enter part of the recipient's name in the filter field to filter your options. Select the desired recipient(s) and click on **Send**.

For more information on the list of favorites, see [Section 2.6.5.1](#).

2.6.5.5 Registering Documents in the Patient Portal

This function registers documents to the eHealth Solutions patient portal. Via multi-selection (**Ctrl** or **Shift**), you can choose and register multiple documents or generic files at the same time.

2.6.5.6 Sharing Documents

This function allows you to release examinations to physicians in private practice as well as medical institutions for electronic diagnosis communication. Select the desired document(s) and choose **Share** from the context menu. Enter the relevant information in the dialog window:

☰ Recipient

The selection list contains all possible recipients, meaning these users are registered participants of the electronic diagnosis communication in the healthcare facility. Either browse the list or enter a term in the search field to filter your results. You can select one or more recipients at once.

☰ Validity period

You can add a validity date for your share. To allow permanent access to your share, check the box **indefinitely**. The default validity date is set by your institution.

☰ Priority

Check the box to mark a share as high priority.

☰ Message

Below the list of recipients, you can add a message to be sent to all selected recipients.

Click on to release the document(s). Every recipient will receive a notification in their **Inbox** (see [Section 2.8](#)) as well as an e-mail notification (if they have provided an e-mail address).

2.7 The Viewer

The Viewer is intended to be used for viewing purposes only. It provides a number of viewing tools, such as zoom and windowing, and includes options for displaying or hiding image information like overlays and calculations. Most of these tools can be found in the global toolbar in the upper part of the Viewer. Depending on the file type, additional tools are available in the specific toolbar.

Note

Editing steps are not saved; the Viewer is only used for viewing.

The Viewer supports many documents and image formats in addition to DICOM images. For unsupported file formats, a notice is displayed.

In case of problems when viewing a video, it may have been imported with a variable bitrate.

The following sections provide an overview of the most important Viewer functions. Availability and applicability of these tools depend on the file type being displayed. You can map certain tools to your mouse buttons in the settings menu (.

2.7.1 Layout

The Viewer offers the possibility to display several documents side by side. Various layouts are available.

All viewports showing series of the same study or files of the same generic container are marked with the same color. This color highlighting can be switched off in the **Settings**, available from the drop-down menu next to your user name.

2.7.2 Pointer

Using the pointer available drawings and annotations can be selected for editing.

2.7.3 Scaling Tools

The Viewer offers various scaling tools, e.g. the magnifier or zoom. The zoom tool allows you to scale the image smoothly by pressing and holding the left mouse button. In the specific toolbar, further scaling options are available.

Magnifier

The magnifier function enlarges a portion of an image and can be used on both DICOM and generic images.

Clicking on the desired area in a image enlarges said area in a separate window. The scale factor is controlled by the mouse wheel and can be increased to a maximum of 10 and reduced to a minimum of 1. If you click and drag the mouse, the magnifier window will follow your movement while maintaining the set scale factor. Click the left mouse button again to close the window.

Annotations are not considered when using the magnifier, however, any overlays that have been saved with the image will be shown.

2.7.4 Scrolling

This tool allows you to scroll backwards and forwards through series and multiframe images simply by moving the mouse up or down.

2.7.5 Pan

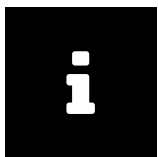
The pan tool allows you to move the image in the desired direction while pressing the left mouse button.

2.7.6 Windowing

The windowing function controls brightness and contrast in DICOM images. Move the mouse up or down while pressing the left button to adjust the brightness. To adjust the contrast, move the mouse left or right.

2.7.7 Calculation Tools

You can perform various calculations using the following tools.

Note

Calculations will not be stored within an image. Calculations are preserved in the image until the button is clicked or a new series is loaded. Please read [Section 1.3](#).

2.7.7.1 Distance

Clicking on the distance calculation tool changes your mouse cursor to a crosshair. Left-click on the starting point of your calculation and move the mouse in the desired direction to draw a line. Left-click again to set the end point. The calculation is now fixed and its length is displayed next to the line in millimeters or pixels. The unit of length shown depends on the image. You can make adjustments as long as the image or DICOM series remains open in the Viewer.

Usually, DICOM images already include a calibration, and thus distance is calculated in millimeters (mm). Generic images do not necessarily provide calibration information and thus the length is shown in pixels. It is also possible to manually calibrate an image, see [Section 2.7.12](#).

2.7.7.2 Angle

This tool calculates an angle. Once activated, simply click on the image in multiple locations to set the angle.

2.7.7.3 Cobb Angle

This tool calculates a Cobb angle. Once activated, simply click on the image in multiple locations to set the angle.

2.7.7.4 Density

This tool performs spot density calculations in CT images (e.g. bone density). Move the mouse over the image to see the density in "Hounsfield" units ("HU") for any spot. You can set a density spot calculation by left-clicking on the image. The density for this spot is now displayed. It is possible to set any number of density spot calculations.

2.7.8 Drawing and Annotation Tools

These tools allow you to add graphic and text annotations to images. You can only edit or delete annotations as long as the image or DICOM series remains open in the Viewer. Annotations only exist temporarily and are not saved. Certain annotations have restrictions which are explained in a note. To remove an annotation, select it using the pointer tool ([Section 2.7.2](#)) and press **DEL**.

2.7.8.1 Rectangle/Square

Use this tool to draw a rectangular shape and calculate its area. For CT images, the average density is also calculated and displayed in HU. To draw a square, press and hold **SHIFT**.

2.7.8.2 Ellipse/Circle

Use this tool to draw a circle or ellipse on the image and calculate its area. For CT images, the average density is also calculated and displayed in HU. To draw a perfect circle, press and hold **SHIFT**.

2.7.8.3 Freehand

This tool lets you draw any desired shape on the image.

2.7.8.4 Text

Use this tool to add textual descriptions to the image.

2.7.9 Reset

This resets all editing steps. Note that this function does not affect manual calibration, see [Section 2.7.12](#).

2.7.10 Image Navigator (Localizer)

When scrolling through multiple overlapping slice image series, the position of the current slice can be shown in the other series for reference. The reference point in another series is called a *Localizer* in the DICOM standard. This function can only be applied to series with certain anatomical positioning characteristics (e.g. orientation, reference point).

2.7.11 Transformation

Use transformation tools to rotate the image clockwise or counterclockwise in 90-degree steps. You can also flip the image vertically and horizontally.

2.7.12 Calibrating Calculations

Use this tool to manually calibrate calculations on DICOM and non-DICOM images. Note that DICOM images often already come with calibration information that is part of the DICOM image or is provided through a DICOM display preset called a *DICOM Presentation State*. If you add a manual calibration, it will be applied to all previously made annotations within the image. In the case of a *DICOM Multiframe Image*, i.e. an image that consists of several subimages, the change will be applied across all subimages.

You can only reset the calibration by using the calibration tool menu rather than the reset function, see [Section 2.7.9](#).

2.7.13 Inversion

The inversion tool displays a negative of the image, i.e. white parts of the image are displayed in black, black parts in white.

2.7.14 Synchronized Scrolling

This function makes it possible to automatically or manually synchronize multiple series while scrolling, meaning that all synchronized series will show the nearest possible anatomic position. A typical application of this tool would be to two associated series, one recorded without and one with a contrast agent.

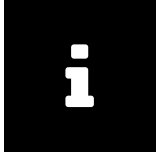
Generally speaking, synchronized scrolling is only possible when the images of the different series contain information on their spatial positioning and their image planes do not exceed a certain angle.

You can activate synchronized scrolling for selected series by clicking on  or by pressing **S** on the desired series. To activate synchronized scrolling for all displayed series containing the necessary information on spatial positioning, click the synchronizing symbol  in the global tool bar.

Depending on whether the series are spatially related and on which image you activate synchronized scrolling, the series are either automatically or manually synchronized:

- Automatic synchronized scrolling is applied to spatially related series. The synchronized series are automatically scrolled to the current image slice position of the active series.
- Automatic synchronized scrolling is also applied to series without a spatial relationship, as long as the function is activated on the very first image in each series, meaning that synchronization between series is based on the anatomically topmost image (starting image). If this does not produce the desired result, you can make manual adjustments (see below).

- Manual synchronized scrolling can be used for series with no spatial relationship, provided that the topmost image is not used as the starting image for scrolling. This allows you to make manual corrections by first selecting the most anatomically similar images across all series you wish to synchronize. By making these similar images your starting images, you can use synchronized scrolling in manual mode.

**Note**

Please check carefully to what extent the slice images share the same anatomical positioning characteristics, as *syngo.share* webview does not provide feedback in this regard. The starting images should possess the same anatomical positioning characteristics in order to avoid errors.

The pan function (see [Section 2.7.5](#)) and all the scaling tools except the magnifying glass (see [Section 2.7.3](#)) are applied to all synchronized series consistently.

You can deactivate synchronized scrolling for all displayed documents by clicking on in the global tool bar.

2.7.15 VOI LUT Presentation

A VOI (Value of Interests) LUT (Lookup Table) is a type of nonlinear windowing. When loading DICOM images with one or more VOI LUTs into the Viewer, the button in the specific toolbar lets you switch between the available VOI LUTs (e.g. High+Contrast) or display the DICOM images without VOI LUTs.

2.7.16 Presets

For grayscale DICOM images, the administrator can configure presets for the optimized windowing of certain body parts (e.g. cranium, lungs). These windowing presets can be selected in the drop-down menu and be applied to the image shown in the Viewer.

2.7.17 Presentation States

If a DICOM study with Presentation States (Grayscale Presentation States) is opened in *syngo.share* webview, the Presentation States are loaded automatically and the last Presentation State created is applied. Use the drop-down menu to select the desired Presentation State or deactivate an active Presentation State.

2.7.18 ECG Tools

For viewing DICOM ECGs, some additional tools are available:

2.7.18.1 Metadata

Hides or shows the metadata block above the ECG.

2.7.18.2 Caliper

Adds visual interval guidelines. Select the start and end point of the interval with the mouse. The selected interval will be automatically repeated across the spectrum.

2.7.18.3 Time Calibration/Paper Feed Speed

Time calibration, i.e. paper feed speed in mm/s, can be chosen in the drop-down menu.

2.7.18.4 Voltage Calibration

Voltage calibration in mm/mV can be chosen in the drop-down menu.

2.7.18.5 Channel Layout

If the active channel group consists of exactly twelve channels, there are several channel layouts available for viewing the ECG.

2.7.18.6 Channel Group

The channel group can be selected in the drop-down menu.

2.7.18.7 Fit in View

Fits the image to the window size.

2.7.19 Whole Slide Images for Digital Pathology

syngo.share webview supports DICOM brightfield microscopy images for Digital Pathology .

The following functions specific to digital pathology are available in the Viewer:

Slide Label

The slide label shows the actual label of the microscope slide, if available.

Zoom Layer

The zoom layer can be selected in the drop-down menu. Alternatively, you can scroll through the zoom layers using the mouse wheel or the zoom tool in the toolbar (see [Section 2.7.3](#)). Zooming is always centered on the current position of the mouse pointer.

Focal Plane

If focal planes are available, they can be selected in the drop-down menu. Alternatively, you can scroll through the focal planes using the mouse wheel while holding Shift.

Thumbnail

If available, the thumbnail shows the slide overview. When the image is displayed in a zoom layer, the thumbnail indicates the location of the zoomed area. You can navigate to any area of the image by clicking on it in the thumbnail.

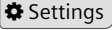
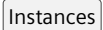
2.7.20 Playback

syngo.share webview offers different types of playback in the Viewer:

Generic video playback

You can play back generic videos. The video toolbar offers basic playback and volume control. You can enable autoplay for videos under  **Settings** > **Viewer**.

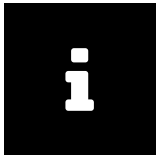
☰ DICOM video playback

DICOM video playback offers the same toolbar as generic video playback. You can enable autoplay for DICOM videos under  **Settings** > **Viewer** by enabling either DICOM modality autoplay or video autoplay. If a DICOM series contains multiple video instances, you can use  in the viewport menu to switch between them.

☰ DICOM series playback

You can also play back DICOM series of certain modalities. The series toolbar offers different playback modes (e.g. looping) and settings for manual browsing (e.g. image step size). You can enable autoplay for DICOM series under  **Settings** > **Viewer**.

- **Playback range:** You can limit playback to a specific range of images by clicking on  to set a beginning and an end point. You can adjust the playback range by dragging the markings to the desired position.
- **Playback rate:** You can adjust the FPS value to slow down or speed up series playback. You can set a standard FPS rate under  **Settings** > **Viewer**. Note that in cases where a DICOM multiframe instance already contains a recommended FPS value, this FPS value will be preferred.
- **Multiframe playback:** You can switch between the individual instances of a multiframe series by clicking on  and  in the toolbar or by selecting the instance via  in the viewport menu.

Note

You may need to grant browser-specific permissions to use autoplay in your web browser of choice. In addition, note that playback speed may depend on the available hardware as well as the size and number of loaded documents.

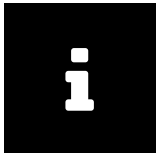
2.8 Inbox

The inbox contains all documents that have been shared with you, such as doctor's letters, findings or medical images.

Click on a share to display the shared documents as well as the (optionally) included message. For requesting documents, see [Section 2.8.2](#).

If a share notification contains more than one document, you can sort them by clicking on the available columns. Select a document to receive a preview of the DICOM series and images or generic files contained therein.

By double-clicking on a shared document, you can open it directly in the Viewer for viewing and editing purposes. You can also open select elements by double-clicking on an entry in the preview area.

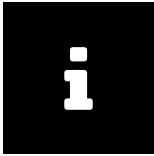
Note

Items in the inbox are actually links to items in the archive which have been shared with the recipient. This means that removing a share from the inbox does not remove the document from the archive.

2.8.1 Actions

☰ Refresh

Your inbox will periodically refresh in order to display the newest shares if so configured by your system administrator. Use the button  to manually update your inbox.

Note

If your system administrator has not set an automatic refresh, you have to refresh your inbox manually in order to see any new shares.

☰ Filter

Filters the contents of your inbox when you enter a term in the filter field. Doing so searches for the term in every share and shows all matches. Entries which do not fulfill the search criteria are not shown.

☰ Sort

See [Section 2.5.2](#).

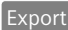
☰ Mark as read/unread

Changes the message status to “read” or “unread”.

☰ Remove share

Removes the selected share from the inbox.

☰ Download

Quick export of a selection compressed as a ZIP file. This function is the equivalent of the  function for the original, non-encrypted data, see [Section 2.6.5.3](#).

☰ Export

Exports a selection from the archive using various options, see [Section 2.6.5.2](#).

2.8.2 Requesting Documents

The sharing of documents to authorized users, e.g. established doctors and other medical institutions/ departments, only occurs with the consent of the patient. Findings/documents are sent to the transferring doctor, thus replacing physical postal delivery.

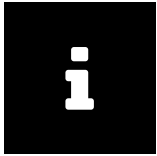
The procedure for requesting findings/ documents is determined by the respective health care institution, although it is usually done by phone.

The drop-down menu next to your user name provides further information regarding the findings request (if applicable).

2.8.3 Availability Period

The hospital operator determines the availability period of shares. The time limit is displayed in the header of the individual share (**Valid until**). For shares that are available indefinitely this field is empty. The shared documents are available in your inbox until the end of the set period.


After the end of the availability period, the share is removed from your inbox. If you were not able to view the shared documents in time, you can request them again.

**Note**

A system administrator can also revoke a document share, e.g. when a faulty/incorrect document is removed from the archive. In this case, the document will no longer appear in your inbox.

2.9 Archiving Documents

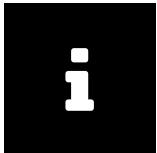syngo.share webview can archive documents within a patient, entity or visit context. This function can be found in the **Archive** tab.

**Note**

Archiving documents is only supported for *syngo.share* core and is not available for external archives.

Use the quick search field to search for the visit ID and/ or patient ID (depending on the settings). To start a more specific search, click on **Choose**. In the new window, determine whether the search should refer to a patient or entity context. Note that you cannot conduct a search for visit ID in this window. In the same window, you can also create new patients or entities by clicking on **Create**.

Alternatively, you can trigger archiving by right-clicking on the patient data in the patient-grouped results list and selecting **Archive** from the context menu. This automatically applies the correct patient context. For entities, open the context menu by right-clicking on the respective document.

You can reset the patient/entity selection by clicking on **Change context**.

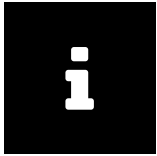
The entry of document metadata can be done in the main window by using the available fields. You can enter the following metadata:

- Producer (required field) to which the documents will be imported
- Accession Number
- Body Part (DICOM element `BodyPartExamined`)
- Description
- Study Date (DICOM element `StudyDate`, `StudyTime`); is only applied to DICOM images when these DICOM elements have not yet been set
- Keywords

You can add documents to the archiving area via drag-and-drop or by clicking on **Add**. In the case of DICOM documents, the patient-related DICOM elements will be compared with the data provided by the current patient context. If these do not match, the system provides the opportunity to overwrite the affected DICOM elements accordingly. If no patient context was set beforehand, the patient data contained in the DICOM elements will be applied automatically.

Click on **Archive** to start the archiving process. You can check its status in the **Job Queue** tab (see [Section 2.10](#)).


It is possible to apply keywords during the archive process. For a description of keyword tagging, see [Section 2.6.3](#).

**Note**

To ensure correct video playback, only import videos with a constant bitrate. Videos with a variable bitrate may not be played back properly.

To keep loading times as short as possible, import videos to an Organizational Unit for which an archiving rule without data compression has been defined.

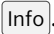2.10 Job Queue

In the **Job Queue** tab, you can view the status of potentially time-intensive jobs such as the sending or restoration of documents. If an error occurs, the tab as well as the affected job are marked with a badge. You can view error details by clicking on . The tab also shows if there are currently jobs running in the background.

The following actions are available in the **Job Queue**:

Cancel

You can cancel certain jobs.

Retry

You can retry canceled or failed jobs. This option is available for all jobs except the upload of documents. Failed uploads have to be reinitiated manually.

The job status list is maintained between sessions and across multiple tabs. So-called “hanging” jobs, i.e. jobs which do not log any progress for more than one day, are automatically assigned the status “Failed”. After one additional day, they are automatically cleared from the queue together with any other failed or canceled jobs.

To manually clear the queue, click on the corresponding button.

2.11 Keyboard Shortcuts syngo.share webview

General Viewer Commands

J	Show/hide Patient Jacket
Q	Quit Viewer (return to search)
R	Reset Viewer
Esc	General: reset selected tool
Esc	Drawing annotations: undo most recent annotation

Viewing Tools

M	Magnifier
P	Pan
S	(De-)Activate synchronized scrolling in active viewport
Z	Zoom
Shift + Mousewheel	Whole Slide Images: Switch between focal planes
Spacebar	Start/pause video/series playback

Viewer Layouts

1	Apply 1x1 layout
2	Apply 1x2 layout
3	Apply 2x1 layout
4	Apply 1x3 layout
5	Apply 2x2 layout
6	Apply 3x3 layout

Annotation Tools

A	Angle
C	Ellipse
D	Density
F or W	Level/Width, Brightness/Contrast
L	Length/Distance
Del	Delete selected annotation

Patient/Entity Search

Del	Delete a selected document/document parts from the archive
-----	--

3 Emergency Access

A note on the login screen will inform you when the emergency access mode is activated.

You can log in with your user name and default password (if you have an LDAP-administered account) or with a specific emergency access account which has been established by your institution. For more information, click on the link on the login screen or contact your system administrator.

After entering your login information, simply click on or press as usual.

In emergency access mode, the search screen as well as the results list are pre-configured, though you can still adjust them according to your needs. Usage does not differ from the standard mode (see [Section 2](#)). However, some actions and/or functions may be deactivated.

4 Mobile Devices

syngo.share webview supports and is optimized for use on the following mobile devices:

- iPad
- iPad Air
- iPad Pro

It is required to enable the mobile view of the browser and recommended to use touch input. It is further recommended to launch *syngo.share* webview via home screen shortcut (“website icon”). This will cause the browser to open in full-screen mode which offers the best user experience of *syngo.share* webview. (How to set a shortcut on the home screen is described in the online iPad user guide.)

Although the mobile version of *syngo.share* webview is largely identical to the desktop version, there are a few notable differences:

- Download, print and export are not available on mobile devices.
- CSV export of search results is not available on mobile devices.
- Opening a selected document in a new tab from the properties window is not available on mobile devices.
- Autoplay of videos is not available on mobile devices.
- The full-screen button is not available on mobile devices. Please use the home screen shortcut described above to open the browser in full-screen mode.
- The volume control for videos is not available on mobile devices. Please use the volume buttons of the mobile device.
- Certain functionalities are altered slightly in order to accommodate the screen size and input methods of mobile devices. These differences are explained in the following.

4.1 Functionality

4.1.1 Search Results List

☰ Displaying document details

The series/preview area of the search results list is initially hidden. Select a document from the results list to view its details.

☰ Selecting multiple entries

In order to select multiple entries in the search results list or the series/preview area, tick the check boxes next to the desired entries. In the series/preview area, you first have to switch to list view by tapping .

☰ Using the context menu

To access the context menu, tap to select an entry, then tap on . In the patient-grouped results list, long-press on the patient header to open the context menu (tapping on the header displays the list of associated documents instead).

☰ Filtering search results

You cannot filter the search results list further. To receive fewer results, please refine your search query.

4.1.2 Viewer

☰ Moving files via drag and drop

To drag and drop series/files from the **Document Preview** or **Patient Jacket** area, long-press on an item and move it to the desired location.

☰ Moving viewports

To move a viewport, long-press on its specific toolbar and move it to the desired location.

☰ Deleting annotations

To delete a selected annotation, tap on .

4.2 Touch Gestures

The viewer supports several touch gestures for easier viewing:

☰ Two-finger swipe

Scroll by swiping two fingers up or down.

☰ Pinch

Zoom in or out by pinching/unpinching your fingers.

☰ Three-finger swipe

Pan an image by swiping three fingers in the desired direction.

In addition, the following gestures are available specifically for whole slide images :

☰ Two-finger swipe

Change the focal plane by swiping two fingers up or down.

☰ Pinch

Change the zoom layer by pinching/unpinching your fingers.

☰ Three-finger swipe

Pan an image by swiping three fingers in the desired direction.

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ITH icoserve technology for healthcare GmbH
Siemens Healthcare GmbH