Release VA32C_BF04 / 2024-07-09 / Revision 9813

syngo.share import

Online Help

Language Directory

Deutsch	3
English	50
Français	94

Release VA32C_BF04 / 2024-07-09 / Revision 9813

syngo.share import

Online-Hilfe

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syngo.share import

Online Help

The instructions for use are available in the information about the product in the user menu.

Contents

1	Introduction	55
1.1	Intended Use	55
1.2	Technical Terms and Notation	55
1.2.1	Generic Document	55
1.2.2	DICOM Document	55
1.2.3	DICOM Information	55
1.2.4	EXIF Information	56
1.2.5	Modality	56
1.2.6	Organizational Unit	56
1.2.7	Entity	56
1.3	Safety Notices	57
	Data Import	57
	Medical Education	57
	Accuracy of Calculations	57
	Units	57
	Access Restriction	57
	Effects of External Programs	57
2	Settings	59
2.1	General	59
2.2	Specific	60
2.2.1	Data Sources	60
2.2.2	Context	60
2.2.3	Context Set by URL or Worklist	60
2.2.4	Special	60
2.2.5	Patient Search	60
2.2.6	File Modifications	61
2.2.7	Barcode	62
2.2.8	Tooth Status	62
2.2.9	Filesystem	63
2.2.10	Video	63
2.2.11	Still Image Capture	63
2.2.12	Quick Import	63
2.2.13	Zero-Click Import	63
2.2.14	Windowing Default Settings	64
2.2.15	Viewer	64
2.3	Communication	64
2.3.1	<i>syngo.share</i> core	64
2.3.2	DICOM	64
2.3.3	HL7	64
2.3.4	Worklist	64
2.3.5	Order List	64
2.3.6	IRWF-Specific Options	65
2.4	Video Postprocessing	65
2.4.1	External Video Postprocessing	65
2.4.2	External Video Transcoding	65
2.5	DICOM Import Profiles	65
2.5.1	Default and Additional Rules	66

2.5.1.1	Default Rules	66
2.5.1.2	Additional Rules	66
2.5.2	Creating a DICOM Import Profile	67
2.5.3	Automatically Applying a DICOM Import Profile	67
2.5.4	Data Minimization	67
2.6	Generic Import Templates	68
2.7	Personal Settings Profile	68
3	Usage Instructions	69
3.1	Getting Started	69
3.1.1	User Interface	69
3.1.1.1	Main Menu	69
3.1.1.2	Main Area	70
3.1.1.3	Document List	70
3.1.1.4	Archiving Area	70
3.1.2	Login and Logout	70
3.1.3	Changing the Password	71
3.1.4	Forced Updates	71
3.2	Setting a Context	71
3.2.1	Searching for an Existing Context	72
3.2.1.1	Quick Search	72
3.2.1.2	Context Selection	72
3.2.1.3	Automatic Context Transfer from the HIS	73
3.2.2	Setting a Random Patient Context	73
3.2.3	Creating a New Context	73
3.2.4	Changing and Editing the Context	73
3.3	Verifying a Patient Context	74
3.4	Adding Documents	74
3.4.1	File System	75
3.4.1.1	File Browser	75
3.4.1.2	Dragging and Dropping Files from Windows Explorer	76
3.4.2	Video	76
3.4.3	Scan	77
3.4.4	Camera	77
3.4.5	Tooth Status	77
3.4.6	Archive	78
3.4.6.1	Types of Search Fields	78
3.4.6.2	Saving a Search Query	81
3.4.6.3	Results List	81
3.4.6.4	Series Area	82
3.4.7	DICOM Q/R	83
3.5	Preparing Documents for Archiving	83
3.5.1	Changing the Order of Documents	84
3.5.2	Editing Documents from the Archive	84
3.5.3	Editing Document Descriptions	85
3.5.4	Setting the Date and Time of an Examination	85
3.5.5	Tagging Documents with Keywords	85
3.5.6	Creating New UIDs	86
3.5.7	DICOM-Wrapping Generic Documents	86
3.5.8	Setting the Generic File Type	87
3.5.9	Applying a Generic Import Template	87
3.5.10	Applying a DICOM Import Profile	87

3.6	Viewing and Editing Documents	87
3.6.1	Viewing Documents	88
3.6.2	Editing Documents	89
3.6.2.1	Annotations	89
3.6.2.2	Rotating and Flipping Images	89
3.6.2.3	Anonymizing Image Pixel Data	89
3.6.2.4	Video Editing	90
3.6.2.5	Splitting DICOM Series and DICOM-Wrapped Documents	90
3.7	Archiving Documents	91
3.8	Tracking the Archiving Progress	91
3.9	Keyboard Shortcuts in <i>syngo.share</i> import	93

1 Introduction

1.1 Intended Use

syngo.share import is software for the electronic conversion and transfer of data and provides the ability to import clinical data from various sources (such as from local filesystems, DICOMDIR media, digital video cameras, digital photo cameras, documents, and DICOM scanners) into connected archives (such as Vendor Neutral Archives, Picture Archiving and Communication Systems, DICOM archives). This product is not intended for diagnosis, prevention, monitoring, prediction, prognosis, treatment, cure or alleviation of diseases or other medical conditions.

1.2 Technical Terms and Notation

1.2.1 Generic Document

This term refers to any non-DICOM file that can be viewed in or archived by *syngo.share*. Non-DICOM files include images produced by digital cameras or imaging applications (e.g. JPEG, MPM, TIFF) as well as various text, video, and audio file formats (e.g. PDF, DOC, MPEG4, MP3) produced by various external applications (e.g. Microsoft Office, Adobe Acrobat).

Generic files are wrapped in generic containers, analogous to DICOM studies.

1.2.2 DICOM Document

A DICOM document consists of at least one DICOM study, series and instance.

DICOM documents are produced by medical imaging devices compliant with the DICOM standard. They can be viewed in or archived by *syngo.share*.

- **Study**

A DICOM study contains all the data of a medical examination during which image data is acquired. This includes the actual image data (e.g. X-ray, CT, ultrasound) as well as the patient data and descriptions relevant to the examination (such as the examination date or type). A DICOM study contains 1-n DICOM series.

- **Series**

A DICOM series usually equates to a specific type of data acquired/produced by a particular device or to the position of the patient during acquisition. A series contains 1-n DICOM instances.

- **Instance**

A DICOM object instance (or simply DICOM instance) is the actual data captured by the device, such as an image or waveform object.

1.2.3 DICOM Information

DICOM (Digital Imaging and Communications in Medicine) is a standardized file format used in medical imaging.

In addition to object data (typically image data), the DICOM format contains additional information such as patient name, admission date, device parameters, and physician's name. The DICOM format also contains the information necessary for data exchange between DICOM-capable devices.

1.2.4 EXIF Information

EXIF (Exchangeable Image File Format) is a standardized data format generated by, e.g., digital cameras when taking a picture.

In addition to the image data commonly contained in JPEG or TIFF format, the EXIF format contains additional information describing the image in more detail, such as date and time information or the type of device used for image capture.

1.2.5 Modality

An imaging modality is a device for acquiring medical images such as X-rays or CTs. In *syngo.share* import, modalities are identified by abbreviations such as CR for Computed Radiography (e.g. when searching).

1.2.6 Organizational Unit

An Organizational Unit is an association of arbitrary actors in a medical organization, e.g. a department, medical devices, or groups of people. Organizational units can be customized and expanded with more constituents as needed, and these constituents can be uniformly managed.

All data to be archived is allocated to exactly one organizational unit.

1.2.7 Entity

In *syngo.share*, entities are an alternative way to identify documents that do not belong to a patient or visit. This is useful e.g. for archiving documents from clinical studies and trials.

1.3 Safety Notices

Data Import

syngo.share import is only intended for the importing of data from various sources.

Medical Education

The use of *syngo.share* import should be granted to qualified staff members only.

Accuracy of Calculations

Images may contain calibrations. Please note that the accuracy of these calibrations correlate to the precision of the program or modality with which they were created. Calibrations must be checked according to the manufacturer's instructions. Calculations, calibrations and results derived therefrom must be checked for plausibility by clinical professionals. Inaccuracies may arise during image capture by clinical professionals or due to the image capturing technology of a given modality. For instance, X-ray images may contain inaccuracies due to the fact that the x-rayed body part is projected onto a two-dimensional imaging surface. Inaccuracies during visualization may be due to hardware settings.

This product rounds calculations as follows: Length and area: if pixel spacing $\geq 0.1\text{mm}$: 1 decimal place, if pixel spacing $< 0.1\text{mm}$: 2 decimal places; density (HU): no decimal place; density average (HU): 1 decimal place; density area (HU): no decimal place. This product uses an interpolation algorithm for image display. This product may not be able to calculate the area of polygons whose edges overlap; this will be indicated by the value "n/a".

Units

The application *syngo.share* import conforms to Directive 80/181/EEC, which contains specifications related to units of calculation.

- Calibrated lengths are calculated in millimeters (mm). Uncalibrated lengths are calculated in pixels (pixel).
- Calibrated surface areas are calculated in square millimeters (mm^2). Uncalibrated surface areas are calculated in square pixels (pixel^2).
- Plane angles are calculated in degrees ($^\circ$).
- Densities are calculated using the industry-standard Hounsfield unit (HU), which is not regulated by Directive 80/181/EEC.

Access Restriction

Be sure to log out of *syngo.share* import when finished working in order to avoid unauthorized access to the system.

Effects of External Programs

The software manufacturer does not guarantee that third-party applications will not negatively affect the system. The standards of any external programs accessed by *syngo.share* import are set by their respective manufacturers.

2 Settings

Note

The settings of *syngo.share* import are largely self-explanatory. The following sections cover complex settings and options with special features.

The *syngo.share* import installation comes with a set of default settings. To access the settings, click on  in the main menu.

You can do the following:

- Browse settings by category.
- Search for particular settings. To reset the search, click on **x** in the search field.
- Check any changes before saving them. Categories which contain changed settings are marked with *. Changes are only implemented when you confirm them with **Apply**. Note that some changes are applied immediately while others require you to restart the application. Settings that require a restart are clearly marked.
- Postpone restarting the application to implement changes. In this case, you will continue working with the previous settings until you restart the program.

Changes to the settings are saved on the server and loaded from there when you log in to *syngo.share* import. If the server cannot be reached, the last locally saved configuration is used. In case you make changes while the server cannot be reached, they may be overwritten when you next start *syngo.share* import. This occurs when the server is reached during start-up and the last known configuration is loaded from there.

2.1 General

This category contains settings that determine the overall look and behavior of *syngo.share* import. You can also determine which areas and functions are available in the main menu.

- **Language**
Selecting the entry **System language** applies the language settings of the Windows operating system.
- **Minimize to taskbar**
This option puts *syngo.share* import in sleep mode upon closing the program, which shortens its startup time. The application icon remains visible in the Windows taskbar.
- **Automatic login**
The Windows single sign-on function logs the user in to *syngo.share* import with their Windows login credentials. If this option has not been enabled by the system administrator, you can still log in manually with your *syngo.share* import credentials. It is also possible to use Kerberos for automatic login.
- **Close automatically when finished**
syngo.share import will immediately close after archiving is complete. To continue working with *syngo.share* import and view detailed information on the archiving process, deactivate this option.

2.2 Specific

This category contains settings for context and data acquisition. You can show/hide various parts of the UI, determine the behavior of features like the quick search, enable file modifications, and configure the use of external devices (e.g. a barcode scanner).

- Data Sources
- Context
- Context Set by URL or Worklist
- Special
- Patient Search
- File Modifications
- Barcode
- Tooth Status
- Filesystem
- Video
- Still Image Capture
- Quick Import
- Zero-Click Import
- Windowing Default Settings
- Viewer

2.2.1 Data Sources

In this section, you can toggle which means of data acquisition are available in the **Archiving** tab.

2.2.2 Context

In this section, you can determine which methods for setting a context are available in the **Context Selection** tab.

2.2.3 Context Set by URL or Worklist

In this section, you can determine which context data can be edited if the context has been set automatically by URL parameters or the worklist.

2.2.4 Special

- **Combine study description and body part examined**
Adds contents of the DICOM tag **BodyPartExamined** to the DICOM tag **StudyDescription**.
- **Creation date threshold for generic containers**
Determines how far a user may backdate an examination without triggering a warning message.

2.2.5 Patient Search

In this section, you can determine the behavior of the patient and quick search.

- **Prefix in quick search**
You can limit the quick search to a particular set of patient/visit IDs by setting a default prefix.
- **HL7-managed patients and visits**
You can limit the search results to HL7-managed patients and visits to ensure consistent data quality.

2.2.6 File Modifications

In this section, you can adjust file conversion settings.

- **DICOM-wrap**

DICOM-wrapping converts generic documents into DICOM upon archiving, thus providing the extensive metadata of the DICOM format to generic documents. Note that enabling this option will cause certain actions specific to generic documents to be unavailable. The DICOM-wrapping of videos is a special case (see Section 2.4).

- **Force creation of new UIDs**

Creates new UIDs for DICOM documents upon archiving.

- **Allow mirroring and rotating**

Enables the mirroring and rotating function for generic images in the Viewer. Note that enabling this option may lead to the accidental archiving of images in a reflected state.

- **Alternative specific character set**

Allows you to automatically correct character errors in DICOM documents by defining one or more alternative character sets here. You have to specify the character sets in DICOM notation, e.g. ISO_IR 100 for the ISO-8859-1 (Latin-1) character set. For more information on specific character sets, please consult the DICOM standard.

- **Enable universal corrector**

Enables a correction mechanism for import-relevant DICOM tags in order to ensure they conform to the requirements of the DICOM standard and of *syngo.share* core.

These corrections are recorded in the document log by stating the previous and corrected value. The **Job Queue** in *syngo.share* import provides information on the corrections carried out upon archiving.

The following corrections are possible:

- **Invalid Specific Character Set; DICOM tag (0008,0005)**

Obvious errors are corrected, e.g. ISO-IR-100 will be corrected to ISO_IR 100. If a correction is not possible, the default character set US-ASCII will be used.

- **Wrong or unacceptable date and time formats**

Obvious errors are corrected. If this is not possible, nullable entries will be set to NULL, all others will be set to the date 19000101.

- **Multiple values in a single-value tag**

Multi-value separators are replaced by a space in order to create a single-value metadata entry.

- **Empty or unknown modality; DICOM tag (0008,0060)**

A correction of upper- and lower-case spelling is attempted. If this is not successful, an empty or unknown modality tag will be set to modality OT.

- **Unknown patient sex; DICOM tag (0010,0040)**

A correction of upper- and lower-case spelling is attempted. HL7 sex information will be transformed into DICOM sex information. If no correction is possible, patient sex information will be deleted.

- **Too many name components or groups**

The last elements of the name components or groups will be deleted.

- **Overlong string values**

The values will be shortened to the maximum length permitted by the database or the DICOM standard.

- **Unsupported characters**

Characters that are not supported by the character set will be eliminated.

- **Unknown Assigning Authority**

If the provided Assigning Authority is unknown, the default Assigning Authority for the respective authority type (e.g. patient ID) will be set. This function is only available if *syngo.share* core is used as the archive. For third-party archives, the authority information can be removed instead (see Section • **Remove Authority Information**).

- **Remove Authority Information**

Automatically removes the Assigning Authority information from DICOM data upon archiving. This allows the import of DICOM data despite incorrect or ambiguous Assigning Authority information. For imports to *syngo.share* core, the default Assigning Authority is applied. For all other archives, the information is removed. This affects the following DICOM tags:

- Issuer of Patient ID; DICOM tag (0010,0021)
- Issuer of Patient ID Qualifiers Sequence; DICOM tag (0010,0024)
- Issuer of Admission ID Sequence; DICOM tag (0038,0014)
- Issuer of Accession Number Sequence; DICOM tag (0008,0051)
- Order Placer Identifier Sequence; DICOM tag (0040,0026)
- Order Filler Identifier Sequence; DICOM tag (0040,0027)

2.2.7 Barcode

In this section, you can configure the use of a barcode scanner for setting a patient context:

- **Start/stop key sequence**

These fields are used to specify key commands for which your barcode scanner has been programmed as well. The commands activate and deactivate the quick search input field, respectively. When scanning a barcode, the barcode scanner automatically sends the start sequence, which allows the barcode to be transferred directly into the quick search input field and trigger the search. If the activation character combines multiple keys, use + as a separator, e.g. **CTRL + SHIFT + F3**.

- **Pattern**

This field specifies how many parts of a barcode have to be read correctly in order to trigger a search.

- **Ignore check digit**

This defines whether the last number (check digit) of the barcode should be ignored. Even when this option is activated, the pattern has to match the barcode including the check digit. This option only deletes the check digit after the barcode was recognized correctly.

- **Ignore padding**

This option defines whether the leading zeros of a barcode should be ignored. Even when this option is activated, the pattern has to match the barcode including the leading zeros. This option only deletes the leading zeros after the barcode was recognized correctly.

2.2.8 Tooth Status

In this section, you can configure the **Archiving > Tooth Status** tab, such as the display size of dental X-rays (the default is 2500x1280 pixels) or the default viewing/editing tool.

2.2.9 Filesystem

In this section, you can configure the **Archiving > Filesystem** tab. You can e.g. hide drives to make their files unavailable for archiving or determine whether data from particular drives should be cached in a temporary directory. Caching may speed up the archiving process. Note that the temporary directory will be cleared automatically after successful archiving or, in case of an error, at the next launch of *syngo.share* import.

2.2.10 Video

In this section, you can configure the **Archiving > Video** tab. You can set up video/image recording devices, determine the maximum permissible length for video recordings to save space or adjust the image quality for lossy compression formats.

To ensure correct video playback in *syngo.share* import, only import videos with a constant bitrate. Videos with a variable bitrate may not be displayed properly.

- **Individual recording device settings**

Use **Edit** to customize the various settings for each recording device, such as the resolution for video recordings and still image capture or the recording bitrate. Moreover, you can (de)activate video preview and audio recording. To record audio, select your preferred source device.

- **Offset correction for audio tracks**

Some video recording devices may introduce an unwanted offset between the video and audio tracks during streaming. To counteract this, *syngo.share* import offers a delay feature for correcting an offset introduced by a given device. You can enter a delay period (in ms) on the **Edit** screen. You may have to test multiple values in order to find the appropriate value for the given device.

Note

Be sure to check the delay setting whenever changes have been made to the device (hardware, drivers, updates, etc.) and make adjustments accordingly. Changes and adjustments should only be made by qualified personnel.

2.2.11 Still Image Capture

In this section, you can configure a footswitch and an acoustic confirmation signal for the capture of still images.

2.2.12 Quick Import

In this section, you can enable the quick import for the easy archiving of DICOMDIRs from **Archiving > Filesystem > DICOM Overview** (see Section 3.4.1.1).

2.2.13 Zero-Click Import

In this section, you can enable the immediate, automatic archiving of DICOMDIRs from CDs or USB flash drives.

- **Destination Type**

The destination for zero-click-imported files can be a DICOM C-STORE target, an AET or an Organizational Unit. The DICOM C-STORE targets are predefined and can be selected, AETs and Organizational Units have to be entered manually.

2.2.14 Windowing Default Settings

In this section, you can create presets for automatic windowing. Windowing values are saved when files are archived. It is recommended that you test various presets and save only those that yield satisfactory results.

- **Lower threshold**
Lower limit of the brightness area (Level/width control).
- **Upper threshold**
Upper limit of the brightness area.

2.2.15 Viewer

You can choose the default tool for the editing mode in the viewer. If you make no changes, the selection remains set to the mouse pointer.

2.3 Communication

This category contains settings for the communication with *syngo.share* core, third-party archives and other systems. Settings in this section should only be made by a system administrator.

2.3.1 *syngo.share* core

If *syngo.share* import is implemented with *syngo.share* core, set the connection parameters here.

2.3.2 DICOM

This section contains the settings for communication with one or more third-party archive. Depending on the use case, the corresponding options and services are configured here, e.g. C-STORE for archiving.

2.3.3 HL7

This section contains the settings for HL7 peers and for the verification of patient data via the patient index (PIX) and the Patient Demographics Query (PDQ). For PDQ, you can determine the fields that should be queried.

2.3.4 Worklist

This section contains settings for the DICOM worklist. Choose the query period and the update interval for the worklist. The set query period determines which documents will be displayed in the worklist. The standard setting is 24 hours.

2.3.5 Order List

This section contains the settings for the order list. You can set the query period, which determines which entries are shown in the order list. Furthermore, you can set the update interval for the list and designate an Alternate Scheduled Station AET.

2.3.6 IRWF-Specific Options

Here you can adjust settings related to the Import Reconciliation Workflow (IRWF) profile. This IHE profile manages the import of image data from external CDs, etc., and attempts to match their identifiers to the values stored in the system.

2.4 Video Postprocessing

This category contains settings for editing video files with external applications.

2.4.1 External Video Postprocessing

In this section, you can configure an external application (FFmpeg) to make changes to video files. This allows *syngo.share* import to postprocess even video formats that it cannot normally postprocess. Videos can be converted to another format or trimmed before importing them into the archive.

Warning

It is not possible to guarantee data integrity of transferred data between external applications (FFmpeg) and *syngo.share*.

2.4.2 External Video Transcoding

Set the target codec, bitrate and maximum resolution for transcoding here.

When videos are set to be DICOM-wrapped (see Section • **DICOM-wrap**), special conditions apply because the DICOM standard only supports three video codecs. Accordingly, *syngo.share* import will provide alerts in certain cases when videos are to be archived.

- If the generic video file is of a format that can be embedded directly into a DICOM object as MPEG2, H.264 or H.265 and its resolution conforms to the DICOM standard, it will be DICOM-wrapped and archived without delay.
- If the generic video file can be embedded directly but its resolution does not conform to the DICOM standard, you will receive a notification and the file will be marked with a yellow warning triangle. Still, the file can be archived as usual.
- If the video file is of a format which cannot be embedded directly into a DICOM object, you have to specify an external application (FFmpeg) for the job. In this case, you will receive a notification that the file is being transcoded. However, if you have not yet specified an external application, the file is rejected from archiving and marked with a red X.

2.5 DICOM Import Profiles

In this category, you can define DICOM import profiles, i.e. bundles of rules for automatically editing DICOM data elements during archiving. Import profiles allow you to do the following:

- Replace existing DICOM tag values with specific values
- Replace existing DICOM tag values with random values
- Remove existing DICOM tags or tag values
- Add missing DICOM tags or tag values

2.5.1 Default and Additional Rules

A DICOM import profile consists of a set of default rules as well as an arbitrary number of customizable additional rules.

- [Default Rules](#)
- [Additional Rules](#)

2.5.1.1 Default Rules

With default rules, you can define how private and personal DICOM data elements are handled. Private data elements allow for the specification of information that is not covered by the standard DICOM elements. Personal DICOM data elements refer to patient data contained in the DICOM tags (0010,XXXX). The following options are available:

- **Keep:** All DICOM data elements of this group keep their original values.
- **Remove:** Almost all DICOM data elements of this group are removed. Only the tags [Patient's Name](#), [Patient ID](#), [Patient's Birth Date](#), and [Patient's Sex](#) are kept since they are required for a valid DICOM document.
- **Remove partially:** This option is only available for personal DICOM data elements (i.e. 0010 DICOM tags). A subset of DICOM data elements which might allow reidentification of the patient is removed, while some DICOM data elements that can be of medical relevance (e.g. [Patient's Weight](#), [Pregnancy Status](#)) are kept. Moreover, the tags [Patient's Name](#), [Patient ID](#), [Patient's Birth Date](#), and [Patient's Sex](#) are kept since they are required for a valid DICOM document.

2.5.1.2 Additional Rules

With additional rules, you can define the handling of any DICOM data elements of your choice. For DICOM sequences, i.e. nested DICOM data sets, you can precisely specify which sequence item and which tag contained in which sequence item you want the rule to affect (see [Section 2.5.2](#)).

This way, you can correct information, provide additional or missing information, or ensure that certain kinds of identifying information are replaced with random values for data minimization purposes (see [Section 2.5.4](#)).

The available options depend on the selected DICOM data element:

- **Set value:** Replaces the existing tag value with the value you specify for this rule.
- **Set user value:** Replaces the existing tag value with a freetext value. You have to enter the value manually during the archiving process.
- **Set user value from list:** Replaces the existing tag value with a value from a predefined list. You have to select the value during the archiving process.
- **Set user value from catalog:** Replaces the existing tag value with a keyword from the keyword catalog you specify. You have to select the keyword during the archiving process.
- **Set random value:** Replaces the existing tag value with a random value.
- **Set empty value:** Removes the existing tag value.
- **Remove tag:** Removes the entire DICOM tag.

Note

Be careful not to create rules which remove required tags or tag values or which set invalid values. Failure to do so will result in DICOM documents that do not conform to the DICOM standard or are unusable. Please consult the DICOM standard before creating or editing rules.

2.5.2 Creating a DICOM Import Profile

You can create new DICOM import profiles or copy and edit existing ones. To set up a profile, proceed as follows:

1. Click on **Add** or **Copy** and provide a unique name for the profile.
2. Select your default rules for private tags and personal data.
3. Optionally, you can add an arbitrary number of additional rules:
 - (a) Select the desired DICOM element from the list.
 - (b) Confirm your element selection with ✓. The element is then added to the list of additional rules. You can remove entries in the list by clicking on x.
 - (c) Select an action for the rule. Depending on the action, a new input field may open where you have to specify a value, select a keyword catalog or provide a list of items for selection.
4. Any changes you make to the import profile are saved automatically.

Creating Rules for DICOM Sequences

For DICOM sequences (i.e. nested data sets), you can specify which level and part of the sequence you want the rule to affect. Click on ✓ to specify a lower level in the nesting hierarchy.

- To affect the entire sequence tag, simply confirm your selection with ✓.
- To affect all sequence items, click on ≡ and select **All available items**. Then click on ≡ again to select the desired tag you want to edit in all these items.
- To affect a specific sequence item, click on ≡ and select its position number. Then click on ≡ again to select the desired tag you want to edit.

Sequences can contain further sequences. If a tag contained in a sequence item is itself a sequence, you can continue to select even lower levels of the nesting hierarchy to create an extremely specific rule.

2.5.3 Automatically Applying a DICOM Import Profile

You can set a particular DICOM import profile to be applied automatically during regular archiving or during DICOMDIR quick import. To do so, select the desired profile or profiles for **Enforce use of import profile** and **Use import profile for quick import or zero-click import**.

2.5.4 Data Minimization

You can use DICOM import profiles to minimize identifying information about patients, e.g. for clinical research purposes. To set up a profile for data minimization, proceed as follows:

1. Set a default rule to remove or partially remove personal data (see Section 2.5.1.1).
2. You may also wish to set a default rule to remove private tags since they may contain identifying information.

3. Set additional rules to apply random values to the following DICOM tags (see Section 2.5.1.2):

- (a) Patient's Name (0010,0010)
- (b) Patient ID (0010,0020)
- (c) Patient's Birth Date (0010,0030)
- (d) Patient's Sex (0010,0040)

Note

Be aware that DICOM import profile for data minimization cannot affect identifying information which may be contained in the images themselves, such as burnt-in text.

2.6 Generic Import Templates

This category contains options for setting up import templates for the purpose of cross-enterprise document sharing (XDS). These templates can be used to provide the necessary XDS metadata for generic documents upon archiving.

You can set up multiple templates for various document types as well as enforce the use of a particular template. You can select these templates prior to archiving (see Section 3.5.9).

To use generic import templates, you have to disable DICOM-wrapping (see Section • **DICOM-wrap**).

Note

Note that XDS requires an eHealth Solutions Adaptor Set. Your system administrator has to configure it in *syngo.share* Configuration.

2.7 Personal Settings Profile

You can save and load your *syngo.share* import settings as a personal user profile. This is useful when you share a workstation with several other users or want to use *syngo.share* import on multiple workstations.

To create a new user profile, click on **Save Profile**. Specify a file name and target folder before clicking **Save**. Your profile will be saved with the .icf (icoserve Config File) extension.

If you would like to access your profile from another workstation, save it profile on an external media device or network drive.

To apply a personalized user profile, click on **Load Profile**. Select the directory and file of the desired profile and click **Open**. *syngo.share* import adopts all the settings contained in the profile.

3 Usage Instructions

Note

These usage instructions provide a comprehensive description of the functions of *syngo.share* import. Depending on the settings of your organization and your user permissions, some of these functions may not be available to you. If you have any questions, please contact your system administrator.

3.1 Getting Started

3.1.1 User Interface

The *syngo.share* import user interface is divided into the following sections:

- Main Menu
- Main Area
- Document List
- Archiving Area

3.1.1.1 Main Menu

The main menu contains tabs for the various functions of *syngo.share* import:

-  **Home**
Login area, contains release and application information.
-  **Context Selection**
Search area for acquiring the context.
-  **Archiving**
Main working area for acquiring data.
-  **Quick Search**
Quick search for a patient or visit ID. Once you set the context, the quick search field is replaced by the **Context** tab.
-  **Context**
The context tab only appears when you have set a context. It displays basic information about the selected patient or entity. Clicking on the patient context tab takes you to the viewing and editing area.
-  **Online Status**
Shows the host to which *syngo.share* import is connected on mouse-over. You can select to work offline here.
-  **Job Queue**
A list of executed tasks and their status. You can view detailed information on each task. The symbol in the main menu also changes to provide visual feedback on queued tasks.
-  **Settings**
Contains the application settings.
-  **Help**
Contains this help file.

- **User**

Contains the user menu with various options, e.g. login/logout, user information, application information.

3.1.1.2 Main Area

The main area is where you do all your work to prepare documents for archiving. It changes depending on the selected main menu tab and on your user permissions.

3.1.1.3 Document List

The document list is available in the **Archiving** tab as well as on the viewing and editing screen. It contains all loaded and recorded documents. Every list entry includes a preview image (either a thumbnail or generic file icon) as well as basic document information. You can view additional information by hovering the mouse over an entry.

Use the document list to prepare documents for archiving (see Section 3.5).

3.1.1.4 Archiving Area

The archiving area is located below the document list. There, you can add keywords and select the archiving destination. See Section 3.5.5 and Section 3.7.

3.1.2 Login and Logout

Generally speaking, *syngo.share* import only requires you to log in when you want to archive documents to a destination that demands user authentication. However, you may wish to log in to make full use of all features.

The login procedure depends on the settings made by your system administrator:

- **Automatic login**

You will be automatically logged in with your Microsoft Windows user credentials.

- **Manual login**

You may be taken to the login screen at startup. You can also access the login screen manually via  **Home** or from the user menu (). To log in, enter your user name and password, select the domain (if applicable) and click on **Login**.

To log out, go to **Home** or the user menu and click on **Logout**.

Note

Always terminate logged-in sessions by clicking on **Logout** to prevent unauthorized access.

Login Message

You may be shown a login message. Accept the message to finish logging in. If you accept the message once, you will not receive it again until your system administrator sets a new login message.

If you decline the login message, your login will be canceled.

Incorrect Password

If you enter your password incorrectly too often, your user account will be locked for security reasons. Only your system administrator can unlock your account.

Screen Lock

If you are logged in, you can manually lock *syngo.share* import to prevent unauthorized access. In the **Home** tab or user menu, click on **Lock**. To continue working, enter your password to unlock the screen.

You can also log out the current user on the lock screen in order to allow another registered user to log in.

Note

If you log out the current user, this user's unsaved work will be lost.

3.1.3 Changing the Password

In the **Home** area, click on **Change password** and follow the on-screen instructions. Confirm your new password with **Change**.

Note that passwords in *syngo.share* import are case-sensitive.

Your administrator may have set passwords to expire; this means you will have to pick a new password after a certain period of time.

Note

The password provides access to patient data. Please keep it confidential and change it regularly.

3.1.4 Forced Updates

The system administrator can force a security update to be installed for *syngo.share* import. A notification window will pop up and inform you about the planned update. You can postpone the update twice for 60 seconds each time to save your work. After the second postponement, the application will be closed and updated without any possibility to save your work or further postpone the update.

3.2 Setting a Context

All documents are archived in the context of a patient or an entity. This means that there are no unassociated documents in *syngo.share* and each document can be easily retrieved at a later date.

Before you can start archiving, you thus have to set a context for the document. You can acquire and set a context by

- [Searching for an Existing Context](#)
- [Setting a Random Patient Context](#)
- [Creating a New Context](#)
- [Changing and Editing the Context](#)

Viewing the Patient Information and QR Code

Once you have set a context, the patient or entity information is displayed in the **Context** tab in the main menu. To view detailed patient information, click on **i**.

This detailed information includes a QR code for the patient ID. You can scan the QR code using the mobile app Visual Capture to quickly find the correct patient and add image, video and audio files gathered with your mobile device to the patient's health record.

3.2.1 Searching for an Existing Context

You can search for and set an existing context in one of the following ways:

- Quick Search
- Context Selection
- Automatic Context Transfer from the HIS

Note

The quick search and the manual patient/visit search in **Context Selection** only search for HL7-managed entries by default. You can adjust this behavior in the **Settings**.

3.2.1.1 Quick Search

You can use the quick search in the main menu to search for a patient ID and/or alternate patient ID and/or visit ID. Enter a string in the search field and press **Enter** to start the search. If a match is found, the context is set automatically.

Quick Search with Barcode Scanner

If a barcode scanner is available and configured, you can use it to automatically start a quick search. This method saves time and prevents typos during manual entry. Simply press the configured key sequence to activate the scanner and scan the visit ID or patient ID barcode. *syngo.share import* applies the scanned ID and automatically triggers the search.

3.2.1.2 Context Selection

In the  **Context Selection** tab, you can set a context in one of the following ways:

- **Patient/visit search**

1. Enter your search criteria in the appropriate fields. You can use `%` and `_` as placeholders (wild-cards) when searching.
2. Start the search by pressing **Enter** or by clicking on **Search**.
3. Double-click on an entry in the results list to set the context.
4. Alternatively, you can click on **List visits** to view and select a particular visit associated with a patient.

- **Entity search**

The entity search largely works the same way as the patient search. Select an entity type from the drop-down menu. The available search fields will change accordingly.

- **Worklist**

At some workstations, *syngo.share import* offers a DICOM worklist which shows work orders and patients to whom documents should be archived. Double-click on an entry to retrieve the patient context from the worklist.

- **Order list**

The order list is analogous to the worklist, but allows you to retrieve non-standardized information like keywords in addition to the patient context. These keywords are then set for the document. If necessary, you can edit the keywords before archiving the document.

3.2.1.3 Automatic Context Transfer from the HIS

If configured, you can pass patient information from the hospital information system (HIS) to *syngo.share* import via the URL interface.

Search for the patient directly in the HIS and click on the *syngo.share* import button there to transfer the data to *syngo.share* import. The context is then set automatically and you can continue working in *syngo.share* import.

3.2.2 Setting a Random Patient Context

To set a random patient context, select ▼ > **Context** > **Random...** from the main menu. Random values will be set or, if a context has already been set, replace the original values in the following fields:

- Patient ID
- Last Name
- First Name
- Sex
- Date of Birth
- Ideographic Last Name
- Ideographic First Name
- Phonetic Last Name
- Phonetic First Name
- Birth Name (if set)

Instead of using this function to set random values, you can use a so-called DICOM import profile, if configured. See Section 2.5 for the general configuration of import profiles, see Section 2.5.1.2 for configuring rules especially for the tags **Patient ID**, **Patient's Name**, **Patient's Sex**, and **Patient's Birth Date**.

3.2.3 Creating a New Context

If the context you need is not available, you can create a new patient or entity by clicking on **Create new** in the appropriate tab.

When creating a new patient, you have to fill in at least the patient ID field. Other information is optional.

When creating a new entity, you have to fill in at least one field. The input fields can vary according to entity type.

You can also generate a random patient context (see Section 3.2.2).

3.2.4 Changing and Editing the Context

To change the patient context, go to ▼ > **Context** > **Change** in the **Context** tab. You can then set a different patient context.

To edit a patient context, go to ▼ > **Context** > **Edit**. Correct or add patient details as desired and confirm your edits with **Change**. On the editing screen, you can also set a randomly generated context for the purpose of data minimization (see Section 3.2.2).

3.3 Verifying a Patient Context

You can check the patient context against a configured target archive. In the **Context** tab, click on ▼ > **Context > Verify** and select the target archive.

The result of the verification is indicated by a symbol in the **Context** tab:

- Success: If the context matches a patient in the target archive, a green checkmark is displayed. The data will be archived to this existing patient.
- Failure: If no patient match is found in the target archive, a yellow exclamation mark is displayed. The set context will be used to create a new patient in the archive.
- Ambiguous: If the context matches multiple patients in the target archive, a yellow exclamation mark is displayed and you will be taken to the **Patient Verification** tab to choose a patient from the list of potential matches. If the correct patient is not available in the list, you may have to create a new context (see Section 3.2.3).

If the automatic query for patient ID is activated under ⚙ > **Communication > IRWF-specific options**, *syngo.share* import checks the DICOM data of the documents against the selected target archive. Depending on the settings, different criteria are queried. If a patient is found and the data can be verified in its entirety, this patient context is set automatically.

3.4 Adding Documents

With a context set, you can add documents to the selected patient or entity for archiving. Go to the  **Archiving** area. There, you can load different document types from the following sources:

- File System
- Video
- Scan
- Camera
- Tooth Status
- Archive
- DICOM Q/R

Data Conflicts

When loading documents for archiving, data conflicts can occur. *syngo.share* import informs you about these conflicts by marking the respective DICOM files with a red cross (✖) or a yellow exclamation mark (!). You will not be able to archive any documents until the conflicts marked with a cross are solved. You can archive documents marked with an exclamation mark, but be aware that this might lead to conflicts.

The following overview shows what data conflicts may occur and how to resolve them. The options are accessible via the context menu or by clicking on the mark on the image thumbnail. One option is always to remove the document in question from the document list.

- The set patient context does not coincide with the information in the loaded DICOM files (✖): Confirm that you want to merge the document to the set patient context.
- The set patient context does not coincide with data already archived with the same UIDs (✖): Generate new UIDs.
- The set patient context is correct, but at least part of the data is already archived with the same UIDs (!): Generate new UIDs to avoid conflicts.

Automatically Search for Existing Documents

syngo.share import can automatically search the archive to determine if a study or images to be imported already exist in the configured target archive. This option has to be activated under  > **Communication**.

In case of a match, the image is marked with an exclamation mark in the document list and the patient context is set automatically. It is not possible to import this image to another patient. If another patient context is selected, the import is blocked. If you still wish to import the image to the set context, you have to enable the creation of new UUIDs (see Section 3.5.6).

3.4.1 File System

This option allows you to load images and other documents that already exist to *syngo.share* import, using the following methods:

- **File Browser**
- **Dragging and Dropping Files from Windows Explorer**

There is also the option to automatically delete the original files after a successful import. This can be configured in the settings.

Note

Note that documents in *syngo.share* import are always recorded with context. Before loading an image or another document, you must record the correct context. This work step is not needed when loading DICOM files that already include patient and examination data and when working with worklists.

3.4.1.1 File Browser

The file system browser resembles Microsoft Windows Explorer. It consists of a navigation tree and an area that shows the contents of the selected drive or folder. You can select a directory to browse in one of two ways:

- Scroll through the navigation tree and click on the desired folder.
- Click on  in the header bar to quickly select a specific folder, device or the Windows Favorites folder.

You can set a selected directory as your start directory by clicking on .

Use  to update the filesystem browser display in order to see any changes to the folder contents that may be happening in the background, e.g. the deletion of a file.

To safely remove an external drive or a CD click .

You can choose between two file display styles: The list view provides detailed information on the elements and lets you sort the various columns. The tile view offers a compact thumbnail view. The DICOM overview (to be selected via the CD symbol) shows all DICOM files in the selected directory (see Section 3.4.1.1).

In the footer bar, you can adjust the size of the preview icons.

Loading Files into the Document List

There are several ways to add files to the document list: Either double-click on the desired item (only possible for single files), drag and drop the element(s) into the document list, or choose **Add selected**. To select multiple files, make use of **Ctrl** or **Shift** while selecting the desired items, or use **Ctrl + A** or **Add all** to select the entire file contents.

Quick-Importing DICOM Files (e.g. from a Patient CD)

In the DICOM overview, *syngo.share* import offers a quick import function for DICOM files accessible via the CD symbol. If the selected directory or medium (e.g. a patient CD) contains a DICOMDIR directory structure, all DICOM files included therein are listed immediately. In case no DICOMDIR directory file is found, you can scan a CD, USB stick or any directory manually.

In the list of all the DICOM files found, you can click on the corresponding **Quick import** button to quick-import

- all data found,
- the data of a specific patient,
- a specific study or series.

Alternatively, drag the desired file(s) into the document list and start the import manually.

The quick-imported files will be archived to the Organizational Unit selected in the archiving area (see Section 3.7).

3.4.1.2 Dragging and Dropping Files from Windows Explorer

syngo.share import supports the option of adding files from Microsoft Windows Explorer directly to *syngo.share* import.

Select a file in Windows Explorer and drag it into the *syngo.share* import document list while holding down the left mouse button.

3.4.2 Video

In this tab, you can directly record videos as well as single images and archive them in *syngo.share* import. This requires a connected video source that supports Microsoft Direct Show. Recorded videos are saved as generic files in the archive.

Note

This tab is grayed out if no video source is connected to your computer or it is not detected/installed correctly.

Select the video source from the drop-down menu. You can make changes to the device configuration under **⚙ > Specific > Video**, see Section 2.2.10.

The video preview shows the current video stream that is transmitted from the connected video signal. To start recording, click on **Record** or use the configured foot switch.

You can pause and resume the recording by clicking on **Pause**. Clicking on **Stop** finishes the recording, and the newly recorded video file is added to the document list.

If a maximum recording duration has been set under **⚙ > Specific > Video**, the recording will automatically stop at the set amount of time.

To take a still image of the current video stream, click on **Capture image** or use the configured foot switch. Still images immediately appear in the document list.

To use a foot switch, you first have to configure its serial port under **⚙️ > Specific > Still Image Capture**.

Note

If you remove recently recorded videos or still images of the video stream from the list, these elements cannot be restored and reloaded.

For information on video editing, see [Section 3.6.2.4](#).

3.4.3 Scan

Physical documents can be scanned and archived directly in *syngo.share* import.

Select a TWAIN source (scanner) from the drop-down menu. If the scanner is not automatically detected or does not seem to be functioning correctly, you can activate **Scanner compatibility mode**.

All scanned documents show up in the preview panel. As soon as a document is chosen for archiving, it shows up in the documents list and is grayed out in the preview panel. If deleted from the documents list, the document is again available for selection in the preview panel. The preview panel can be cleared by clicking **Clear**.

Scanned documents can be saved in TIFF, JPEG, or PDF/A format.

3.4.4 Camera

In this tab, you can load images directly from a connected digital photography device (e.g. camera, tablet).

It resembles the **Filesystem** tab in functionality and appearance. Double-click on the desired device in the area or use the ▼ symbol to expand its storage directory. The right-hand area shows a preview of all the elements contained in a selected folder. You can adjust the preview area according to your needs (see [Section 3.4.1.1](#)).

Load the desired images into the document list via double-click or drag and drop (see [Section 3.4.1.1](#)).

Note

If a device is not shown correctly, check its connection and click on .

3.4.5 Tooth Status

This tab allows you to assign individual tooth X-rays to a patient's dental chart.

1. Set the patient context.
2. Load the desired X-ray image(s) into the document list.
3. Click on the image to load it into the viewer area.
4. Select the teeth visible on the X-ray by clicking the corresponding checkboxes on the dental chart. You can switch between an adult's set of teeth and children's deciduous teeth in the drop-down menu. It is possible to check different teeth on one X-ray as permanent and deciduous teeth. Furthermore, permanent and deciduous teeth of the same position can be marked on both dental notations. If applicable, you can check one or more complete teeth's quadrants at a time.

5. When you then select the next X-ray for assignment, the already assigned teeth can no longer be checked in the chart.
6. Incorrect assignments can be easily corrected by re-selecting the image in question and making the necessary corrections in the dental chart. If you want to undo all assignments at once, select **Clear** in the footer bar.
7. Assigned images are highlighted with a tooth symbol in the document list.

The **Tooth Status** tab also offers rudimentary viewing and editing tools such as contrast, rotating and zoom. The tools are available in the context menu (by right-clicking in the image) and the toolbar. You can adjust the toolbar according to your preferences: Simply right-click to open the configuration window and use the arrow buttons to add/remove tools or reorder them.

Edits made in the **Tooth Status** are also shown in the image preview in the bottom half of the screen.

Once you are done assigning images, click on **Archive**.

syngo.share import can also create a composite image from the assigned X-rays for overview purposes. Simply click on **Create image** in the footer bar.

3.4.6 Archive

This tab allows you to search for documents which have already been archived to *syngo.share* core and edit their context and properties. A variety of customizable search fields are available for use (see Section 3.4.6.1 for more information).

- **Startig a search**
Enter your search terms in the desired search fields and click on **Search**. The results are then listed in the **Studies** area.
- **Customizing search fields**
You can customize the displayed search fields by clicking on **Search ▼, Configure search fields**. In the configuration window, use ► and ◀ to add or remove search fields to your liking. Use ▲ and ▼ to change the order of the displayed search fields. You can preview your changes with **Apply**. Click on **OK** to make your changes.
- **Working with the results list**
The results list is split into the **Studies** area and the **Series** area. The **Studies** area contains the result items; the **Series** area offers a content preview when you select a specific result. For more information, see Section 3.4.6.3 and Section 3.4.6.4.
- **Loading a result to the document list**
Double-click on a result or click on **Open** to add it to the document list. This will also automatically set the archived context.

Note

Due to visibility restrictions, you might not be authorized to view all parts of a document. This can lead to an adjustment in the series and/or object count once you click on an entry in the results list.

3.4.6.1 Types of Search Fields

Depending on the type of search field, input methods vary and certain additional conditions may apply. The following search field types are available:

- **Free Text**

- Selection List
- Date
- Keyword

Free Text

In free text search fields (e.g. First and Last Name, Referring Physician, Patient ID), you can enter your search terms directly. The search is case-insensitive. You can also use wildcard characters to widen or narrow the scope of your search:

- **Wildcards % and *:** You can use % or * as a placeholder for any number of characters. Note that you cannot search using only % or *. In addition, % is automatically and invisibly appended in most free text fields, with the exception of ID fields (e.g. Patient ID, Accession Number or Visit ID) in order to avoid false positives when searching for patients via barcode scanner.
- **Wildcards _ and ?:** You can use _ or ? as a placeholder for a single character, which allows for a narrower search than % or *.

Example 1: Search using %

If you search for last names using "W%", the results will include not only names like Wheeler and Walker, but also Waterfield, Whorton and Workowski.

Example 2: Search using _

If you search for last names using "m_l", the results will include names like Miller and Muller, but not Mueller. However, longer names such as Mellerson and Mullerbach are still included because % is automatically appended to the search.

The following free text fields have special conditions:

- **Mother's Maiden Name**

The mother's maiden name may contain special characters. Due to technical reasons, the search field does not support some of these special characters. If the name you are searching for contains one or more of the special characters \ | ~ & ^, please use a wildcard at this position to conduct your search.

- **Description**

This field searches the DICOM tag **StudyDescription**. The % wildcard is added automatically to the front and back of the search term.

- **Patient Age, Study Age**

These fields require a particular syntax. In addition to the desired number, you have to enter the unit of time: y or a = years, w = weeks, d = days, h = hours. Searches with only the numerical values are not possible. To search for age ranges, add > or <; without these characters, the search will only include the precise age.

Example 3:

Searching for patients younger than 20 weeks: <20w

Searching for patients exactly 70 years of age: 70a or 70y

Selection List

In selection list search fields (e.g. Sex or Document Type), you have to choose an entry from a predefined list. Certain selection lists contain items with subcategories. Simply click on the arrow symbol next to the item in question in order to view and select a subcategory.

In certain selection fields (e.g. Modality), you can select multiple list entries to expand the search.

The following selection list fields have special conditions:

- **Region**

This search field filters for the body part or body region examined. The available items are based on the DICOM tag **BodyPartExamined**.

Date

In date fields, you can enter a date in several ways: You can either set the current date by simply clicking in the field, input the date manually, or select a date from the calendar by clicking on ▼.

The date format and calendar entries depend on the format and region settings of your operating system. Under Microsoft Windows, you can change the settings under **Control Panel > Clock, Language, and Region**.

The following date fields have special conditions:

- **Study Date**

This field combines a selection list with a date search. First, select a time period indicator from the list. If the indicator is related to the current date or time (e.g. "today" or "yesterday"), you can start the search immediately. If you want to search for a certain date ("on", "from", "until") or a certain date range ("between"), one or two date fields will open where you can enter the date.

Keyword

In keyword fields, you can search for documents in *syngo.share* core according to their keyword tags. The available keyword fields depend on your user permissions.

The method for entering a keyword varies depending on the keyword field type. Some fields allow you to select multiple keywords. To remove a keyword, simply click on x:

- Free text: Manually enter keywords of your choice.
- Selection list: Click on ▼ and select a keyword from the list. You can also enter part of a keyword manually to filter the list.
- Hierarchical selection list: If a selection list offers hierarchical tagging, the keywords are displayed in tree form. Click on ► to view and select the subordinate keywords.

A search for keywords is always an exact search, with the exception of keywords from free text catalogs. In this case, a placeholder is always added automatically at the beginning and the end of the text entered into the search field. These search terms are displayed in italics.

A search for keywords finds all documents that are tagged with at least one of the specified keywords on at least one tagging level. For example, if only one instance of a series has been tagged with a given keyword, the entire study is included in the results.

For more information on keywords, see Section 3.5.5.

3.4.6.2 Saving a Search Query

You can save frequently used search queries to avoid having to reenter them every time. Doing so will also save the entered search terms, which will be restored when the saved search query is selected.

1. Fill in the desired fields in the search area.
2. Click **Search** to start the query.
3. If you are satisfied with the result, you can click on  to save it.
4. Enter a name for the query (max. 50 characters, comma and semicolon are invalid) and click **OK**.
5. You can find the saved search query as a new button above the search field area. Clicking it will execute the saved query.

Setting a Search Query as Standard

You can set a saved search query to start automatically as soon as you log in to *syngo.share import*. Click on the arrow next to the desired search query button and select **Set as default** from the sub-menu. The standard search will be starred. At your next login, *syngo.share import* automatically switches to the appropriate tab and loads the results list of the standard search query.

Editing a Saved Search Query

Click on the arrow next to the button of a saved search query to modify it. You can edit the displayed fields, rename the query or delete it.

3.4.6.3 Results List

Similar to the search fields, you can configure the results list according to your needs. Via , you can use the arrow buttons to add or remove columns from the list or change their order of appearance.

In addition, you can

- drag columns to change the order,
- adjust the column width manually by dragging their border or automatically by double-clicking on it,
- sort the columns by clicking on them (the displayed arrow indicates the current sorting).

Grouping

You can group the results based on up to five columns. These columns have to be arranged in the exact order according to which grouping is to be performed. Grouping will subsume entries with matching values within one column into a group. In case of multiple groups, grouping is performed hierarchically from left to right. The grouping icon  appears in the header of every grouped column, indicating the number of grouping levels.

Setting Automatic Updates for the Results List

The results list is static between searches, meaning that if archive data is changed following a search query, these changes will not be reflected in the results list. You can update your results list manually by repeating the search or by setting an update interval by clicking on .

Note

The automatic update of the results list causes a higher system load. The shorter the update interval, the higher the load. To avoid an unnecessarily high system load, choose longer intervals or, if unneeded, disable the automatic update function altogether.

Copying Results to Microsoft Excel

You can copy search results to a Microsoft Excel spreadsheet as follows:

1. Select the desired items from the results list. To select multiple items at once, press **Shift** (for consecutive entries) or **Ctrl** (for non-consecutive entries).
2. Either press **Ctrl + C** or click on  to copy the items to the clipboard. If you only want to copy a particular column entry, you can also open the context menu and select **Copy**.
3. In Microsoft Excel, click inside a spreadsheet cell and press **Ctrl + V** or click on **Paste** to paste the items.

Restricted Document Parts

Documents may contain restricted elements, i.e. series or generic files which you do not have permission to view. These restricted elements will not be shown in the preview area or the document list. However, depending on your other permissions, you can still perform operations which affect the entire document.

Documents with restricted elements are indicated in the **Objects** column of the results list, which lists the total number of elements contained in a document (series or generic files). When you select a document with restricted elements, its total number of elements will be divided into visible and hidden elements.

3.4.6.4 Series Area

The series area below the results list allows you to preview the contents of selected search results and receive detailed information on the individual documents. You can adjust the area size by clicking and dragging the  symbol, or minimize it by clicking on .

A thumbnail preview of each document is shown. These thumbnails are created when documents are uploaded to *syngo.share* core; if available, they are shown for file formats which can be displayed in *syngo.share* import. Files from external applications are displayed with their generic file icon.

Further information on the documents is available on mouse-over via tooltip (e.g. Patient ID, Examination Date, Document Description). The available information varies depending on the type of document.

- **Image number/File name/File count**

Depending on the presentation options, files in the thumbnail view are displayed with their image number (DICOM series) or name (generic document) or the number of files in a stack (only available in stack view).

- **Stackability**

The stack symbol indicates elements that are stackable or, if stack view is activated, elements that have been stacked. Note that generic files cannot be stacked.

- **Tagging**

An orange triangle lets you know which files have been tagged with keywords.

- **Version**

To receive information about the document version, hover the mouse pointer over an element and click on .

When you select multiple results or when a result contains multiple documents, you can order and filter these documents in series view.

- **Filter according to ICOTAGS keywords**

You can filter the elements shown in thumbnail view by choosing to display only those tagged with a certain keyword from the special ICOTAGS catalog, e.g. "findings-relevant". Simply choose the desired keyword from the selection list.

- **Grouping and sorting**

Grouping documents is useful when e.g. multiple series belonging to different patients are shown in the thumbnail view. Select the desired entry from the grouping list to group the thumbnail elements beneath their respective headers (e.g. grouping elements by patient arranges the elements beneath headers bearing their associated patient's name). You can also change the sorting of a group by ticking the checkbox **Sort descending** (the checkbox is deactivated by default, meaning that groups are sorted in ascending order).

 Note

Grouping according to study always shows the most current study first. The older the study, the further down it is found on the list. The same is true for grouping according to examination date. If you select **Sort descending**, older groups are shown first.

- **Stack series**

You can use the **Stack series** checkbox to achieve a better overview of series containing a great number of images (>100). When stack view is activated, the number of files contained in a stack is displayed (e.g. #167) alongside the stack symbol.

- **Adjust thumbnail size**

To increase or decrease the size of the thumbnails, either drag the slider to the left or right or click on the  and  icons.

3.4.7 DICOM Q/R

This tab allows you to search for files in a configured third-party archive. If more than one external PACS has been configured, you can select the PACS you wish to search in the drop-down menu of the header bar. Similar to the archive search, there are various fields available for entering search parameters (see Section 3.4.6.1). Double-click on a result to load it into the document list.

3.5 Preparing Documents for Archiving

The document list provides an overview of all loaded documents and allows you to prepare them for archiving.

The action menu  appears when you hover the mouse over an entry in the document list. Depending on the type and level of the document (i.e. DICOM study/series/instance or generic container/generic file), various actions are available.

Click on  to perform the following actions:

- **View**

Opens the selected document in the viewing and editing area (see Section 3.6). Documents of a non-supported format are opened in an appropriate external application, if available.

- **Information**

Displays the UID and other information about a selected DICOM document.

- **Tag**

Lets you add keywords to a document before archiving. The available keywords depend on your permissions. For more information, see Section 3.5.5.

- **Create new UIDs/Use existing UIDs**

Activate/deactivate the creation of new UIDs for the selected DICOM document during import (see Section 3.5.6).

- **Rotate**

Rotates a generic image.

- **Apply generic import template**

You can select a generic import template for the XDS registration of generic documents (see Section 2.6).

- **Remove**

Removes the document from the document list.

Note

If you remove recently recorded videos or still images from the document list, these elements cannot be reloaded.

In addition, you can edit document properties, provide additional information, or convert generic documents into DICOM documents:

- [Changing the Order of Documents](#)
- [Editing Documents from the Archive](#)
- [Editing Document Descriptions](#)
- [Setting the Date and Time of an Examination](#)
- [Tagging Documents with Keywords](#)
- [Creating New UIDs](#)
- [DICOM-Wrapping Generic Documents](#)
- [Setting the Generic File Type](#)
- [Applying a Generic Import Template](#)
- [Applying a DICOM Import Profile](#)

3.5.1 Changing the Order of Documents

You can change the order of documents in the document list by dragging and dropping them to the desired position.

Generic and DICOM-wrapped documents are initially sorted according to the creation date recorded in their EXIF metadata or, if this information is missing, according to filesystem information. The order of generic or DICOM-wrapped documents in the document list is preserved during archiving. In other words, if you manually change the order of the generic or DICOM-wrapped documents, the archived documents will be listed in that order.

3.5.2 Editing Documents from the Archive

If you add documents from *syngo.share core* (**Archive**) to the document list, you can edit them just like other documents.

Since these documents have already been archived, their context is set automatically upon loading. You can edit or change the context by clicking on ▼ in the **Context** tab, i.e. you can assign the documents to a different patient or edit the patient context itself (e.g. patient ID, name, sex).

Depending on the configuration of *syngo.share* import, edited documents will either be saved separately from the original documents or the old version will be deleted and replaced by the edited version (⚙ > **Specific** > **File Modifications** > **Delete Source Files when Archived**).

3.5.3 Editing Document Descriptions

For DICOM documents, *syngo.share* import displays the description contained in the DICOM information by default. For generic and DICOM-wrapped documents, *syngo.share* import assigns either the file name to the description field or, if it is a newly captured file (e.g. a video recording or still image), a basic descriptive label.

You can edit these descriptions of DICOM and generic documents on every available level. To edit a description, simply click in the description field and enter a string. You can also completely remove an existing description by clicking on x and leaving the field blank.

3.5.4 Setting the Date and Time of an Examination

The date and time of the examination are shown in the document list. *syngo.share* import automatically sets them based on the DICOM metadata or, in the case of generic or DICOM-wrapped documents, the first available information:

1. Study/creation date from worklist or URL call-up
2. Oldest creation date from the EXIF metadata of all generic images in a generic container
3. Current date and time

You can also edit the date and time, e.g. if you want to retroactively archive a document to a patient or visit. Note that you can only edit the date and time of generic or DICOM-wrapped documents.

- To edit the date, either enter the information directly or click on ▼ and select the date from the calendar.
- To edit the time, either enter the information directly or click on ▲▼ to adjust the time.
- To quickly set the current date and time, click on 🕒.

If the study date has been set to non-editable in the settings, you can only edit the date and time if this information has not been set by the worklist or a URL call-up.

Note

If you have activated the DICOM-wrapping of generic files, the creation date from the EXIF metadata is saved separately from the date of DICOM-wrapping. In the DICOM metadata, the EXIF creation date is recorded in **ContentDate** and **ContentTime**; the date of DICOM-wrapping is recorded in **InstanceCreationDate** and **InstanceCreationTime**. For more information, see Section 3.5.7.

3.5.5 Tagging Documents with Keywords

You can use keywords to add descriptive, searchable details to a document. The keyword tags are archived along with the document. The available keyword catalogs and groups are set up by your system administrator and may vary depending on the archiving destination.

You can tag documents on the examination and DICOM series/generic file level. The tagging process varies a little depending on the document level. If you have loaded several instances of a DICOM series, any series-level keywords you select will be applied to all loaded instances.

To tag a document with keywords, follow these steps:

1. In the archiving area, select an archiving destination (Organizational Unit). If the destination provides keyword tagging, the keyword selection will be displayed.
2. To tag individual series/files, select the entry in the document list and click on **⋮ > Tag**. If configured, you can quickly select a special destination-independent keyword (ICOTAGS), or click on **Add** to see the keyword tags available for your destination.
3. To tag the examination, stay in the archiving area.
4. Select a keyword group from the drop-down menu to see the associated keyword fields.
5. Select the desired keywords. Some fields allow you to select multiple keywords. To remove a keyword, simply click on **x**:
 - Free text: Manually enter keywords of your choice.
 - Selection list: Click on **▼** and select a keyword from the list. You can also enter part of a keyword manually to filter the list.
 - Hierarchical selection list: If a selection list offers hierarchical tagging, the keywords are displayed in tree form. Click on **▶** to view and select the subordinate keywords.
6. If necessary, confirm your keyword selection with **OK**.

3.5.6 Creating New UIDs

You can generate new UIDs for documents to ensure that they are archived as new documents. This way, you can prevent accidental updates to existing documents.

To generate new UIDs by default, enable **Force creation of new UIDs** in the settings (see Section 2.2.6). This means that *syngo.share import* will automatically generate new values for the DICOM tags **StudyInstanceUID**, **SeriesInstanceUID** and **SOPInstanceUID** of every document in the document list.

You can also manually toggle the creation of new UIDs in the document list. Click on **⋮ > Create new UIDs/Use existing UIDs** to enable/disable this function for individual documents.

3.5.7 DICOM-Wrapping Generic Documents

DICOM-wrapping allows you to convert certain generic documents into DICOM documents upon archiving. This allows you to make use of the enhanced metadata of the DICOM format.

You can enable DICOM-wrapping in the settings (**⚙ > Specific > File modifications**); see Section • **DICOM-wrap**.

If you also enable the creation of new DICOM series per image/PDF/video, each DICOM-wrapped file you load into the document tree will be added as a separate series. If you decide you want to add certain files to the same series after all, you can move them freely via drag-and-drop (see Section 3.6.2.5).

Note that activating DICOM-wrapping means that you will no longer be able to perform certain actions specific to generic files, such as setting the generic file type (see Section 3.5.8). If you want to make use of these actions after all, you have to disable DICOM-wrapping first.

3.5.8 Setting the Generic File Type

The generic file type can be used to trigger certain workflows and/or affect the display order of a document in other *syngo.share* applications. The available generic file types are determined by your system administrator. You can select the generic file type for each generic file in the document list.

Note that you have to disable DICOM-wrapping to set the generic file type (see Section 3.5.7).

3.5.9 Applying a Generic Import Template

You can use generic import templates to provide additional metadata for generic documents. These templates have to be configured by your system administrator. You can set the generic import template for an entire generic container or individually for each generic file by selecting the desired level in the document list and clicking on **i > Apply generic import template**.

Note that you have to disable DICOM-wrapping to apply a generic import template (see Section 3.5.7).

3.5.10 Applying a DICOM Import Profile

DICOM import profiles are bundles of rules for editing DICOM tags (see Section 2.5). They allow you to e.g. add or correct information or minimize patient-identifying information for privacy purposes.

To apply a DICOM import profile, select it in the archiving area. Depending on the profile, new fields may appear where you have to provide additional information.

Note

Certain DICOM files cannot be displayed in *syngo.share* import (e.g. Presentation States). However, *syngo.share* import can still change their DICOM tags and archive them like any other file.

3.6 Viewing and Editing Documents

After setting the patient context and adding documents to the document list, you can view and edit images.

You can access the viewing and editing area in one of the following ways:

- Double-click on the file you wish to view/edit in the document list.
- Select **Q View** from the document list menu (i).
- Click on the context set or the icon next to the quick search field in the main menu.

Note

Unsupported file formats are opened in an appropriate external program, provided it has been installed on your workstation.

The viewing and editing area offers a variety of tools in the context menu. You can also use the listed keyboard shortcut to quickly select a tool or add frequently used tools to the toolbar at the bottom. To adjust the toolbar, right-click on it and use the arrow buttons in the configuration window to add/remove tools and arrange them in the desired order.

Note that you can also choose your preferred default tool for viewing/editing under **⚙ > Specific > Viewer > Standard tool**.

Note

Any changes you make to the image with editing tools will remain active if you clear the viewer or load another file for viewing/editing. Editing changes will be saved permanently upon archiving. However, changes made with viewing tools will neither be retained when the viewer is cleared nor saved during archiving.

3.6.1 Viewing Documents

Depending on the document type, *syngo.share* import offers a variety of viewing tools.

- **Pointer**
This is the normal mouse setting. Use it to deactivate a previously selected tool and reset the mouse to the standard pointer.
- **Scroll**
Scroll back and forth between the individual images of a DICOM series or between the individual frames of a multiframe image. Hold down the left mouse button and move the mouse down to scroll forward, or move the mouse up to scroll back. Alternatively, you can use the mouse wheel for scrolling.
- **Zoom**
Changes the zoom factor of the displayed image. Use either the mouse wheel or hold down the left mouse button while moving the mouse up (to zoom out) or down (to zoom in).
- **Magnifier**
Enlarges a section of the image. You can freely move magnifying window by dragging it with the mouse.
- **Brightness/Contrast**
Changes the brightness and contrast values of generic images like JPGs or PNGs. Drag the mouse right or left to increase or decrease the contrast. Drag the mouse up or down to increase or decrease the brightness.
- **Level/Window**
Allows you to adjust brightness and contrast of DICOM images. It is used the same as the Brightness/Contrast tool. In addition, you can use **Level/Window Presets** to apply predefined brightness/contrast values (can be configured under **⚙ > Specific**).
- **Pan**
Lets you move the image freely within the image display by dragging it with the mouse.
- **Clear Viewer**
Empties the viewer window. When using the keyboard shortcut, take care to click inside the viewer window first; otherwise the document list will be emptied instead.
- **Fit to window**
This function allows you to adjust the display of an image to the size of the current viewer window. This is useful when you have changed the display for a single image (e.g. by enlarging it with a zoom tool) and want to quickly return to an optimal display size.
- **Video Viewing**
The toolbar offers rudimentary video replay functions. ▶ and ⏸ start and stop the playback. Use the slider to adjust the volume.

3.6.2 Editing Documents

Depending on the document type, *syngo.share* import offers a variety of editing features and tools.

3.6.2.1 Annotations

Annotations allow you to do calibration and length calculation.

- **Calibration**

Draw a line whose length is known to you with the crosshair cursor (click to set a start and end point). Enter the length of the drawn line in millimeters in the calibration window. The line now serves as a benchmark for all following calculations.

- **Calibration Presets**

For taking pictures in a photo laboratory, where a camera with the same lens is used for various types of shots (e.g. full-body, face, hand, etc.) and is placed on different tripods or markers for this purpose, you can save calibration presets and use them on the captured images.

Select **Add new calibration presets** in the context menu and draw a line on a reference whose length is known to you. Enter the length of the drawn line in millimeters and name the preset. These new preset contains the camera type, the focal length of the lens as well as the custom description, e.g. Canon 70D 24mm: Full Body.

You can select more than one image and apply a chosen calibration in one go.

The existing calibration presets can be removed by selecting **Calculations > Calibration presets > Edit** from the context menu.

- **Show Length**

Draw a line using the crosshair cursor. The distance is calculated according to the current calibration. Left-click to set the start and interim points for the line. Double-click to set the endpoint. Use the pointer tool to edit your annotation by clicking on and dragging the desired line or start/end/interim points. To remove an annotation, press **Delete**. To remove all calculations at once, use the context menu entry **Remove all annotations**.

Note

If you remove an image from the document list, any annotations you may have made are not saved or archived.

3.6.2.2 Rotating and Flipping Images

The **Rotate and Flip** submenu offers a number of tools for rotating and mirroring generic images. As soon as you mirror an image, the respective tool button in the toolbar is set to active to clearly indicate that the image is mirrored. By default, mirroring and rotating is deactivated in the settings in order to prevent accidental use.

3.6.2.3 Anonymizing Image Pixel Data

Using the anonymization function, you can manually hide pixel information from the image by drawing an arbitrary polygon over the desired area of the viewer. This function can be used to obscure information in the image that could identify the patient, like patient data in an ultrasound image or a tattoo.

1. Select **Anonymize pixel data** from the context menu of the viewer.
2. Click at the edge of the area you want to obscure.

3. Continue clicking to set each additional point of your desired polygon. Note that you cannot set a point outside the image.
4. Double-click to set the final point and finish the polygon.

Pressing and holding the **ALT** key at the same time will draw the polygon over every image of the series (global anonymization).

Note

Global anonymization is not supported for images of different dimensions.

3.6.2.4 Video Editing

The video toolbar offers the following editing functions:

• Video Cutting

You have two means of shortening a video:

- By dragging the start and/or end marker in the cutting bar (below the playbar) to the desired position. The video will be trimmed accordingly and only show the trimmed sequence upon replay.
- By cutting a sequence from the middle of the video. Move the play marker to the desired spot, which will cause a small dot to appear in the cutting bar. Drag this dot to the left or right to select the sequence to be cut. You can adjust the selected sequence to your liking. Click outside the selected sequence to cut it. To undo the edit, click on ↶.

Note

Cutting sequences from the middle of the video is not possible when you have configured an external application (FFmpeg) for trimming. FFmpeg only allows the editing of start and end points.

Note

Note that video cutting has a tolerance for imprecision of up to four seconds.

• Bookmarking and Commenting

You can mark and comment on parts of a video by clicking on 📌 and entering a description. Bookmarks appear as small markers in the playback bar. To remove a bookmark, either click on its position on the slider and press **Delete** or click on ▼ next to 📌 and select **Remove bookmark**.

• Capturing Still Images

To capture a still image from a video, click on 📷. The still image is added to the document list and can be archived as a generic file or DICOM image. To archive a still image as a DICOM image, you have to enable DICOM-wrapping in the settings (see Section 3.5.7).

3.6.2.5 Splitting DICOM Series and DICOM-Wrapped Documents

You can split DICOM series and DICOM-wrapped generic documents which contain more than one instance into multiple series. Note that you have to enable DICOM-wrapping in the settings in order to be able to split generic documents (see Section 3.5.7).

To split a DICOM series or DICOM-wrapped document:

- Click on the **Split** icon (✂️) to open a preview bar showing all the instances of the current series.

- Select the instances you want to split and simply drag them to the document list to split the series.
- Alternatively, you can select the images and click on **Split** next to the preview bar to accomplish the same.
- The new series will be shown in the document list immediately.

If you enable the creation of new DICOM series per generic image/PDF/video under **⚙ > Specific > File Modifications**, each file you load into the document tree will be added as a separate series. In other words, splitting will happen automatically.

Merging Split Series

You can freely move instances from one series to another by dragging them with the mouse. Moreover, you can easily merge split series again by dragging one series towards the other in the document list.

Note that you can only move instances between series of the same study and you can only merge series that were previously split.

3.7 Archiving Documents

Once you have finished preparing a document, you can archive it as follows:

1. In the archiving area, select the archiving destination (Organizational Unit) under **Destination**.
2. Check to make sure that you have provided all necessary information. You have to at least set a context and fill in any required fields. *syngo.share* import provides feedback about missing information.
3. Click on **Archive** or press **F8** to start the archiving process.

You can continue working while archiving is in progress. *syngo.share* import automatically clears the context and document list once archiving is complete.

Note

syngo.share import immediately closes once archiving is complete if you have enabled **Close automatically when finished** in the settings. If you would like to continue working and track the archiving progress, be sure to disable this option.

3.8 Tracking the Archiving Progress

The **Job Queue** icon in the main menu () provides visual feedback on the archiving process. Open the **Job Queue** for an overview of all current jobs and more detailed information.

Canceling the Archiving Process

You can cancel the archiving process by going to the **Job Queue** and clicking on **Cancel**. By default, any data that has already been transferred will remain in the archive.

Delaying the Archiving Process

If you need to delay the import, you can click on  > **Work offline** in the main menu. You will have to manually reestablish the connection in the same way before you can transfer data to the archive again.

Retrying a Failed Archiving Attempt

In case the connection between *syngo.share* import and the archive is disrupted, *syngo.share* import will keep the newly acquired data until the connection can be reestablished. At this point, *syngo.share* import will automatically retry the archiving attempt once. You can also manually retry the import by going to the **Job Queue** and clicking on **Retry**.

3.9 Keyboard Shortcuts in *syngo.share* import

General

F1	Help
F6	New context
F7	Random context
F8	Archive
F10	Switch between archiving and editing area

Search

F5	Start search
Crtl + C	Copy selected search results

Viewer

→	Next image in series
←	Previous image in series
Page ↓	Next entry in document list
Page ↑	Previous entry in document list
Del	Remove image from viewer
Esc	Pointer tool
B	Scroll through series
F	Windowing
Shift + F	Default windowing value
Shift + 1-3	Windowing presets
L	Length
V	Pan
Z	Zoom
Backspace	Reset

Document List

F2	Edit context
F3	Change context
F4	Clear document list
Del	Remove all selected entries from document list
↓	Next entry in document list
↑	Previous entry in document list

Video Viewer

Space	Play/Pause
→	Next video frame
←	Previous video frame
↓	30 seconds forward
↑	30 seconds backward
Home	Move to beginning of video
End	Move to end of video
F5	Capture screenshot

PDF Viewer

Ctrl + A	Select all
Ctrl + Home	Move to beginning of document
Ctrl + End	Move to end of document
Page ↓	Next page
Page ↑	Previous page

Video Capture

Ctrl + Shift + Alt + R	Start recording
Ctrl + Shift + Alt + S	Stop recording
F5	Capture screenshot
Ins	Set/edit bookmark

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© ITH icoserve technology for healthcare GmbH / Siemens Healthcare GmbH 2024. All rights reserved.
Siemens Healthcare GmbH
ITH icoserve technology for healthcare GmbH