Release VA32C_BF04 / 2024-07-09 / Revision 9813

syngo.share webview

Online Help

Language Directory

Deutsch	3
English	66
Français	124

Release VA32C_BF04 / 2024-07-09 / Revision 9813

OnlineHilfe

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syngo.share webview

Online Help

The instructions for use are available in the information about the product in the user menu.

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	-		
	2.4.3	Wildcard	Is and Further Actions
	2.4.4	Search M	Modes
		2.4.4.1	Smart Search
		2.4.4.2	Classic Search
	2.4.5	Saving a	Search Query
		2.4.5.1	Editing a Saved Search Query
		2.4.5.2	Setting a Search Query as Default
2.5	Result	s List	
	2.5.1	Working	with the Results List
		2.5.1.1	Filtering Search Results
		2.5.1.2	Exporting Search Results
		2513	Customizing the Results List Lavout 90
		2514	Setting Automatic Lindates for the Results List 90
	252	Preview	ing Documents 91
	2.5.2	Referen	ng Documents
	2.5.5	Othor Ex	$\frac{1}{2}$
26	Z.J.4 Dation	t Informat	aninations
2.0	Contor	t IIIIOIIIIdt	
Z./			
	2.7.1	Opening) Documents
		2.7.1.1	Opening Documents in the Viewer
		2.7.1.2	Opening Documents in External Applications
		2.7.1.3	Opening Documents in Concentriq $Dx^{(i)}$
		2.7.1.4	Retrieving Documents
	2.7.2	Quality	Assurance Operations
		2.7.2.1	Deleting Documents
		2.7.2.2	Copying Documents
		2.7.2.3	Moving Documents
		2.7.2.4	Reassigning Documents
		2.7.2.5	Rearranging Studies and Series
		2.7.2.6	Viewing and Editing Patient and Document Properties
		2.7.2.7	Setting a Document Workflow State
		2.7.2.8	Rotating Generic Images
		2.7.2.9	Setting the Thin Slice State
		2.7.2.10	Viewing and Correcting Warnings and Errors
	2.7.3	Keyword	d Tagging
	2.7.4	Searchir	ng Externally
		2.7.4.1	Searching in Other syngo share Applications
		2742	Searching for External Documents
	275	Distribu	ting Documents 101
	2.7.5	2751	Sending Documents 101
		2.7.5.1	Exporting Documents 107
		2.7.5.2	Downloading Documents 102
		2.7.5.5	Province and Poly Poly Poly Poly Poly Poly Poly Poly
		2.7.3.4	Registering Decuments in the Patient Portal
		2.7.5.5	Charing Documents III the Patient Portal
	276	2.7.3.0 Dational	Sharing Documents
2 0	2.7.0		Telateu Activits
۷.۷	viewir		enus
	2.8.1	Patient I	sanner
	2.8.2	Docume	ent Navigator
	2.8.3	Patient	Jacket
	2.8.4	Viewpor	t Area
		2.8.4.1	General Tools

	2.8.4.2 Navigation Tools	108
	2.8.4.3 Viewing Tools	109
	2.8.4.4 Display Tools	110
	2.8.4.5 Annotation and Calculation Tools	111
	2.8.4.6 Capture Tool	113
	2.8.4.7 Special DICOM Data Tools	113
	2.8.4.8 Playback Tools	115
	2.9 Inbox	116
	2.9.1 Actions	116
	2.9.2 Requesting Documents	117
	2.9.3 Availability Period	117
	2.10 Printing Documents	117
	2.10.1 Printing in the Search Tab	117
	2.10.2 Printing in the Viewer	117
	2.11 Archiving Documents	117
	2.12 Job Queue	119
	2.13 Keyboard Shortcuts in syngo.share webview	120
3	Emergency Access	121
4	Mobile Devices	122
	4.1 Functionality	122
	4.2 Touch Gestures	123

1 Introduction

1.1 Intended Use

syngo.share webview is a web application to be used in clinical image and result distribution and provides access to clinical data from connected systems (such as Vendor Neutral Archives, Picture Archiving and Communication Systems, DICOM archives, XDS/-I Affinity Domains, Electronic Medical Record Systems, Clinical Information Systems) and the local file system for the electronic display of data. This product is not intended for diagnosis, prevention, monitoring, prediction, prognosis, treatment, cure or alleviation of diseases or other medical conditions.

1.2 Safety Notices

Image and Result Distribution

syngo.share webview is only intended for clinical image and result distribution. **Do not use** *syngo*.share webview for direct diagnoses.

Medical Education

The use of syngo.share webview should be granted to qualified staff members only.

Accuracy of Calculations

Images may contain calibrations. Please note that the accuracy of these calibrations correlate to the precision of the program or modality with which they were created. Calibrations must be checked according to the manufacturer's instructions. Calculations, calibrations and results derived therefrom must be checked for plausibility by clinical professionals. Inaccuracies may arise during image capture by clinical professionals or due to the image capturing technology of a given modality. For instance, X-ray images may contain inaccuracies due to the fact that the x-rayed body part is projected onto a two-dimensional imaging surface. Inaccuracies during visualization may be due to browser or hardware settings.

This product rounds calculations as follows: Length and area: 1 decimal place; density (HU): no decimal place; area density (HU): 1 decimal place. This product uses an interpolation algorithm for image display.

Units

The application *syngo*.share webview conforms to Directive 80/181/EEC, which contains specifications related to units of calculation.

- Calibrated lengths are calculated in millimeters (mm). Uncalibrated lengths are calculated in pixels (pixel).
- Calibrated surface areas are calculated in square millimeters (mm²). Uncalibrated surface areas are calculated in square pixels (pixel²).
- For DICOM Whole Slide Images (WSI), the unit display changes to micrometers (μm) / square micrometers (μm^2) if the length falls below 1 mm or the surface area falls below 0.1 mm².
- Plane angles are calculated in degrees (°).
- Densities are calculated using the industry-standard Hounsfield unit (HU), which is not regulated by Directive 80/181/EEC.

Access Restriction

Be sure to log out of *syngo*.share webview when finished working in order to avoid unauthorized access to the system.

Printing of Images

To print, *syngo*.share webview transfers the data to the print function of the browser. Thus there may be differences between the view in the Image Display Area in *syngo*.share webview and the printout, e.g. the image could be cut off, annotations could be incomplete or missing entirely.

Possible transformations such as mirroring, rotation, and applied presentation states are not marked explicitly in the printout.

Effects of External Programs

The software manufacturer does not guarantee that third-party applications will not negatively affect the system. The standards of any external programs accessed by *syngo*.share webview are set by their respective manufacturers.

Network Connection

syngo.share webview requires an active network connection. For IT security reasons, a (temporary) failure of the network connection may result in you being logged out and having to log in again.

Workstation Accessibility

When using *syngo*.share webview, make sure that you can operate the application at all times.

1.3 Technical Terms and Notation

1.3.1 Generic Document

This term refers to any non-DICOM file that can be viewed in or archived by *syngo*.share. Non-DICOM files include images produced by digital cameras or imaging applications (e.g. JPEG, MPM, TIFF) as well as various text, video, and audio file formats (e.g. PDF, DOC, MPEG4, MP3) produced by various external applications (e.g. Microsoft Office, Adobe Acrobat).

Generic files are wrapped in generic containers, analogous to DICOM studies.

1.3.2 DICOM Document

A DICOM document consists of at least one DICOM study, series and instance.

DICOM documents are produced by medical imaging devices compliant with the DICOM standard. They can be viewed in or archived by *syngo*.share.

• Study

A DICOM study contains all the data of a medical examination during which image data is acquired. This includes the actual image data (e.g. X-ray, CT, ultrasound) as well as the patient data and descriptions relevant to the examination (such as the examination date or type). A DICOM study contains 1-n DICOM series.

• Series

A DICOM series usually equates to a specific type of data acquired/produced by a particular device or to the position of the patient during acquisition. A series contains 1-n DICOM instances.

Instance

A DICOM object instance (or simply DICOM instance) is the actual data captured by the device, such as an image or waveform object.

1.3.3 DICOM Information

DICOM (Digital Imaging and Communications in Medicine) is a standardized file format used in medical imaging.

In addition to object data (typically image data), the DICOM format contains additional information such as patient name, admission date, device parameters, and physician's name. The DICOM format also contains the information necessary for data exchange between DICOM-capable devices.

1.3.4 EXIF Information

EXIF (Exchangeable Image File Format) is a standardized data format generated by, e.g., digital cameras when taking a picture.

In addition to the image data commonly contained in JPEG or TIFF format, the EXIF format contains additional information describing the image in more detail, such as date and time information or the type of device used for image capture.

1.3.5 Modality

An imaging modality is a device for acquiring medical images such as X-rays or CTs. In *syngo*.share webview, modalities are identified by abbreviations such as CR for Computed Radiography (e.g. when searching).

1.3.6 Organizational Unit

An Organizational Unit is an association of arbitrary actors in a medical organization, e.g. a department, medical devices, or groups of people. Organizational units can be customized and expanded with more constituents as needed, and these constituents can be uniformly managed.

All data to be archived is allocated to exactly one organizational unit.

1.3.7 Entity

In *syngo*.share, entities are an alternative way to identify documents that do not belong to a patient or visit. This is useful e.g. for archiving documents from clinical studies and trials.

1.3.8 Hounsfield Unit

Density calculations of body tissue with the density calculation tool use the Hounsfield scale. The following equation is used to calculate the attenuation coefficients of various body tissues in CTs. It is assumed that the absorption value of tissues and the physical density have a linear dependence (μ = linear attenuation coefficient):

$$HU = \left(\frac{(\mu Object - \mu Water)}{\mu Water}\right) \times 1000$$

Table 1: Relative Attenuation Coefficients of Body Tissues and Water

Coefficient	Object
0	Water
1000 bis 100	Bone
100 bis 0	Liver/Muscle/Kidney
-100 through -200	Fat
-1000	Air

The relative attenuation values can be determined easily from any point or area of the image. Although it is technically possible to render any number of attenuation values in grayscale, the human eye is not equipped to distinguish between more than 20 shades of gray. Instead, the use of windowing is recommended in order to visually highlight different kinds of body tissue. ²

Please read the safety notices, see Section 1.2.

1.4 Supported File Formats

syngo.share webview supports the viewing of various DICOM and generic files listed in the following.

1.4.1 DICOM Storage SOP Classes

syngo.share webview specifically supports the DICOM Storage SOP Classes listed below. In addition, *syngo*.share webview supports viewing of all SOP Classes used for the purpose of image and video storage. Technical limitations (e.g. regarding the Transfer Syntax) may apply.

- 12-lead ECG Waveform Storage (1.2.840.10008.5.1.4.1.1.9.1.1)
- General ECG Waveform Storage (1.2.840.10008.5.1.4.1.1.9.1.2)
- Grayscale Softcopy Presentation State Storage (1.2.840.10008.5.1.4.1.1.11.1)
- VL Whole Slide Microscopy Image Storage (1.2.840.10008.5.1.4.1.1.77.1.6)
- Basic Text SR Storage (1.2.840.10008.5.1.4.1.1.88.11)
- Key Object Selection Document Storage (1.2.840.10008.5.1.4.1.1.88.59)
- X-Ray Radiation Dose SR Storage (1.2.840.10008.5.1.4.1.1.88.67)
- Radiopharmaceutical Radiation Dose SR Storage (1.2.840.10008.5.1.4.1.1.88.68)
- Encapsulated PDF Storage (1.2.840.10008.5.1.4.1.1.104.1)

1.4.2 Generic Files

syngo.share webview supports the following generic files for viewing:

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Video/Audio

syngo.share webview supports the playback of various video and audio file formats that use the following codecs:

AAC	MP3	VP9
FLAC	MPEG1	WMA
H.264	MPEG2	WMV1
H.265	MPEG4	WMV2
INDEO2	OPUS	WMV3
INDEO3	PCM	
INDEO4	VP8	

This means that *syngo*.share webview supports the playback of e.g. M4A audio files that use AAC codec or MPEG video files that use the MPEG-2 codec.

Text

CDA	JSON	TXT
CSV	PDF	XML

2 Usage Instructions

2.1 Notes Regarding the Usage of syngo.share webview

Results List Updates

Search results always reflect the status of documents in the archive at the precise moment of the query. Note that the results list is not updated automatically if you or another user make changes to documents after you have triggered the search. Either rerun the search manually or set an automatic update interval for the results list (see Section 2.5.1.4) to see the most recent data.

Caching of Document Data

When viewing documents, note that the document data is cached for a maximum of 10 minutes by default. This caching speeds up reopening documents and applying tools, e.g. windowing of DICOM images. Changes to cached documents only become visible when the data is automatically removed from the cache. *syngo*.share core documents are cached on the user level.

Browser Zoom

syngo.share webview is not optimized for use with activated browser zoom. Please disable any zoom settings in your browser preferences.

Browser-Dependent Differences

Different web browsers render websites differently, which might lead to slight differences in the display of *syngo*.share webview. These display differences do not affect the functionality of *syngo*.share webview. However, certain proprietary browser features, e.g. picture-in-picture display of videos or auto-fill, might interfere with the user interface of *syngo*.share webview. *syngo*.share webview cannot control these browser features; you may have to turn them off in the browser settings.

Scope of Functions Described

These usage instructions provide a comprehensive description of the functions of *syngo*.share webview. Due to the configuration settings of your organization or missing permissions, some of these functions may not be available to you. If you have any questions, please contact your system administrator.

2.2 Getting Started

2.2.1 Login and Logout

On the login screen, enter your user name and password, select the domain (if applicable) and click on **Login**.

To log out, select **Logout** in the user menu. Always terminate your sessions by logging out in order to prevent unauthorized access.

Note

Not all options may be available for your institution or workstation, meaning that you may not see some or any of these messages and/or the logout option.

At/after login, the following events may occur:

• Login message

You may be shown a login message. Click on **Accept** to finish logging in to *syngo*.share webview. If you accept the message once, you will not receive it again. If you click on **Decline**, your login will be canceled.

Incorrect password entry

If you enter your password incorrectly too often, your user account will be locked for security reasons. The lock can only be undone by your system administrator.

• Screen lock or automatic logout

syngo.share webview may engage a lock screen or initiate an automatic logout to prevent unauthorized access. The time remaining until screen lock or logout is visualized next to your user name.

• Screen lock

This function will blur the screen and display the lock screen. To continue working, enter your password on the lock screen. This unlocks *syngo*.share webview and you will see the current status of your work. You can also log out the current user on the lock screen in order to allow another registered user to log in.

Note

Note that logging out the current user means that this user's unsaved work will be lost.

• Automatic logout

Automatic logout logs out the current user, and the login screen is displayed.

You can change your password in the settings, see Section 2.3.1.1.

2.2.2 Main Menu

Use the main menu at the top to access to the main features of *syngo*.share webview:

• **iQ** Patient Search

In the Patient Search tab, you can search for documents within a patient context. See Section 2.4.1.

• 🕒 Entity Search

In the **Entity Search** tab, you can search for documents within an entity context. An entity is an alternative way of identifying documents in *syngo*.share, e.g. for clinical studies. See Section 2.4.2.

• 🗖 Inbox

In the **Inbox** tab, you can access all the documents that have been shared with you. See Section 2.9 for more information.

• 🧘 Archive

In the Archive tab, you can archive documents. See Section 2.11 for more information.

• Patient Banner/Viewer

This tab only appears when you open at least one document for viewing. In the Viewer, you can view the document in detail and perform various actions. See Section 2.7.1 and Section 2.8 for more information.

• 🗐 Job Queue

The **Job Queue** tab provides a list of potentially time-intensive operations. See Section 2.12 for more information.

• User Menu

The user menu opens when you click on your user name. It contains various functions, including the settings (see Section 2.3 for more information), the logout option and product information.

2.2.3 Adjusting the User Interface

You can hide or display certain parts of the user interface using \checkmark or \succ . You can also adjust the width and/or height of certain areas by clicking and dragging the mouse in the desired direction.

2.2.4 Full-Screen Mode

syngo.share webview can also be used in full-screen mode. You can activate and deactivate the full-screen mode by opening the user menu and selecting **Full Screen**.

2.3 Settings

You can open the settings window from the user menu.

All settings changes (except for a password change) have to be confirmed by clicking the **Ok** button. You can undo your changes (except for a password change) by clicking on **D** next to the changed setting. Some changes require a page reload to become effective.

2.3.1 General

2.3.1.1 User Profile

Password

Click on **Edit** and enter your old as well as your new password in the respective fields. Save by clicking **Confirm**.

Note

Follow the instructions on changing your password. These may inform you that your password needs to contain certain characters or be of a certain length. The **Confirm** button is only set to active if you fulfill all password conditions. Note that passwords are case-sensitive.

If the **Edit** button is deactivated, this means that your user access is managed by an external system (e.g. LDAP) and you are not allowed to change your password. Please contact your system administrator.

2.3.1.2 Language and Date

• Language

By default, *syngo*.share webview uses the browser language. You can select a different language from the drop-down menu.

• Date Format

By default, *syngo*.share webview uses the date format of the tenant. You can select a different date format from the drop-down menu.

2.3.1.3 Open Documents

Double-click

You can choose if a double-click on a document opens a viewer tab or a pop-up window. This setting is overruled if your system administrator has configured an external application to open on double-click for that specific case.

• Pop-ups

You can choose if a new pop-up window is opened for each document or if newly opened documents are displayed in an already open pop-up window.

2.3.2 Search

2.3.2.1 Other Examinations

You can choose the position of the initial examination in the Viewer when comparing with an other examination (see Section 2.5.4).

2.3.3 Tools

2.3.3.1 Mouse

You can map certain tools to the mouse buttons. Open the drop-down menu for each mouse button to assign one of the available tools. Mouse buttons not supported by your hardware are grayed out. The tools are described in the chapter Section 2.8.4.

2.3.4 Viewer

2.3.4.1 Image Quality

You can adjust the display quality of images for viewing and editing. This is particularly useful when you are accessing *syngo*.share webview over a slower network connection. The current quality value is shown in a text overlay for each image (see Section • **PNG** for more information).

Changes to the image quality settings are only applied to new viewports. If you want to apply them to images in an existing viewport, you have to close the viewport and reopen the images.

Please read the Section 1.2 before adjusting the image quality settings. The following settings are available:

• JPEG quality for interaction

You can choose between **Manual** and **Adaptive**. Selecting **Manual** allows you to determine all image quality settings yourself. Selecting **Adaptive** means that *syngo*.share webview will continuously adjust the display quality for interactions depending on your network connection.

• JPEG quality value for interaction

If you set the display quality to **Manual** above, you can move the slider to select the quality value for interactions. When you perform continuous actions such as zooming, scrolling or windowing in the Viewer (see Section 2.8), the quality of the displayed image is adjusted to this value.

• JPEG quality value for final display

Move the slider to set the regular display quality. When you stop performing continuous actions in the Viewer, the quality of the displayed image is adjusted to this value.

2.3.4.2 Playback

For details on playback of videos and multi-frame series, see Section 2.8.4.8. The following settings are available:

• DICOM image playback FPS

You can set a standard FPS rate for DICOM multiframe series playback. In cases where the DICOM multiframe instance already contains a recommended FPS value, this recommended value will be preferred.

• Autoplay DICOM modalities

You can enable autoplay for certain DICOM modalities. Only the selected modalities will enter automatic playback when opened.

• Autoplay videos

You can enable or disable autoplay for all videos on a general level. If enabled, all kinds of videos, regardless of their modality or whether they are DICOM or generic, will play back automatically.

2.3.4.3 Visualization

• Viewport coloring per document

You can switch on and off the color-coding of viewports for multiple documents of the same patient. For details on viewport coloring, see Section 2.8.2 and Section 2.8.4.

2.4 Searching for Documents

syngo.share webview offers two ways of searching for documents:

• 🔁 Patient Search

In the Patient Search tab, you can search for documents within a patient context. See Section 2.4.1.

• 🕒 Entity Search

In the **Entity Search** tab, you can search for documents within an entity context. An entity is an alternative way of identifying documents in *syngo*.share, e.g. for clinical studies. See Section 2.4.2.

2.4.1 Patient Search

To start a patient search, proceed as follows:

- 1. The Patient Search tab is active by default when you log in to syngo.share webview.
- 2. Select a search mode (see Section 2.4.4).
- **3.** Enter your search terms. You can refine or broaden your search by combining different terms. You can clear a search term by clicking on \mathbf{x} in the search field, or clear the entire search at once by clicking on $\mathbf{T}\mathbf{x}$.
- **4.** You may have more than one archive available for searching. In this case, you have to select at least one archive before you can start the search. Click on **Sources** to select one or multiple archives.
- 5. Press Enter or click on Search to start the search. You can cancel the in-progress search at any time by clicking on Cancel.
- **6.** The results will be displayed in the results list below the search area. See Section 2.5 for information on customizing and working with the results list.

When you run a search in third-party archives, certain search fields either cannot be considered or are unavailable.

2.4.2 Entity Search

The entity search works in a similar way to the patient search but offers entity-specific search fields. To start an entity search, proceed as follows:

- 1. Click on the Entity Search tab and select an entity from the drop-down menu.
- 2. Enter your search terms. The available fields depend on the selected entity.
- **3.** Press **Enter** or click on **Search** to start the search. You can cancel the in-progress search at any time by clicking on **Cancel**.
- **4.** The results will be displayed in the results list below the search area. See Section 2.5 for information on customizing and working with the results list.

2.4.3 Wildcards and Further Actions

You can use the following characters as wildcards (placeholders) for all search terms:

- % and * stand in for any number of characters.
- _____ and ? stand in for a single character.

Note

% is also automatically appended to the search term, except in ID search fields (e.g. **Patient ID**, **Accession Number**, **Visit ID**). This avoids false positives when you search for patients using a barcode scanner.

Searching for "M%er" will provide results such as Mustermann and Musterfrau, but also Meier, Mayer, Moser or similar. If you only enter "m", % is automatically appended and *syngo*.share webview searches for all records that begin with "m".

Example 2: Using the

Searching for "m_ier" will provide results such as Meier and Maier. Because % is automatically appended, results like Meierle and Maierle will also be found.

You can also do the following:

- Switch between search modes (see Section 2.4.4)
- Convert searches from one mode to another (see Section 2.4.4)
- Customize your search with a variety of search fields (see Section 2.4.4.2)
- Save custom search queries (see Section 2.4.5)

- Edit your saved search queries (see Section 2.4.5.1)
- Set a saved search query as default (see Section 2.4.5.2)

2.4.4 Search Modes

The Patient Search offers two search modes:

Smart Search

The *Smart Search* mode lets you quickly and easily search for documents by recognizing your inputs and offering search field suggestions for selection. You can run searches using only your keyboard. No additional search field customization is necessary. The *Smart Search* supports the most common search fields. See Section 2.4.4.1 for more information.

• Classic

The classic search mode offers a variety of search fields that you can customize according to your needs. Input methods vary depending on the type of search field. See Section 2.4.4.2 for more information.

You can switch between the classic and *Smart Search* mode by clicking on $\Rightarrow > Q$ Search mode and selecting the desired mode. In this manner, you can also convert existing searches (see Section 2.4.5).

Note

Since the *Smart Search* supports a limited number of search fields, not all searches can be fully converted. If you choose to convert a classic search with filled search fields that are not supported by the *Smart Search*, these fields will be ignored. If your classic search contains invalid search terms, it cannot be converted until you correct those search terms.

2.4.4.1 Smart Search

The *Smart Search* supports the most common search fields (see Section 2.4.4.1). When you enter a search term, the *Smart Search* suggests appropriate search fields in the form of a drop-down list. Use the arrow keys to select the desired field for your search term and press **Enter** to add the term to the *Smart Search* (you can also make your selection with the mouse).

Smart Search suggestions are based on several criteria, such as whether the search term is an exact match for a field. The Smart Search will offer more refined suggestions the more you use it, i.e. if you select particular fields more often, these fields will be ranked higher as a suggestion. To receive the last suggestion, press \downarrow .

If you have selected the local archive as a data source, the *Smart Search* provides the predicted number of results in the local archive before you execute the search. If this predicted number is high, you may want to refine your search terms before executing the search. Since the predicted number is based on data in the local archive, the actual number of search results may differ if you have selected more data sources (see Section 2.4.1).

Supported Search Fields in the Smart Search

Table 2 lists the currently supported search fields and the inputs that will cause them to be presented as suggestions. If a search field is not available in the *Smart Search*, please select classic mode for searching.

Note that the *Smart Search* recognizes your inputs based on the current language and date format settings (see Section 2.3.1.2). Input matching for suggestions is not case-sensitive.

The *Smart Search* automatically corrects certain inputs such as invalid dates. For example, if you enter "29.02.2022" as a birth date, the *Smart Search* converts your input to the next valid date, in this case, "01.03.2022".

Search Field	Input	Additional Information
Accession Number	String/Number	Enter a number for this field to be ranked higher as a suggestion.
Assigned Patient Location	String	Multi-selection is possible. Use HL7 notation for this field to be ranked higher as a suggestion.
Birth Name	String	-
Body Part	String	-
Date of Birth	Date	Only selectable when the field Patient Age is not selected.
Description	String	-
Document Workflow State	Fixed list	Multi-selection is possible.
First Name	String	Only selectable when the field Name is not selected.
Keyword Fields	String	Multi-selection is possible.
Last Name	String	Only selectable when the field Name is not selected.
Modality	Fixed list ^a	Multi-selection is possible.
Mother's Maiden Name	String	Only selectable when you have already selected at least one of the following search fields: First Name , Last Name or Name . If the name contains one or more special characters listed in Table 5, please use a wildcard (see Section 2.4.3).
Name	String	Combination name field. Only selectable when you have not selected First Name or Last Name . For details on valid inputs, see Section 2.4.4.2.
Patient Age	Number + Time unit	Only selectable when the field Date of Birth is not selected. For the available time units see Section • Patient Age/Study Age .
Patient ID	String/Number	Enter a number for this field to be ranked higher as a suggestion.
Producer	Fixed list	Multi-selection is possible; available producers depend on your permissions
Sex	Fixed list	Suggested when you enter "Male", "Female", "Other" or "Unknown".
SSN	String/Number	Enter a number for this field to be ranked higher as a suggestion.
Station Name	String	
Study Age	Number + Time unit	Only selectable when the field Study Date is not selected. For the available time units see Section • Patient Age/Study Age .
Study Date	Date/fixed list	Specify dates or time periods by entering a date and selecting "on", "from", "to" or "between" from the suggestions. To receive "between" as a suggestion, you have to enter two dates separated by - or a space. Search for recent documents with relative terms like "last 4/8/12/24 hours", "yesterday", "today", "last 2/3/7/30 days". Only selectable when the field Study Age is not selected.
Visit ID	String/Number	Enter a number for this field to be ranked higher as a suggestion.
XAD Patient ID	String/Number	Enter a number for this field to be ranked higher as a suggestion.

Table 2: Supported Search Fields in the Smart Search

^a Search field is suggested when you enter a string that matches a term on the fixed list of this search field, e.g. if you enter "CT", the search field **Modality** will be suggested since "CT" is on the fixed list for this search field.

2.4.4.2 Classic Search

The classic search offers a wide variety of search fields in the categories **Patient**, **Document**, **Order**, **Series**, **Visit**, and **Keywords** (if defined). You can show or hide these search fields according to your needs. To customize your search, proceed as follows:

- Click on > Configure search fields. The configuration window shows the available search fields and the displayed search fields. For details on the various kinds of search fields, see Section 2.4.4.2.
- 2. Add or remove a search field by selecting it in the appropriate column and clicking on \rightarrow or \leftarrow .
 - To select multiple entries at once, press Shift or Ctrl while clicking.
 - To add a search field at a specific position, additionally select an entry in the displayed search fields column. The new search field will be added directly below this entry.
- **3.** Change the order of the displayed search fields by selecting the fields you want to rearrange and clicking on \uparrow or \downarrow .
- 4. Optionally, you can preview your changes by clicking on Preview.
 - To undo your unsaved changes, click on Reset.
 - To completely revert all customization, click on **Restore default**.
- 5. Save your changes by clicking on **Ok** or reject them by clicking on **Cancel**.

Types of Search Fields in the Classic Search

Each search field is of a certain type that determines how you can enter your search terms. The following types of search fields are available in the classic search:

• Free text

Most search fields are free text fields, such as **Last Name**, **First Name** or **Patient ID**. You can enter your search term directly or paste text from the clipboard by pressing **Ctrl + V**. Leading and/or trailing spaces and/or tab characters are removed automatically. Free text fields are not case-sensitive for searches in *syngo*.share core.

• Selection list

In selection list fields, you have to click on \checkmark and select an entry from the predefined list. Some selection list fields allow multi-selection (e.g. **Modality** or **Producer**), meaning you can simply select one entry after the other. To remove a selected entry, click on the x right next to it.

• Date

In date fields, you can either enter the date manually or select it from the calendar by clicking on **i**. The calendar shows you the current date format.

• Keyword

In keyword fields, you can enter a keyword manually or select it from the associated keyword catalogs. To remove a particular keyword, click on \mathbf{x} right next to it.

- If you enter a string and select a keyword from the suggestions in the drop-down menu, the search will only include this exact keyword. If you enter a string and simply press **Enter**, your input will be shown in italics and the search will include all keywords containing this string.
- To use the keyword catalogs, click on . The catalogs show the keywords in hierarchical order. Click on ▶ to expand further hierarchical levels. To filter the catalog, enter part of the desired keyword.

In case is not displayed, the loading of keywords has been restricted to improve performance. Use the filter option as described to find the desired keyword.

Click on an entry in the catalog to add it to the search field. You can add more than one keyword to a search. The search automatically includes all keywords that are hierarchically below the selected level.

Example 3: Selecting Hierarchical Keywords If you select "J00-J06 Acute upper respiratory infections" from the ICD-10 catalog, all diseases/keywords in that section are included. If you select "J01.4 Acute pansinusi-tis" on the lowest level, only results matching that exact keyword will be shown.

Automatically included keywords are marked with a square, keywords you select yourself are marked with a checkmark.

• Keywords of the ICOTAGS catalog might appear crossed out. This means that the keyword is deprecated. You can still search for this keyword, but it may not be used for tagging anymore.

Note that if you run a search with a set keyword, the results will only include documents which have been tagged with this keyword on at least one tagging level. For more information, see Section 2.7.3.

Special Search Fields

Most search fields are straightforward to fill, however, certain search fields function in particular ways and/or require particular inputs.

• Body Part

Here, you can filter for the part of the body examined (contents of the DICOM tag BodyPartExami ned).

• Description

Description of an examination/study (contents of the DICOM tag StudyDescription). The wildcard % is automatically prepended and appended to the search term.

• Mother's Maiden Name

The mother's maiden name may contain special characters. Due to technical reasons, the search field does not support some of these special characters. If the name you are searching for contains one or more of the special characters described in Table 5, please use a wildcard at this position to conduct your search (see Section 2.4.3).

Name

This search field allows you to search for last name and/or first name and different combinations thereof. The following input patterns are valid and will be mapped correctly to the database fields **First Name** and **Last Name**:

- firstname lastname
- lastname, firstname
- lastname,
- , firstname
- lastname
- DICOM notation:

- lastname^firstname
- lastname^
- ^firstname
- lastname^firstname^^^
- lastname^^^^
- ^firstname^^^
- Any other combination of firstname and lastname with up to four carets that follows the standard DICOM notation (i.e. lastname always before firstname, always one caret before firstname).

Note

Spelling in the list above represents the way search terms have to be entered. If first name and last name are not separated by a comma or a caret, they may only consist of one word each because the space marks the separation between the two in this case. Names containing hyphens or other special characters are considered as one word, e.g. Smith-Miller or O'Brien.

• Patient Age/Study Age

These fields require a particular syntax. In addition to entering the desired number, you have to indicate the desired unit of time (y = years, w = weeks, d = days, h = hours). If no unit of time is indicated, years are presumed as unit. Units of time may not be combined. By adding > or <, >= or <= you can search for certain age ranges. Without a relational operator, the search will only include the precise age.

Example 4:

- Searching for patients younger than 20 weeks: <20w
- Searching for patients exactly 70 years of age: 70y
- Searching for studies of the past week: <=1w

Producer

The department (Organizational Unit) which produced the images/documents. Only departments of which you are a member are displayed.

• Study Date

This search field combines a selection list with a date search. First, select a time period from the selection list. You can search for time periods related to the present by selecting "last 4/8/12/24 hours", "yesterday", "today", "last 2/3/7/30 days". When you want to search for a specific date or time period, select "on", "from", "to" or "between". In these cases, one or more date fields will open after selecting the time period. Enter the desired dates manually or select them from a calendar.

2.4.5 Saving a Search Query

You can save frequently used **Patient Search** queries to avoid having to reenter them every time. Any customizations you apply to the results list (see Section 2.5.1.3) are saved alongside the search query and will be applied automatically when you execute the saved search query.

Note that saving search queries is not available for the Entity Search.

Example 5: Search Query: Display all CTs of a given day in the results list If you have a daily CT meeting and want to see all CTs of that particular day in the results list, you can create a CT_Meeting search query, save it, and execute it by clicking on its name.

To save a search query, proceed as follows:

- **1.** Prepare your search as described in Section 2.4.1. If you are using the classic search mode, you can customize the available search fields (see Section 2.4.4.2).
- **2.** Optionally, execute the search query and adjust the display of the results list to your liking. Changes to the look of the results list are saved automatically (see Section 2.5.1.3).
- **3.** Once you are satisfied, click on **Save as** to save your query. A new window opens where you can name the search query.
- 4. Your query will appear as a separate tab next to the Search tab.

To execute a saved search query, simply click on the corresponding tab.

2.4.5.1 Editing a Saved Search Query

You can rename or delete a saved search query by selecting it and clicking on the corresponding entry in **‡**.

To edit a saved search query, select it and apply the desired changes (e.g. enter additional search parameters, convert a search from classic to *Smart Search* or vice versa), then click on **Save** to overwrite the previous query. If you want to save the changes as a new search query instead, click on \checkmark > **Save** as and enter a new name. If you would rather not save the changes, click on \checkmark > **Discard changes** to revert them.

2.4.5.2 Setting a Search Query as Default

You can set a saved search query to be executed automatically as soon as you log in to *syngo*.share webview. Select the desired search query tab and click on \diamondsuit > Set as default. At your next login, *syngo*.share webview automatically switches to the patient search and executes the default search query.

2.5 Results List

After performing the search, the results are displayed in the results list. A result can be a DICOM study (examination) or a generic document (generic container). For each result, detailed information is displayed in a variety of columns (e.g. patient ID, name, date of birth, modality, producer, and many more).

Note

Depending on permissions and/or system settings, the results list might not show all documents or document parts matching your query. In case you are missing results, please contact your system administrator to check your permissions and/or settings.

You can customize the results list according to your needs. You can do the following:

- Filter your search results (see Section 2.5.1.1)
- Export your search results (see Section 2.5.1.2)

- Customize the displayed columns and layout (see Section 2.5.1.3)
- Set automatic updates for the results list (see Section 2.5.1.4)

The results list is your starting point for working with documents. You can do the following:

- Preview documents (see Section 2.5.2)
- View references (see Section 2.5.3)
- View other examinations (see Section 2.5.4)
- View the patient QR code (see Section 2.6)
- Perform numerous actions via the context menu (see Section 2.7)

Special Results List Columns

Most columns and their contents are self-explanatory; for certain columns, details are provided in the following:

• Availability

The availability column offers information on where documents are stored and how quickly they can be made available for use. Documents can either be stored in *syngo*.share core and synchronized third-party archives (VNAs) and long-term archives (LTAs), or in so-called remote nodes (i.e. thirdparty archives not synchronized with *syngo*.share core). Generally, the retrieval of data stored in remote nodes can take some time. Table 3 provides an overview of the indicators and their meaning:

Indicator	syngo.share core	Status Details	Remote Node	Status Details
•	Online	The document is immediately available.		
•	Nearline	The document is stored in a synchronized third-party VNA or LTA. Retrieval may take some time.	Remote online / Remote nearline	The document is available, but retrieval may take some time. / The document is stored on a relatively slow medium and retrieval will take some time.
•	Partially online	Parts of the document are online, other parts are stored in a synchronized third-party VNA or LTA. Retrieving the nearline parts may take some time.		
•			Offline / Unavailable	The document cannot be retrieved.
•			Unknown	The remote node does not provide availability information.

Table 3: Availability of Documents

• LTA State

The LTA state is displayed if a document has to be retrieved from the long-term archive. If the document is online or the retrieval was successful, there is no entry in the column.

2.5.1 Working with the Results List

2.5.1.1 Filtering Search Results

You can refine the displayed results by entering a string in the filter field.

2.5.1.2 Exporting Search Results

You can export the search results to a CSV file by clicking on the **CSV** (.) Note that if you use the CSV export with an active filter, only the filtered results will be exported.

You can also copy individual column entries to the clipboard for further use, e.g. for filling out a form or spreadsheet. Right-click on the desired entry and select **Copy to clipboard** from the context menu.

2.5.1.3 Customizing the Results List Layout

The results list offers two view modes, i.e. ways of displaying the results: the patient view (**i**) and the document view (**i**). To switch between the two view modes, click on the displayed icon and select either **i** Patient or **I** Document.

In both view modes, you can do the following:

- Move the columns freely by simply dragging them with the mouse
- Adjust the column width by clicking on their border and dragging it with the mouse
- Sort the columns by either
 - clicking on the arrow shown in the column header on mouse-over and selecting 12 or 12
 - clicking on the column header (quick-sorting)

Note

Some columns cannot be sorted because *syngo*.share pre-sorts the results in the back-ground.

• Show or hide the columns as desired. Click on the arrow shown on mouse-over in the column header and (de)select the desired columns in the **Columns** list

Your customizations are saved automatically. Note that for saved search queries, you have to manually save any changes you make via \Rightarrow > Save (see Section 2.4.5).

2.5.1.4 Setting Automatic Updates for the Results List

By default, the results list is static between searches. This means that changes made to the data in the background, such as a file being deleted from the archive, are not automatically shown in the list (note that changes that you make to the search results, such as setting a new document workflow state, are shown immediately). To update the results list manually, click on **Search**.

You can set an automatic update interval for the results list to ensure that the results are refreshed periodically. To set automatic updates, click on **()** and select the desired update interval.

The selected update interval is valid for saved search queries as well as for manual searches. If you have set a saved search query as default, the update interval starts with your login to *syngo*.share webview.

Note

The automatic update of the results list causes a higher system load. The shorter the update interval, the higher the load. To avoid an unnecessarily high system load, use a longer interval between updates, or, if unneeded, disable the automatic update function.

2.5.2 Previewing Documents

You can preview a document by selecting it in the results list. The preview area next to the results list displays the document contents in detail:

Series/Files

The **Series** tab displays the DICOM series or generic files contained in the document. You can switch between a list view and a thumbnail view by clicking on \equiv . In the list view, you can customize the columns just like in the results list. The **Series** tab offers its own context menu with various actions for the series level (see Section 2.7).

Grouping of OP and OPT Series

Referenced OP and OPT series are grouped together. The series information and the thumbnail of the OPT is not displayed. The reference lines of the linked OPT series, however, are included in the thumbnail of the OP series. For more information on working with ophthalmic images, see Reference Lines for Ophthalmic Images.

Instances/File

The **Instances** tab displays the instances of the selected series or the selected generic file. Just like in the **Series** tab, you can switch between a list view and an image preview.

- **Scrolling:** If the selected series contains multiple instances or an instance is multiframe, you can scroll through the instances/frames in the image preview.
- **Keywords:** DICOM instances that have been tagged with keywords are marked with a dot. Note that if a series contains a large number of instances, the markings may be grouped together. You can hover your mouse over the dot to view detailed tagging information. Moreover, you can jump to a tagged instance by clicking on the dot. For more information on keyword tagging, see Section 2.7.3.
- Key Image Notes: DICOM studies can contain Key Image Notes (KINs). KINs are a way to mark images as significant and provide additional information, e.g. for teaching purposes or consultation with other departments. KINs are DICOM instances of the modality KO.

Images referenced by a KIN are marked with a dot just like images with keywords. You can hover your mouse over the dot to receive more information about the referencing KIN. Moreover, you can jump to a marked instance by clicking on the dot.

• Actions: You can change the brightness and contrast of the preview image by clicking and dragging the mouse (see Section • Window). Click on to access various actions for the instance level (see Section 2.7).

2.5.3 References

The **References** tab in the preview area displays generic documents that are linked to the selected document, such as reports or discharge letters that have been associated with a DICOM study.

References are limited to documents from the same patient (based on the patient ID). Just like in the **Series** tab, you can switch between a list view and a thumbnail view (see Section 2.5.2).

References are ordered by their generic file type. References without an assigned generic file type are listed in a separate section at the bottom of the list. You can assign or change the generic file type in the document properties (see Section 2.7.2.6).

2.5.4 Other Examinations

In the **Other Examinations** area below the results list, you can view available other examinations for found DICOM studies. The **Other Examinations** area is collapsed by default; click on \checkmark to expand it. If any authority of the DICOM study selected in the results list is from another site, the other examinations list is deactivated.

To view other examinations, proceed as follows:

- 1. Expand the area and select a study from the results list to see the matching other examinations.
- **2.** Alternatively, you can use the results list context menu entry **Show other examinations** to expand the area and see the matching examinations.
- **3.** You can filter the other examinations by prior examinations (checked by default), modality, or body part.
- **4.** You can also add/remove, rearrange and sort the various columns just like in the results list (see Section 2.5.1.3).

Select an other examination to receive a preview of the contained series and instances. For guidance purposes, the current study remains highlighted in the results list.

Double-click on an other examination to open it in the Viewer. To display an other examination alongside the initial examination, choose **Compare with initial examination** in the context menu of the **Other Examinations** area. You can then view and edit the images contained in either examination (see Section 2.8).

Note that you can change the position of the initial examination in the settings (see Section 2.3.2.1). If multiple Viewer tabs are configured, the context menu entry **Compare with initial examination** will open a new tab for the comparison even if there already is an open tab for the same patient.

Note

Note that other examinations are from the same archives selected for the initial search. If you suspect other examinations in other archives, you have to first select these archives to be included in the search. See Section 2.4.1 for details.

2.6 Patient Information and QR Code

syngo.share webview provides quick access to the most important patient information such as name, birthdate and patient ID from a variety of locations, most notably the results list when set to the patient view (**‡**) and the viewer tab/patient banner (see Section 2.8.1).

Click on **i** Patient Information in the context menu (see Section 2.7) or on **i** where available.

The patient information screen also displays a QR code for the patient ID. You can scan this QR code with a mobile device and open the patient record with e.g. the Visual Capture application.

For viewing and editing patient details, see Section 2.7.2.6.

2.7 Context Menu and Actions

The context menu offers a variety of actions for documents. You can access it in the following locations:

Patient Search

Right-click on an entry in the search results list or in the **Other Examinations** area. To execute actions on individual DICOM series, instances or generic files, open the context menu in the **Series/-Files** or **Instances/File** area.

• Entity Search

Right-click on an entry in the search results list.

• Viewer

The following Viewer areas offer access to a limited context menu. For actions specific to these areas, please refer to the relevant chapters:

- Patient Banner: Click on §.
- Document Navigator: Right-click on an entry in the list.
- Patient Jacket: Right-click on an entry in the list.

Certain actions are also available in the specific toolbar (see Section 2.8.4).

Availability of Actions

The available actions depend on several factors, such as the document type, the producer (Organizational Unit) where the document is archived or the origin of an authority.

Most actions are available on any document level, i.e. on the DICOM study/series/instance level and on the generic container/file level. When you execute an action on a higher document level, the action affects all subordinate document levels.

The following sections provide information on the various context menu actions:

- Opening Documents
- Quality Assurance Operations
- Keyword Tagging
- Searching Externally
- Distributing Documents
- Patient-Related Actions

List of Favorites

Certain actions offer a list of favorites, such as frequently used recipients for document-sharing. These lists of favorites may be initially empty or contain some preconfigured entries, but you can adjust them according to your needs:

- To add an entry to your list of favorites, click on its star symbol (\bigstar).
- To remove an entry, click on its star symbol in the list of favorites (1).

Some actions will show your favorites as quick-selection items in the context menu.

2.7.1 Opening Documents

syngo.share webview offers several means of opening documents:

• Opening Documents in the Viewer

- Opening Documents in External Applications
- Opening Documents in Concentriq Dx®
- Retrieving Documents

2.7.1.1 Opening Documents in the Viewer

You can open any part of a document in the Viewer by either double-clicking on it or selecting **Open** from the results list context menu. Note that if you open a lower-level item, the corresponding higher-level item(s) will also be opened in the **Document Navigator** of the Viewer. For example, if you open a DICOM series in the Viewer, the corresponding study and any other series it contains will also be displayed in the **Document Navigator** (see Section 2.8.2).

Opening Documents in a Pop-up Window or on Another Display

You can open any part of a document in a pop-up window by selecting **Open in pop-up window** from the context menu. Depending on your browser, you can alternatively select a specific display on which the document is opened (provided you have multiple displays and you grant the browser permission to access the displays).

In the settings (see Section 2.3.1.3), you can select if a new pop-up window is opened for each document or if the already opened pop-up window should be used to display a newly opened document. You can also set a double-click to open a pop-up window instead of a viewer tab. Pop-up windows are closed automatically when you log out of *syngo*.share webview.

Opening Additional Documents

The Viewer tab stays open when you switch back to the search tab. Like this, you can easily open additional documents for the currently displayed patient. The documents are added to the **Document Navigator** (see Section 2.8.2). The already opened viewports remain in place. To view a newly opened document, either change the layout (see Section • Layout) to have more viewports available and open the document in an empty viewport, or drag the document into an active viewport to replace the displayed document.

Opening Multiple Documents at Once

You can open up to nine DICOM studies and/or generic containers of the same patient for viewing at once. The documents have to be in the same archive. Press **Ctrl** and click to select multiple documents, then select **Open** from the context menu to open all the selected documents in the Viewer. Each document will be opened in its own viewport in one Viewer tab.

Opening Documents from Multiple Patients

Depending on the configuration, you can open up to three Viewer tabs for different patients and/or entities. Adding more documents to already displayed patients/entities is possible without limitation.

Note

Please be aware that any quality assurance operations (see Section 2.7.2) performed on an opened document will only become visible once you close and reopen the document.

2.7.1.2 Opening Documents in External Applications

syngo.share webview allows you to open documents in external applications, e.g. another viewing application or a print application. These external applications are configured by your system administrator and are available in the context menu. For opening documents, one or more default applications can be set that all open simultaneously when you double-click on the desired study or series.

2.7.1.3 Opening Documents in Concentriq Dx®

You can open generic documents in Concentriq $Dx^{(B)}$, a platform for digital pathology. Select the desired document and click on **Open with Concentric Dx^{(B)}** in the context menu. You can also select and open multiple documents. A new browser tab will open for each document.

Note

This action is only available for documents which originate from Concentriq $Dx^{(B)}$. In order to use this feature, access to Concentriq $Dx^{(B)}$ is required.

2.7.1.4 Retrieving Documents

You can retrieve *syngo*.share documents that have been moved to a synchronized third-party or long-term archive as well as documents from remote nodes.

syngo.share documents (i.e. documents archived in syngo.share core) that have been moved to a synchronized third-party or long-term archive are given the status *Nearline*. Open the context menu and select **Retrieve** to return them to the online status. Once retrieved, these documents can be displayed in syngo.share webview and all actions are available as usual.

When you use **Retrieve** on documents stored in a remote node, the documents are imported into the archive (*syngo*.share core or a third-party archive). You can retrieve documents that have the status *Remote online* or *Remote nearline* from remote nodes.

For more information on availability, see Section • Availability .

2.7.2 Quality Assurance Operations

syngo.share webview offers a variety of functions to assure quality of the data archived. Table 4 provides an overview of the functions, states their availability for DICOM and generic data, and links to the corresponding sections.

Function	DICOM	Generic	Description
Change patient metadata ^a	~	~	Section 2.7.2.6
Change study/document metadata	✓	✓	Section 2.7.2.6
Change series metadata	~	×	Section 2.7.2.6
Change document workflow state	~	×	Section 2.7.2.7
Move series	~	×	Section 2.7.2.4, Section 2.7.2.5
Move study	✓	✓	Section 2.7.2.3, Section 2.7.2.4
Merge series	~	×	Section 2.7.2.5
Merge study	~	×	Section 2.7.2.5
Copy study	✓	✓	Section 2.7.2.2
Split study ^b	~	✓	Section 2.7.2.2, Section 2.7.2.1
Delete study/generic container	~	✓	Section 2.7.2.1
Delete series	~	×	Section 2.7.2.1
Delete instance/generic file	~	✓	Section 2.7.2.1
Delete report (generic document)	×	✓	Section 2.7.2.1
Rotate image (generic file)	×	~	Section 2.7.2.8
Change thin slice state	~	×	Section 2.7.2.9

Table 4: QA Operations in syngo.share webview

^a Patient data can only be changed for non-HL7-managed patients.

^b via Copy, Delete

Note

While quality assurance (QA) operations are being carried out on a document, the corresponding study or generic container is locked in order to prevent multiple users from executing QA operations at the same time. Users can still view the document but will be informed of the ongoing QA operations via warning.

2.7.2.1 Deleting Documents

This function allows you to delete documents or parts of documents from the archive. After clicking on **Delete**, you have to select a reason for deletion for quality assurance purposes.

2.7.2.2 Copying Documents

This function allows you to copy documents to a different producer (Organizational Unit) and make them available to other users. Copying means that the document will subsequently exist multiple times in the archive, once in each Organizational Unit. All keywords and annotations are also copied.

To copy a document, proceed as follows:

- 1. Select one or more documents. To select multiple documents at once, click while pressing Ctrl or Shift.
- **2.** Right-click to open the context menu and choose **Copy**.
- **3.** If you have favorite targets, you can select them directly from the context menu. To select another or multiple targets, click on **Other targets** to open a selection list of all available producers.
 - You can enter part of the target's name in the filter field to narrow your options.

- Targets that you have added to your list of favorites are listed separately.
- **4.** Click to select the desired target from the list. To select multiple targets at once, click while pressing **Ctrl** or **Shift**.
- **5.** Confirm your selection with **OK** to start the copying process. You can track the progress of copy operations in the **Job Queue** (see Section 2.12 for details).

2.7.2.3 Moving Documents

This function allows you to move documents to a different producer (Organizational Unit).

To move a document, proceed as follows:

- 1. Select one or more documents. To select multiple documents at once, click while pressing Ctrl or Shift.
- 2. Right-click to open the context menu and choose Move.
- **3.** If you have favorite targets, you can select them directly from the context menu. To select another target, click on **Other targets** to open a selection list of all available producers.
 - You can enter part of the target's name in the filter field to narrow your options.
 - Targets that you have added to your list of favorites are listed separately.
- **4.** Click to select the desired target from the list.
- 5. Confirm your selection with OK to move the documents. The results list is updated immediately.

2.7.2.4 Reassigning Documents

You can reassign documents to another patient or visit, as well as to another order. To reassign a document, select it in the results list and choose **Reassign** in the context menu.

Reassigning Documents to Another Patient or Visit

To reassign a document to another patient or visit, select **To another patient/visit**.

On the **Reassign** screen, search for either the exact visit ID or the patient. Note that you cannot search for a visit ID and patient simultaneously. Select the desired patient or visit in the results list and click on **OK**.

Reassigning Documents to Another Order

To reassign a DICOM study to a requested procedure or a DICOM series to a scheduled procedure step, select **To another order**.

On the **Reassign** screen, search for the correct order. Note that you cannot search for the order ID or requested procedure ID simultaneously. Select the desired order in the results list and click on **OK**.

Reassigning Documents to Another Entity

This action is only available in the **Entity Search**. To reassign a document to another entity, select **Reassign > Entity** from the context menu.

On the Reassign screen, search for the desired entity. Select it in the results list and click on OK.

2.7.2.5 Rearranging Studies and Series

syngo.share webview allows you to rearrange DICOM studies and series via a cut-and-paste operation. This function is useful for correcting acquisition errors. The following operations are available:

- Merge studies
- Merge series (of the same modality)
- Move series to another study

To rearrange studies or series, take the following steps:

- 1. Select the desired studies or series in the corresponding results list. Press **Ctrl** to select more than one study or series.
- 2. Open the context menu and choose on **Cut**. The selected studies or series are cut to the clipboard.
- 3. Search for the target study or series and select the desired target document.
- 4. Open the context menu on the desired target study or series and click Paste. A dialog box opens informing you about what exactly will be done and what will be the consequences of the desired operation. Confirm the operation by choosing whether you want to keep or delete the source document. The copied studies or series are pasted into the target study or series, and the source is deleted or kept depending on the previous selection.

Once the rearrange operation is completed, the results list is refreshed and the target document is selected. If you chose to delete the source document, it will no longer be listed.

As long as a document is part of a rearrange operation, it is locked and highlighted in color. You cannot cut and paste a locked document or merge it with another document.

The cut documents remain in the clipboard until

- they are pasted into another series or study. Upon completion of the operation, the clipboard is cleared.
- new documents are cut to the clipboard. In this case, the previously cut documents are overwritten.

2.7.2.6 Viewing and Editing Patient and Document Properties

You can access the properties of the patient and document on any level in the results list by clicking on **Properties of** in the context menu. Note that the context menu entry varies to match the document type and level.

The **Properties** window contains tabs for every available level of the document, including the patient. You can edit the desired properties directly. Click on **OK** to apply your changes or click on **Cancel** to discard them.

Note

The patient and document properties are also accessible from the specific toolbar in the Viewer (see Section 2.8.4).

Mother's Maiden Name

The field mother's maiden name is only available for Spanish-speaking countries. It allows several special characters, however, these have to be entered using the escape sequences described in Table 5:

Special Character	Escape Sequence
1	\E\
I	\F\
~	\R\
^	ISI
&	\T\

2.7.2.7 Setting a Document Workflow State

The document workflow state is used to reflect the status of a document as it moves through the steps of a workflow in your organization.

You can set the document workflow state by opening the context menu in the results list and selecting the desired step listed under **Document workflow state**.

2.7.2.8 Rotating Generic Images

In the image preview, you can rotate JPEG images clockwise and counterclockwise in steps of 90 degrees. This function is only available from the preview image, not from the list view. The rotation can be saved. Please note that only the rotation, no other operation on the image, is saved.

2.7.2.9 Setting the Thin Slice State

On the series level, you can (de)select a checkbox to specify whether a DICOM series is a Thin Slice Series . Thin Slice Series are specially highlighted in the thumbnail view.

2.7.2.10 Viewing and Correcting Warnings and Errors

syngo.share webview provides a document error and warning log. An error log is created when an error occurs during import. In case automatic corrective measures are taken during a re-import, a warning log is created.

To find documents with an error or warning, use the search filter **Document Log**.

Follow these steps to access the document log and review entries:

1. Right-click on a document in the results list to open the context menu and select **Show document** log.

- 2. A new window opens and lists all errors and warnings logged for the selected document.
- 3. Click on the desired entry to see the error/warning details.
- **4.** You can select entries and remove them from the document log by clicking on **Set as resolved**. This is useful for indicating that you have acknowledged a warning or when an error has been resolved with a re-import. You can skip this step if you wish to keep the log entries for traceability reasons.

Note

The actual correction of errors is handled by your system administrator.

2.7.3 Keyword Tagging

syngo.share webview lets you apply keywords to documents. Documents tagged with keywords can be searched for and grouped by their keywords.

Keywords can be based on catalogs (e.g. ICD 10) and/or on free text. The advantage of catalogs is that they contain exactly defined keywords which can be easily searched. The advantage of free text keywords is that any desired keyword can be applied to a document. However, the disadvantage is that it might become difficult to find entries when a keyword has multiple forms of expression, e.g. "femoral fracture" and "broken thigh bone".

• Catalog

This contains predefined keywords.

ICOTAGS

This is a special catalog that allows only one keyword to be set. This exclusive behavior is useful for keywords like "diagnosis-relevant".

• Free Text

These keywords are not restricted to a predefined catalog. Any desired keyword can be applied to a document.

You can assign any number of keywords from any level of the keyword trees to studies, series and images as well as general documents and generic files. If keywords were incorrectly assigned, they can be easily removed as well.

To assign keywords on the various levels, right-click on the element on the desired level and choose **Add keywords** from the context menu. You can also assign keywords in the document properties at every level (see Section 2.7.2.6) via the **Add keywords** button. Both ways open the form to assign keywords.

The keyword groups can be chosen from a selection menu. If a catalog is extendable, you can enter a new keyword directly in the text field. It is also possible that no keyword catalog has been assigned to a field. These are so-called free-text catalogs which allow keyword tagging via manual text entry.

If available, keywords from the ICOTAGS catalog are displayed directly in the context menu entry **Tags** for quick selection. ICOTAGS keywords may also be assigned from the specific toolbar in the Viewer (see Section 2.8.4).

Click on the arrow key next to the keyword search field to view the keywords of the respective keyword catalog. Navigate the tree to find the desired keyword or use the filter function to search for the keyword in the catalog. To actually apply the keyword tagging, select and click on the appropriate entries in the catalog. The keyword tagging is saved when you click **Save**.

To remove a keyword, open the properties of a study, series or instance (or a generic document or file) and remove the desired keywords by clicking on \mathbf{x} . Confirm your changes with **OK**.

2.7.4 Searching Externally

You can search for documents in other syngo.share applications or across healthcare institutions.

2.7.4.1 Searching in Other syngo.share Applications

Use **External search** to find patients or visits in *syngo*.share import or *syngo*.share view. To use this function, *icourl* must be installed.

2.7.4.2 Searching for External Documents

You can use **Search for external documents** to start a cross-enterprise search (XDS search) for documents for a given patient. This allows you to find documents from affiliated healthcare enterprises that have agreed to share documents with each other.

The found external documents are displayed in a pop-up window which offers many of the same customization options as the regular search results list (e.g. showing/hiding columns, sorting or filtering the results). You can open and edit the external documents in the Viewer. Note, however, that external documents are not added to the **Patient Jacket** (see Section 2.8.3) and cannot be archived.

2.7.5 Distributing Documents

syngo.share webview offers various means of distributing documents for further processing or use, such as sending documents to connected systems, sharing documents with colleagues or downloading documents to a local storage device:

- Sending Documents
- Exporting Documents
- Downloading Documents
- Registering Reference Pointers
- Registering Documents in the Patient Portal
- Sharing Documents

2.7.5.1 Sending Documents

You can send DICOM and generic documents to other systems or devices connected to *syngo*.share. These devices are shown as recipients (targets) and are configured by your system administrator. You can send DICOM studies, series and instances, as well as generic containers and generic files.

To send a document, proceed as follows:

- 1. Select one or more documents. To select multiple DICOM documents or multiple generic documents at once, click while pressing **Ctrl** or **Shift**.
- 2. Right-click to open the context menu and choose:
 - Send to send DICOM documents to a DICOM-capable application;
 - Send to filesystem to send generic documents to a connected filesystem;
 - Send to external system to send generic documents to an HL7-capable application.

- **3.** If you have favorite targets, you can select them directly from the context menu. To select another or multiple targets, click on **Other targets** to open a selection list of all available system-s/devices.
 - You can enter part of the target's name in the filter field to narrow your options.
 - Targets that you have added to your list of favorites are listed separately.
- **4.** Click to select the desired target from the list. To select multiple targets at once, click while pressing **Ctrl** or **Shift**.
- **5.** Confirm your selection with **Start sending**. You can track the progress of send operations (i.e. Send and Send to filesystem) in the **Job Queue** (see Section 2.12).

2.7.5.2 Exporting Documents

You can export archived documents to a local data storage device. The **Export** function opens a panel where you can choose between ZIP and ISO as the export file format. The options available for each format are described below. For both formats, you can select the minimization of DICOM patient data (see Section 2.7.5.2).

Exporting a ZIP file

When exporting the selected documents as ZIP file, you can choose to encrypt the file. In this case, you have to set a password. Furthermore, *syngo*.share webview can create subfolders for the individual DICOM series during export. According to your selection, these subfolders will be named after the contents of one of the following DICOM tags:

- SeriesInstanceUID
- SeriesDescription: If the SeriesDescription tag contains any of the special characters \/:*?"<>|, they will be replaced by _____ in the subfolder name. If SeriesDescription is empty, the subfolder will be named after SeriesInstanceUID. If two or more series have the same SeriesDescription, the duplicate subfolder names will be numbered.

Exporting an ISO file

When you choose to export the selected documents as an ISO file, a DICOMDIR is created and wrapped into an ISO file that you can burn to a CD/DVD via an external application.

You have the following options:

• Add nonconvertible DICOM files

With this option, you can include DICOM instances in your ISO export that cannot be converted to the designated transfer syntax *Explicit VR Little Endian*, such as videos. Doing so can result in the creation of a DICOMDIR that does not conform to the DICOM standard, meaning that you may not be able to fully view it in a third-party application. However, it allows you to export all DICOM instances regardless of their type.

Only ISO 9660 level 1

ISO 9660 is a file system standard for optical media like CDs and DVDs. Level 1 imposes severe restrictions on the file size as well as on the length of and the permitted characters in file and directory names.

With this option, you can ensure that the CD/DVD can be read by legacy operating systems and devices. Please be aware that selecting this option may lead to issues if the export includes one or more DICOM instances that exceed 4 GiB.

Exporting with Minimized Patient Data

You can minimize the following patient data during the export by either manually entering new values or by clicking on **Random**:

- Patient ID
- Last Name
- First Name
- Sex
- Date of Birth
- Birth Name
- Ideographic Last Name (if available in original data set)
- Ideographic First Name (if available in original data set)
- Phonetic Last Name (if available in original data set)
- Phonetic First Name (if available in original data set)

All other attributes belonging to the patient module (DICOM tag group (0010,XXXX)) are removed.

Please keep in mind that the minimization of patient data only applies to metadata of the patient module in the DICOM header and to the name of the export file. Patient data contained in documents, especially in images, PDFs and CDAs, cannot be altered. In case DICOM tags of the patient module are nested in DICOM elements other than the Modified Attributes Sequence (0400,0550) of the Original Attributes Sequence (0400,0561), the values of these tags are not minimized.

2.7.5.3 Downloading Documents

The **Download** function saves your selected documents as an unencrypted ZIP file to a local data storage device. It presents a one-click alternative to the **Export** function (see Section 2.7.5.2).

2.7.5.4 Registering Reference Pointers

You can manually register documents in order to make them available to subsystems. This means that for each selected document, a reference pointer containing all relevant information about the document is sent to the subsystem.

To register a reference pointer for a document, proceed as follows:

- 1. Select one or more documents. To select multiple documents at once, click while pressing Ctrl or Shift.
- 2. Right-click to open the context menu and choose Register reference pointer.
- **3.** If you have favorite targets, you can select them directly from the context menu. To select another or multiple targets, click on **Other targets** to open a selection list of all available subsystems.
 - You can enter part of the target's name in the filter field to narrow your options.
 - Targets that you have added to your list of favorites are listed separately.
- **4.** Click to select the desired target from the list. To select multiple targets at once, click while pressing **Ctrl** or **Shift**.
- 5. Confirm your selection with Start sending.

2.7.5.5 Registering Documents in the Patient Portal

This function registers documents to the eHealth Solutions Patient Portal. Via multi-selection (**Ctrl** or **Shift**), you can choose and register multiple documents or generic files at the same time.

2.7.5.6 Sharing Documents

This function allows you to share examinations with other users.

To share a document, proceed as follows:

- 1. Select one or more documents. To select multiple documents at once, click while pressing Ctrl or Shift.
- 2. Right-click to open the context menu and choose Share.
- 3. You can share documents with individual users and/or user groups. To select a recipient, you can
 - manually enter part of the recipient's name in the corresponding search fields and select the desired result from the drop-down menu, or
 - click on With users or With user groups to open a selection list with all available recipients.
 - Select the desired recipient from the list. To select multiple recipients at once, click while pressing **Ctrl** or **Shift**.
 - You can enter part of the recipient's name in the filter field to narrow your options.
 - Recipients that you have added to your list of favorites are listed separately.
- 4. Optionally, you can change the validity period for your share. The default validity period is set by your institution. To change the validity period, either manually enter a new date or select it from the calendar via **m**. To allow permanent access to your share, check the box **indefinitely**.
- 5. Optionally, you can check the box High priority to mark your share as important.
- **6.** Optionally, you can add a personal message that will be sent to all recipients. Take care not to include confidential patient information.
- **7.** Confirm with **OK** to share the documents. Every recipient will receive a notification in their **Inbox** (see Section 2.9) as well as an e-mail notification if they have provided an e-mail address.

2.7.6 Patient-Related Actions

The **Patient** category contains actions related to or based on the patient. You can do the following:

- View patient information (Section 2.6)
- Edit patient details (Section 2.7.2.6)
- Search for other patient IDs. This allows you to see if the same patient has been registered with different patient IDs in other Sites and add their documents to the results list. Sites are other healthcare facilities (e.g. individual hospitals of one hospital group) that are part of your *syngo*.share installation.
- Search for external documents based on the patient ID (see Section 2.7.4.2)
- Search for the patient ID in other syngo.share applications (Section 2.7.4.1)
- Set the patient as the context for archiving new documents (Section 2.11)

2.8 Viewing Documents

When you open a document in *syngo*.share webview, you are taken to the Viewer. The Viewer opens in a separate tab (per patient) that displays the patient context of the document (see Section 2.7.1). The Viewer is divided into the following areas:

- Patient Banner
- Document Navigator
- Patient Jacket
- Viewport Area

The Viewer is intended for viewing only and provides a variety of tools for this purpose. Editing steps are not saved.

The Viewer supports many documents and image formats in addition to DICOM images. For unsupported file formats, a notice is displayed.

Note

In case of problems when viewing a video, it may have been imported with a variable bitrate.

2.8.1 Patient Banner

The patient banner in the Viewer tab displays basic information on the patient whose documents you have opened. Click on **‡** to access the context menu, which offers the following options:

- Patient Information and QR Code
- Searching for External Documents
- Archiving Documents

2.8.2 Document Navigator

The **Document Navigator** displays basic information about the documents you have opened alongside a thumbnail image. You can collapse/expand the **Document Navigator** by clicking on > or <.

To collapse/expand the individual levels of a document, click on the document entry. Double-click on an entry to open the document in the viewport area. You can also drag and drop the document to a specific viewport (see Section • Layout).

When you open multiple documents of the same patient, they are color-coded for your convenience. The corresponding viewports in the viewport area are highlighted in the same color (see Section 2.8.4). You can switch off the color highlighting in the settings (see Section 2.3.4.3).

Document Navigator Context Menu

The **Document Navigator** offers its own context menu with a limited selection of actions which varies slightly depending on the type and level of the document. Most of the available actions are general actions that are described in Section 2.7; however, the following context menu entries are specific to the **Document Navigator**:

- **Clear viewer and hang document** is available on the document level (i.e. when you right-click on a study or generic container). This action clears the Viewer and hangs the selected document.
- Clear viewer and hang series is available on the series level. This action clears the Viewer and hangs the selected series.

Grouping of OP and OPT Series

Referenced OP and OPT series are grouped together. The series information and the thumbnail of the OPT is not displayed. The reference lines of the linked OPT series, however, are included in the thumbnail of the OP series. For more information on working with ophthalmic images, see Reference Lines for Ophthalmic Images.

2.8.3 Patient Jacket

The **Patient Jacket** provides a list of all documents which are associated with the current patient and are located in the same archive as the initially opened document. The *Jacket* is also available for entities and offers many of the same features.

You can expand/collapse the Patient Jacket by clicking on > or < or by pressing J.

To collapse/expand the individual levels of a document, click on the document entry. Double-click on an entry to open the document in the viewport area. You can also drag and drop the document to a specific viewport (see Section • Layout).

In addition, you can do the following:

- Sort the documents in ascending or descending order by clicking on 17
- Narrow down the number of listed items or search for particular items by manually entering a string in the search field
- Filter the documents
- View the references to a document by right-clicking on the document and selecting List references from the context menu
- View the prior examinations associated with a document by right-clicking on the document and selecting **List prior examinations** from the context menu
- List documents from all connected archives by clicking on SQ Search in all sources

Grouping of OP and OPT Series

Referenced OP and OPT series are grouped together. The series information and the thumbnail of the OPT is not displayed. The reference lines of the linked OPT series, however, are included in the thumbnail of the OP series. For more information on working with ophthalmic images, see Reference Lines for Ophthalmic Images.

Filtering the Patient Jacket

You can apply filters to narrow down the number of listed items. To apply a filter, click on \mathbf{T} and select one or more of the available filter criteria. The filter will remain in effect until you clear it by clicking on \mathbf{x} . You can also remove the active filter via \mathbf{T} > Clear filter menu.

You can save filters for reuse:

• Saving a filter

To save a filter, select the desired filter criteria and click on **Save as**. You can then enter a name for your new filter. Click on **OK** to save your filter.

• Applying a filter

To apply a filter, click on \bullet next to \mathbf{T} and select the desired filter.

• Editing a filter

To edit a filter, apply it first. Then click on **T**, make the desired changes to the filter criteria and click on **Save**. If you want to save your changes as a new filter instead, click on **\neg** > **Save** as and enter a new name. To undo all unsaved changes, click on **\neg** > **Undo changes**.

• Renaming a filter

To rename a filter, apply it first. Then click on $\mathbf{T} > \mathbf{x} > \mathbf{Z}$ Rename and enter a new name.

• Deleting a filter

To delete a filter, apply it first. Then click on $\mathbf{T} > \mathbf{x} > \mathbf{T}$ Delete.

• Setting a default filter

You can set a filter as the default, meaning it will be applied automatically the first time you open the **Patient Jacket**. To set a filter as default, apply it first. Then click on $\Upsilon > \diamondsuit > \bigstar$ Set as default. To remove the default filter, either deselect it in the filter settings or set a different filter as default.

Note If your administrator has set filters to read-only, you can only apply predefined filters.

2.8.4 Viewport Area

The central viewport area can display one or more documents in individual viewports.

Navigation

The Viewer offers several helpful features for quick and easy orientation and navigation purposes:

• Viewport coloring

All viewports showing series of the same study or files of the same generic container are highlighted in the same color. The color highlighting matches the entry in the **Document Navigator** (see Section 2.8.2).

• Viewport resizing

When using a multi-viewport layout, you can maximize a viewport to focus on its displayed image. Click on \mathbb{X} in the specific toolbar or simply press F to maximize the selected viewport. To return to the multi-viewport layout, click on \mathbb{X} or press F again.

Keyword-based navigation

DICOM instances that have been tagged with keywords are marked with a dot. Note that if a series contains a large number of instances, the markings may be grouped together. You can hover your mouse over the dot to view detailed tagging information. Moreover, you can jump to a tagged instance by clicking on the dot. For more information on keyword tagging, see Section 2.7.3.

• KIN-based navigation

DICOM studies can contain Key Image Notes (KINs). KINs are a way to mark images as significant and provide additional information, e.g. for teaching purposes or consultation with other departments. KINs are DICOM instances of the modality KO.

Images referenced by a KIN are marked with a dot just like images with keywords. You can hover your mouse over the dot to receive more information about the referencing KIN. Moreover, you can jump to a marked instance by clicking on the dot.

You can also open KINs in the Viewer and jump directly to the referenced images, provided that the images are part of the same DICOM study as the referencing KIN.

Toolbars

The viewport area contains toolbars which offer a variety of tools for viewing purposes:

• Global toolbar

The global toolbar is located at the top of the viewport area and offers general tools as well as tools that affect all viewports, e.g. viewing tools.

• Specific toolbar

The specific toolbar is located at the top of each viewport and offers special tools depending on the kind of document. It also offers access to certain actions (see Section 2.7).

• Playback toolbar

The playback toolbar is only visible when you are viewing videos or DICOM series of certain modalities. It is located at the bottom of the relevant viewports and offers tools for multimedia playback.

Depending on the selected Viewer layout and the size of your browser window, the arrangement of tools in the toolbars may change slightly: some tools may be grouped together or subsumed in a menu (=).

For quick access to certain tools, you can map them to mouse buttons: Open your user menu, select **Settings > Tools** and configure the buttons to your liking (see Section 2.3.3).

syngo.share webview offers the following tools:

- General Tools
- Navigation Tools
- Viewing Tools
- Display Tools
- Annotation and Calculation Tools
- Capture Tool
- Special DICOM Data Tools
- Playback Tools

2.8.4.1 General Tools

• Clear viewer

Use this tool to clear all the viewports. The Viewer tab remains open, you still have access to the Section 2.8.2 and the Section 2.8.3.

Pointer

You can use the pointer to move and edit drawings and annotations.

• Reset

Use the reset tool to reset all editing steps. Note that this function does not affect manual calibration, see Section 2.8.4.5.

2.8.4.2 Navigation Tools

• Scrolling

Use the scrolling tool and move the mouse up or down to scroll backwards and forwards through series and multiframe images. Alternatively, you can scroll using the mouse wheel.

Reference line

Use the reference line tool as a navigation aid when scrolling through multiple overlapping slice image series. The reference line shows the position of the current slice in the other series. Series from certain modalities can contain a primary reference image called a localizer image (see Section 2.8.4.7).
Note that you can only use the reference line tool with series that have certain anatomical positioning characteristics, e.g. orientation.

• Reference point

Use the reference point tool to identify locations in spatially related series. You can additionally activate the reference line tool for further navigation aid.

• Synchronized Scrolling

Use synchronized scrolling to automatically or manually synchronize multiple series while scrolling. This means that all synchronized series will show the nearest possible anatomic position. This is particularly useful when viewing two related series, e.g. one recorded with and one without a contrast agent.

Note

You can only use synchronized scrolling when the images of the different series contain information on their spatial positioning and their image planes do not exceed a certain angle. Please check carefully to what extent the slice images share the same anatomical positioning characteristics, as *syngo*.share webview does not provide feedback in this regard. To avoid errors, ensure that the starting images possess the same anatomical positioning characteristics.

Click on \mathscr{O} in the specific toolbar or press **S** to activate synchronized scrolling for individual series, or click on \mathscr{O} in the global toolbar to activate synchronized scrolling for all displayed series which contain the necessary spatial positioning information.

The following factors determine whether *S* activates automatic or manual synchronized scrolling:

Automatic synchronized scrolling

Spatially related series are synchronized automatically. The synchronized series are automatically scrolled to the current image slice position of the active series.

If the series are not spatially related, they are still automatically synchronized as long as you activate synchronized scrolling on the very first image in each series. In this case, the synchronization is based on the anatomically topmost image (starting image). If this does not produce the desired result, you can make manual adjustments (see below).

Manual synchronized scrolling

If the series are not spatially related and you do not use the topmost image as the starting image for scrolling, you can make manual corrections to the synchronized scrolling. To do so, select the most anatomically similar images across all the series you want to synchronize. By making these similar images your starting images, you can use manual synchronized scrolling.

The move tool (see Section • Move) and all scaling tools except the magnifier (see Section • Magnifier) are applied consistently to all synchronized series.

• Filtered scrolling

Use filtered scrolling to scroll only between instances that have been tagged with keywords and/or are referenced by Key Image Notes (KINs). You can also combine filtered scrolling with synchronized scrolling. In this case, the scrolling behavior depends on the tagged instances in the active viewport.

2.8.4.3 Viewing Tools

Layout

Use the layout buttons to display several documents side by side in various arrangements. You can also drag the viewports to rearrange the order of the documents.

• Zoom

Use the zoom tool to scale the image smoothly by holding the mouse button and moving the mouse. Further scaling options are available in the specific toolbar.

• Magnifier

Use the magnifier to enlarge a portion of an image. Click on the desired area in a image to enlarge said area in a separate window. Scroll the mouse wheel to control the scale factor. You can increase the scale factor to a maximum of 10 and reduce it to a minimum of 1. If you click and drag the mouse, the magnifier window will follow your movement while maintaining the set scale factor. Click the mouse button again to close the window.

Annotations are not considered when using the magnifier, however, any overlays that have been saved with the image will be shown.

Move

Use the move tool to move the image in the desired direction while pressing the mouse button.

2.8.4.4 Display Tools

• Window

Use the window tool to control level and width in DICOM instances or the brightness and contrast in generic images. For DICOM instances, the window tool changes slightly depending on the algorithm used in the background:

- Standard: For standard windowing, you control the level and width separately. Move the mouse up or down while holding the mouse button to adjust the level. To adjust the width, move the mouse left or right.
- Percentage-based: For percentage-based windowing, you control the level and width simultaneously. Move the mouse in any direction while holding the mouse button to adjust level and width. To adjust the upper/lower thresholds of the window to specific values, you can use **Advanced windowing** (see Section • **Enhanced visualization**).

• VOI LUT Presentation

A VOI (Value of Interests) LUT (Lookup Table) is a type of nonlinear windowing. If you open DICOM images with one or more VOI LUTs, you can use **VOI** in the specific toolbar to switch between the available VOI LUTs (e.g. High+Contrast) or to display the DICOM images without VOI LUTs.

• Presets

Under **Presets**, you can select and apply presets for the optimized windowing of certain body parts (e.g. cranium, lungs) to DICOM grayscale images.

Presentation States

When you open a DICOM study with Presentation States (Grayscale Presentation States), the most recently created Presentation State is automatically applied. Use the **PR** menu to select the desired Presentation State or deactivate an active Presentation State.

• Transform

Use transformation tools to rotate the image clockwise or counterclockwise in 90-degree steps. You can also flip the image vertically and horizontally.

Invert

Use the inversion tool to display a negative of an image, i.e. white parts of the image are displayed in black, black parts in white.

• Color Adjustment

For all color images (DICOM and generic color images, DICOM Whole Slide Images) a color adjustment tool is available. You can adjust the following values with a slider or by entering a numerical value:

- Gamma
- Brightness
- Contrast
- RGB saturation (split up into Red, Green and Blue)

If brightness and contrast have already been adjusted with the windowing tool (see Section • Window), the changed values are applied in the color adjustment tool. To restore the default values, click on **Reset**.

For the display of Whole Slide Images (see Section 2.8.4.7), you can save the settings for your user account. When you open another whole slide image, the saved color settings are automatically applied. To restore the default values, click on **Reset** and save them explicitly.

• Enhanced visualization

The **Enhanced visualization** category offers tools for highlighting visual details in DICOM instances:

- **Pseudo colors:** With the **Pseudo colors** tool, you can apply various color palettes to DICOM grayscale images. Click on the tool and select the desired palette. To return the image to grayscale display, select the first palette.
- Advanced windowing: With the Advanced windowing tool, you can define an upper and lower threshold to improve the visibility of particular details in the image. Simply drag the sliders into position in the gradient bar to achieve the desired result.

• PNG

The current quality value is shown in a text overlay for each image and may change when you perform continuous actions such as zooming, scrolling or windowing (depending on your settings). Click on **PNG** to view the current image in a lossless format. Note that if you perform further actions on the image, the display quality will revert to the configured values. You can adjust the display quality for images in the settings (see Section 2.3.4.1).

2.8.4.5 Annotation and Calculation Tools

You can use the following tools to add graphical and textual annotations to images. Many of these tools also allow you to perform length, angle, density or area calculations.

Note

Annotations and calculations are not stored within the image. They are preserved until you either click on **Reset** or open a new series or file in the same viewport. The unit used for calculations depends on the tool and the type of image. Please read the Safety Notices.

Adding Annotations

You can freely add and edit annotations while the series or file is open in the Viewer:

• Add

To add an annotation, select the desired tool in the global toolbar. Click to set your starting point and drag the mouse until the annotation has the desired size and shape. Click again to finalize the annotation.

• Change size and shape

Hover your mouse over the annotation to see its corner points. To change its size and shape, drag any corner point in the desired direction.

Move

To change the position of an annotation, select it and make sure none of the corner points is highlighted. Drag the annotation to the desired position.

Remove

To remove an annotation, select it and press **Del**.

Calibrating Calculations

You can manually calibrate the scale used for calculations on DICOM and non-DICOM images. Note that DICOM images often already come with calibration information that is part of the DICOM image or is provided through a DICOM display preset called a DICOM Presentation State .

To perform a manual calibration, proceed as follows:

- 1. Select Calibrate in the specific toolbar.
- 2. Click in the image to draw a calibration line.
- 3. Specify the length of the calibration line.
- **4.** The manual calibration is then applied to all new and previously made annotations within the image. For DICOM multiframe images, the calibration is also applied to all subimages.
- 5. To reset the calibration, select **Reset calibration** from the calibration tool menu.

Available Tools and Their Functions

The following annotation and calculation tools are available:

• Distance

Use this tool to calculate distances/lengths. If necessary, you can manually calibrate images to adjust the scale, see Section 2.8.4.5.

• Angle

Use this tool to calculate an angle or cobb angle. Click on the image to draw the two sides of the angle (two clicks per side). Three angle sizes can be calculated:

- Alpha: The acute angle between the two sides (< 90°).
- Beta: The obtuse angle between the two sides (180° alpha).
- Gamma: The reflex angle between the two sides (360° alpha).

When drawing the angle, the current default angle is calculated. Hovering over the other angles shows each angle as dashed helper lines. Click to calculate another angle. The selected angle (alpha, beta or gamma) is then stored as the new default angle.

• Pixel lens

Use this tool to calculate modality-dependent pixel values in DICOM grayscale images and spot density in CT images (e.g. bone density). Move the mouse over the image to view the pixel value or the density value of any given spot. Click to set the displayed pixel value/density for points of interest. This allows you to compare the values of different locations.

• Ellipse/Circle

Use this tool to draw an ellipse on the image and calculate its area. For CT images, the tool additionally calculates the average density. To draw a perfect circle, press and hold **Shift**.

• Rectangle/Square

Use this tool to draw a rectangle and calculate its area. For CT images, the tool additionally calculates the average density. To draw a square, press and hold **Shift**.

Polygon

Use this tool to draw a polygon and calculate its area. Note that you can only calculate the area if the polygon has no intersecting lines. Click to set each corner of the polygon. To finish the polygon, either double-click or press **Esc**.

• Freehand

Use this tool to draw any desired shape on the image. Hold the mouse button for the whole drawing process.

• Text

Use this tool to add textual descriptions to the image. You can change the size and color of the text. If you select **With arrow** when selecting the tool, you can draw an arrow together with the text annotation. In this case, you have to draw the arrow before you can enter the text.

• Arrow

Use this tool to draw an arrow on the image.

2.8.4.6 Capture Tool

With the **Capture** tool, you can photograph the image in the currently selected viewport, including all shown information overlays, DICOM overlays and annotations.

To capture an image, select the desired viewport and click on **O**. You can then do the following:

- Select whether you want to burn the text information and/or the annotations into the captured image.
- Edit or delete the default series or file description and save the captured image as a new series (modality: OT) or file to the current study or container. (Only images captured of documents archived in *syngo*.share core can be saved.)
- Drag and drop the captured image into external applications (e.g. a presentation).
- Download the captured image to your local filesystem in PNG format.
- Copy the captured image to the clipboard (availability depending on browser).

To quickly capture an image and copy it to the clipboard, select the viewport and press Crtl + C (availability depending on browser).

2.8.4.7 Special DICOM Data Tools

- DICOM Metadata View
- ECG Tools
- Tools for Whole Slide Images for Digital Pathology
- Reference Lines for Ophthalmic Images

DICOM Metadata View

With the **Metadata** tool, you can view, search and export the metadata, i.e. the DICOM header, of any given DICOM instance.

- Search: In the metadata view, the DICOM metadata is fully searchable. Enter a string in the search field to highlight any hits. You can use < and > or the keyboard shortcuts F3 and Shift + F3 to jump back and forth between hits. To clear the search field, simply click on x.

• Export: You can export the metadata by clicking on Export as CSV.

ECG Tools

For viewing DICOM ECGs, some additional tools are available:

• Fit to window

Use this tool to fit the ECG image to the window size.

• Caliper

Use this tool to add visual interval guidelines. Select the start and end point of the interval with the mouse. The selected interval will be automatically repeated across the spectrum.

• Paper speed calibration

Use this tool to select the paper feed speed in mm/s in the drop-down menu.

• Voltage calibration

Use this tool to select the voltage calibration in mm/mV in the drop-down menu.

• Channel layout

Use this tool to select the display arrangement for the available channels. If the active channel group consists of exactly twelve channels, there are several channel layouts available for viewing the ECG.

Channel group

Use this tool to select the channel group in the drop-down menu.

• ECG metadata

Use this tool to show or hide the ECG metadata block above the ECG.

Tools for Whole Slide Images for Digital Pathology

syngo.share webview supports the display of DICOM brightfield microscopy images for digital pathology .

Note

Changing the zoom layer or focal plane will cause your annotations, drawings and calculations to be discarded.

The following functions are available specifically for the display of digital pathology images:

• Zoom Layer

Select the zoom layer in the drop-down menu. Alternatively, you can scroll through the zoom layers using the mouse wheel or use the zoom tool in the toolbar (see Section • Zoom). Zooming is always centered on the current position of the mouse pointer.

• Focal Plane

If focal planes are available, you can select them in the drop-down menu. Alternatively, you can scroll through the focal planes using the mouse wheel while holding **Shift**.

• Thumbnail

If available, the thumbnail shows the slide overview. When the image is displayed in a zoom layer, the thumbnail indicates the location of the zoomed area. You can navigate to any area of the image by clicking on it in the thumbnail.

• Grid

Use the grid tool to display a rectangular grid for guidance. You can customize the grid as follows:

- Move the grid: Click and drag a horizontal or vertical grid line in the desired direction to reposition the grid.
- Adjust the grid spacing: Click and drag a grid corner in the desired direction to quickly resize the grid. You can also adjust the spacing in the toolbar by clicking on and entering a value.
- Reset the grid spacing: To return to the default grid spacing, click on ▼ > Reset.
- Save the grid spacing: To save your preferred grid spacing, click on ▼ > Save. Be aware that doing so overwrites the default value. If you wish to return to the previous grid spacing, you will have to manually enter the previous value and save it.

Note that the grid will remain in the same position relative to the image. This means that if you use certain tools, e.g. zoom, the grid may end up outside the visible area. To keep it in frame, you will have to select a smaller grid spacing.

Reference Lines for Ophthalmic Images

syngo.share webview supports the viewing of ophthalmic images, i.e. ophthalmic tomography (OPT) and ophthalmic photography (OP).

When you open an ophthalmic series that references an OP or OPT image, the two instances will either be opened side by side or one below the other in a split viewport. You can close either part of the split viewport via **x** and reopen it via **OP|OPT** in the specific toolbar.

The reference line tool (see Section • **Reference line**) is automatically activated for the OP image and highlights the displayed slice of the linked OPT. The OP image serves as the localizer image.

The appearance of the reference lines in the localizer image varies depending on the orientation of the ophthalmic images. If you open a multiframe series, all corresponding reference lines are displayed on the localizer image and may overlap.

You can switch reference lines off and on in the global toolbar.

2.8.4.8 Playback Tools

syngo.share webview offers different types of playback in the Viewer. You can set a standard FPS rate and enable/disable autoplay in the settings (see Section 2.3.4.2).

• Generic video playback

You can play back generic videos. The video toolbar offers basic playback and volume control.

• DICOM video playback

DICOM video playback offers the same toolbar as generic video playback. If a DICOM series contains multiple video instances, you can use **Instances** in the viewport menu to switch between them.

• DICOM series playback

You can also play back DICOM series of certain modalities. The series toolbar offers different playback modes (e.g. looping) and settings for manual browsing (e.g. image step size).

- Playback range: You can limit playback to a specific range of images by clicking on [] to set a beginning and an end point. You can adjust the playback range by dragging the markings to the desired position.
- Playback speed: You can adjust the frame rate (FPS value) to slow down or speed up series playback.
- Multiframe playback: You can switch between the individual instances of a multiframe series by clicking on **▲** and **▶** in the toolbar or by selecting the instance via **Instances** in the viewport menu.

You may need to grant browser-specific permissions to use autoplay in your web browser of choice. In addition, note that playback speed may depend on the available hardware as well as the size and number of opened documents.

2.9 Inbox

The **Inbox** tab contains all documents that have been shared with you, such as doctor's letters, findings or medical images.

Click on a share to display the shared documents as well as any included message. For requesting documents, see Section 2.9.2.

If a share notification contains more than one document, you can sort them by clicking on the available columns. Select a document to receive a preview of the DICOM series and images or generic files contained therein.

By double-clicking on a shared document, you can open it directly in the Viewer for viewing and editing purposes. You can also open select elements by double-clicking on an entry in the preview area.

Items in the inbox are actually links to items in the archive which have been shared with you. This means that removing a share from the inbox does not remove the document from the archive.

2.9.1 Actions

Refresh

If configured by your system administrator, your inbox will periodically refresh in order to display the newest shares. If your system administrator has not set an automatic refresh, you have to refresh your inbox manually in order to see any new shares. Click on \mathbf{C} to manually refresh your inbox.

• Filter

Filters the contents of your inbox when you enter a term in the filter field. Doing so searches for the term in every share and shows all matches. Entries which do not fulfill the search criteria are not shown.

• Sort

See Section 2.5.1.3.

Mark as read/unread

Changes the message status to "read" or "unread".

Remove share

Removes the selected share from the inbox.

Download

Quick export of a selection compressed as a ZIP file. This function is the equivalent of the **Export** function for the original, non-encrypted data, see Section 2.7.5.3.

• Export

Exports a selection from the archive using various options, see Section 2.7.5.2.

2.9.2 Requesting Documents

Your health care institution determines the procedure for requesting documents, although this is usually done by phone.

If configured by your system administrator, you can find more information on the procedure by clicking on **Request document** in the user menu.

2.9.3 Availability Period

Your health care institution determines the availability period of shares. The time limit is displayed in the header of the individual share (**Valid until**). For shares that are available indefinitely, this field is empty. The shared documents are available in your inbox until the end of the set period.

After the end of the availability period, the share is removed from your inbox. If you were not able to view the shared documents in time, you can request them again.

Note

If a document is deleted from the archive, it will no longer appear in your inbox.

2.10 Printing Documents

You can print images and textual documents (e.g. PDFs, Structured Reports) in the search tabs or in the Viewer. Please read the Safety Notices concerning the printing of images.

2.10.1 Printing in the Search Tab

To print a document in the **Patient Search** or **Entity Search** tab, go to the **Instances** area, select the **Preview** view. If there are multiple instances available, you have to scroll to the instance (i.e. the image or text document) you want to print first. Then click on **S** > **Print** to open your browser's print function.

Note that any adjustments you have made to an image (e.g. contrast) are printed as well.

2.10.2 Printing in the Viewer

To print a document in the Viewer, open the desired file or series. If there are multiple instances available, scroll to the desired instance in the viewport (see Section 2.8.4).

- Images: Click on Export > Print in the specific toolbar to open your browser's print function. Note that any adjustments you have made to the image (e.g. contrast, annotations) are printed as well.
- **PDFs:** Click on 🖶 in the PDF toolbar.

2.11 Archiving Documents

Note

Archiving documents is only supported for *syngo*.share core and is not available for external archives.

In the **1** Archive tab, you can archive documents to a patient, visit or entity. To archive a document, follow these steps:

1. Set the context (i.e. the patient, visit or entity to which the documents will be archived). You can either quick-select or search for an existing context, or create a new context. You can reset the selected context and choose a different one by clicking on **Change context**.

Quick-Selecting a Context

You can quick-select an existing context from any of the following locations:

- Patient Search Results List
 Switch the results list to the patient view () and select Archive to patient in the context
 menu.
- Entity Search Results List Select Archive to entity in the context menu.
- Patient Banner
 Click on : > Archive to patient.

In each case, you will then be taken to the **Archive** tab with the context already set.

Searching for a Context

In the Archive tab, you can search for an existing context in one of two ways:

Quick search

You can use the quick search field to search for the visit ID and/or patient ID (depending on the settings). Enter the ID and confirm with **Enter** to search for it and set the context.

• Detailed search

You can start a more detailed search by clicking on **Choose** and selecting the **Search** tab. Choose the appropriate tab to search for either a patient or an entity (in the case of entities, you have to additionally select an entity type from the drop-down menu). Enter your search criteria and click on **Search**. Select the desired search result and confirm with **Ok** to set the context.

Creating a Context

You can create a new context by clicking on **Choose** and selecting the **Create** tab. Choose the appropriate tab to create either a patient or an entity (in the case of entities, you have to additionally select an entity type from the drop-down menu). Fill out the required fields and confirm with **Ok** to create and set the new context.

- **2.** Add the documents you want to archive by either clicking on **Add** or by dragging and dropping the documents into the upload area.
- 3. Enter any relevant document metadata in the **Options** area. The following fields are available:
 - Producer (required field): This is the Organizational Unit to which the documents will be archived.
 - Accession Number
 - Body Part (DICOM element BodyPartExamined)
 - Description
 - Study date and time: These values can only be edited for generic documents.
- **4.** Optionally, you can add keywords to the document in the **Keywords** area. The available keywords depend on the selected Producer. See Section 2.7.3 for more information.

5. Click on **Archive** to start the archiving process. You can check its status in the **Job Queue** tab (see Section 2.12). Note that while archiving is in progress, you may only have access to part of the data in question and you may not be able to execute certain actions until archiving is complete.

Recommendations for Archiving Videos

To ensure correct video playback, only archive videos with a constant bitrate. Videos with a variable bitrate may not be played back properly.

2.12 Job Queue

In the **III** Job Queue tab, you can view the status of potentially time-intensive jobs such as the sending or restoration of documents. If an error occurs, the tab as well as the affected job are marked with a badge. You can view error details by clicking on **Info**. The tab also shows if there are currently jobs running in the background.

The following actions are available in the Job Queue:

Cancel

You can cancel certain jobs.

Retry

You can retry canceled or failed jobs. This option is available for all jobs except the upload of documents. Failed uploads have to be reinitiated manually.

The job status list is maintained between sessions and across multiple tabs. So-called "hanging" jobs, i.e. jobs which do not log any progress for more than one day, are automatically assigned the status "Failed". After one additional day, they are automatically cleared from the queue together with any other failed or canceled jobs.

To manually clear the queue, click on the corresponding button.

2.13 Keyboard Shortcuts in *syngo*.share webview

Patient/Entity Search		Display Tools				
Del General Viewer Comma	Delete a selected document/document parts from the archive	W Shift + W Shift + 1-9 O	Level/Width, Brightness/Contrast No windowing preset Apply windowing preset Color Adjustment			
l	Show/hide Patient Jacket	Annotation Toolo				
Q	Quit viewer (return to search)					
R	Reset selected tool (Unde most recent appointation	Δ.	Angle			
F	Properties	C C	Filinse			
K	Add keywords	D	Pixel lens			
F	Maximize/downsize viewnort	G	Polygon			
		Ĺ	Length/Distance			
		Del	Delete selected annotation			
Viewer Layouts		_				
1	Apply 1x1 layout	Playback				
2	Apply 1x2 layout					
3	Apply 2x1 layout	Spacebar	Start/pause video/series playback			
4	Apply 1x3 layout	→	Next video frame			
5	Apply 2x2 layout	←	Previous video frame			
6	Apply 3x3 layout					
		Navigation				
Viewing Tools						
		- 1	Next instance/frame			
Μ	Magnifier	Ť	Previous instance/frame			
Р	Move	\rightarrow	Next series/file/instance			
S	(De-)Activate synchronized scrolling in active viewport		Previous series/file/instance			
Z	Zoom	Page↓	Next set of images (layout view)			
Shift + Mousewheel	Whole Slide Images: Switch between focal planes	rage I	Frevious set of images (layout view)			
Ctrl + C	Copy captured image to clipboard	nome End	First set of images (depending on layout)			
		Ena	Last set of images (depending on layout)			
			Set selected viewport at first position and reset hanging			
			Provious viewport			
		Ciu + 🕶	FIEVIOUS VIEWPOIL			

3 Emergency Access

syngo.share webview has an emergency access mode in case e.g. the hospital information system (HIS) of your organization becomes temporarily unavailable. This allows you to carry out routine tasks in *syngo*.share webview without interruption (e.g. searching for patients).

A note on the login screen will inform you when your system administrator has activated the emergency access mode.

You can log in with your user name and default password (if you have an LDAP-administered account) or with a specific emergency access account established by your organization. For more information, click on the link on the login screen or contact your system administrator.

After entering your login information, click on Login or press Enter as usual.

In emergency access mode, the search screen as well as the results list are pre-configured, although you can still adjust them according to your needs. Usage does not differ from the standard mode (see Section 2). However, some actions and/or functions may be deactivated.

4 Mobile Devices

syngo.share webview is optimized for use on the following mobile devices supporting iPadOS 16:

- iPad (≥ 2020)
- iPad Air (≥ 2019)
- iPad Pro (≥ 2017)

You must enable the mobile view of the browser. The use of touch input is recommended. It is further recommended to launch *syngo*.share webview via home screen shortcut ("website icon"). This will open the browser in full-screen mode, which offers the best user experience of *syngo*.share webview. (How to set a shortcut on the home screen is described in the online iPad user guide.)

Although the mobile version of *syngo*.share webview is largely identical to the desktop version, there are a few notable differences:

- Download, print and export are not available on mobile devices.
- CSV export of search results is not available on mobile devices.
- Opening a selected document in a new tab from the properties window is not available on mobile devices.
- Opening documents in external applications is not available on mobile devices.
- Autoplay of videos is not available on mobile devices.
- The volume control for videos is not available on mobile devices. Please use the volume buttons of the mobile device.
- The full-screen button is not available on mobile devices. Please use the home screen shortcut described above to open the browser in full-screen mode.
- Certain functionalities are altered slightly in order to accommodate the screen size and input methods of mobile devices. These differences are explained in the following.

4.1 Functionality

• Displaying document details

The series/preview area of the search results list is initially hidden. Select a document from the results list to view its details.

• Selecting multiple entries

In order to select multiple entries in the search results list or the series/preview area, tick the check boxes next to the desired entries. In the series/preview area, you first have to switch to list view by tapping **I**.

• Using the context menu

To access the context menu, tap to select an entry, then tap on •••. In the patient-grouped results list, long-press on the patient header to open the context menu (tapping on the header displays the list of associated documents instead).

• Filtering search results

You cannot filter the search results list further. To receive fewer results, please refine your search query.

• Navigating through tagged instances

To view keyword details in the instance preview or the Viewer, tap the marking (dot) once. To jump to an instance, tap the dot twice.

• Dragging and dropping files

To drag and drop series/files from the **Document Navigator** or **Patient Jacket** (see Section 2.8.2 and Section 2.8.3), drag the thumbnail to the desired location.

• Moving viewports

To move a viewport, long-press on its specific toolbar and move it to the desired location.

• Selecting desired angle

On mobile devices the dashed helper lines are always visible. Tap on the desired angle to select it for calculation and to set it as the new default angle.

Deleting annotations

To delete a selected annotation, tap on 👕.

• Finishing a polygon annotation

To finish a polygon annotation, tap the checkmark button \checkmark in the toolbar.

4.2 Touch Gestures

The viewer supports several touch gestures for easier viewing:

• Two-finger swipe

Scroll by swiping two fingers up or down.

• Pinch

Zoom in or out by pinching/unpinching your fingers.

• Three-finger swipe

Pan an image by swiping three fingers in the desired direction.

In addition, the following gestures are available specifically for whole slide images :

• Two-finger swipe

Change the focal plane by swiping two fingers up or down.

• Pinch

Change the zoom layer by pinching/unpinching your fingers.

• Three-finger swipe

Pan an image by swiping three fingers in the desired direction.

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İthalatçı ve Türkiye Yetkili Temsilcisi Siemens Healthcare Sağlık A.Ş. Yakacık Caddesi No 111 34870 Kartal İstanbul Türkiye © ITH icoserve technology for healthcare GmbH / Siemens Healthcare GmbH 2024. All rights reserved.
Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen Germany Phone: +49 9131 84-0 siemens-healthineers.com
ITH icoserve technology for healthcare GmbH Innrain 98 6020 Innsbruck Austria Phone: +43 512 89059